

November 22, 2022

Technical Bulletin: Updates to the ORTHO VISION® and ORTHO VISION® Max Reference Guide

Dear Valued Customer,

This Technical Bulletin provides important information regarding upcoming updates being made to the ORTHO VISION® and ORTHO VISION® Max Reference Guides.

Updating Reference Guides (At a later date)	Publication Number
ORTHO VISION® ID-MTS™ Gel Card	J40050
ORTHO VISION® Max ID-MTS™ Gel Card	J55656

The next version of the Reference Guides will incorporate the updates mentioned in the Technical Bulletin.

Technical Bulletin Summary

This Technical Bulletin discusses planned improvements to **Chapter 3 System Components – Section: Imaging System** of the Reference Guide to better describe the instrument operations that occur during the card pre-processing quality check, and a technical clarification on the error code, CIMS72 Wrong Fluid Volume (WFV), that is reported if the system detects incorrect supernatant and/or gel level(s) during the card pre-processing quality check.

Obtaining Technical Bulletin

The Technical Bulletin can also be found on our Technical Documents Library at <https://techdocs.ocdx.com>.

Note: Users must be logged in to view.

Technical Bulletin	Publication Number
Updates to the Imaging System and Results Flags and Codes	J68799

**General
Awareness
for VISION
Users**

The VISION analyzers will reject Gel Cards if the Imaging System cannot detect appropriate liquid and/or gel layers and/or if other unacceptable artifacts are present.

The VISION analyzers were not designed to detect liquid drops above the liquid layer and/or in the upper reaction chamber, previously referred to as splashing in the ID-MTS Interpretation Guide.

**Contact
Information**

If you have questions, please contact our Ortho Care™ Technical Solutions Center at 1-800-421-3311.

Enclosure: Technical Bulletin (J68799)