


PROFICIENCY SAMPLES WHOLE BLOOD GLUCOSE

1. Do NOT shake the samples – hold vial horizontally between the palms of the hands and roll the vial back and forth 20-30 seconds. Continue to mix by inverting the specimens until the red cells are completely re-suspended. Then gently invert 8-10 times immediately before sampling.
2. Biohazard warning: treat as potentially infectious and capable of transmitting disease.
3. **Please split the samples between several operators. One operator should NOT test all 5 samples. PLEASE DO NOT DELAY TESTING.**
4. Test survey samples by pressing ▶ at the Main Menu and choosing “Proficiency.” **Enter by hand the ID on the vial ie. 01, 02 etc. There are no bar codes for the vials.** Wipe the tip of each vial before and after dosing of the strip. No dried material should be present.

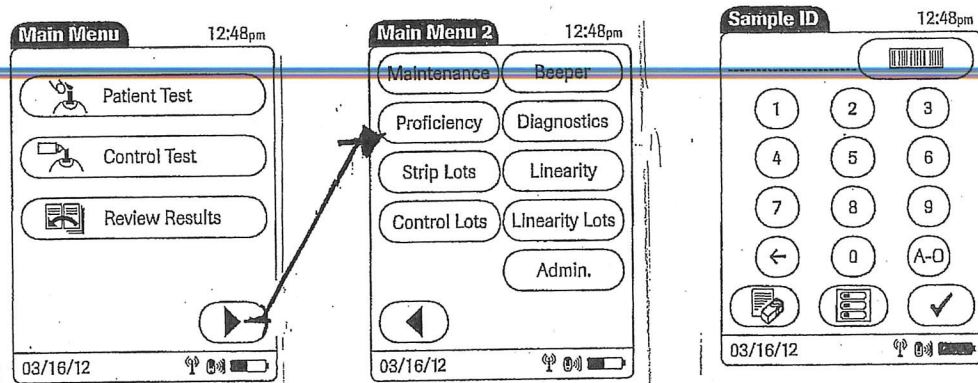

- 1 Touch ▶ in the *Main Menu* to open the *Main Menu 2* screen.
- 2 Touch *Proficiency* to start the proficiency test.
- 3 You will now be asked to enter the sample ID.
Use the keypad to manually enter the sample ID.

5. ******Sign last page in the spaces provided for testing personnel****or risk being locked out of further patient testing.**

ATTACHMENT B