



KAISER PERMANENTE®

DOCUMENT NUMBER: SCPMG-PPP-0166
DOCUMENT TITLE: BD Eclipse Blood collection Needle
DOCUMENT NOTES:

LOCATION: SCPMG-rel	VERSION: 01
DOC TYPE: Preanalytic	STATUS: Release

EFFECTIVE DATE: 17 Aug 2017	NEXT REVIEW DATE: 17 Aug 2019
RELEASE DATE: 17 Aug 2017	EXPIRATION DATE:

AUTHOR: K053260	PREVIOUS NUMBER:
OWNER: RRL ED Consultant	CHANGE NUMBER: SCPMG-CR-0373

BD Eclipse® Blood Collection Needle

Introduction

This procedure provides instructions for using the Becton Dickinson Eclipse® Blood Collection Needle. The Eclipse blood collection needle is designed to meet the requirements for engineered safety sharps. The Eclipse needle has a pink safety shield that is locked onto the needle using a one handed technique after removing the needle from the vein. The purpose of this needle is to reduce the incidence of accidental needle-stick injuries.



Scope

This procedure is intended for all users of blood collection devices.

Policy

Always engage the needle safety device when the venipuncture has been completed.

Safety

Refer to the safety manual for general safety requirements.

Procedure

Follow the steps below to use the Eclipse Blood Collection Needle:

Step	Action
1	Open the needle cartridge by holding both colored shields and twist to break the seal. Remove the white shield. The threaded hub with the rubber boot is now exposed.
2	Attach the needle to the needle holder by inserting the needle into the holder and turning the hub counterclockwise until it is securely attached.
3	Rotate the pink safety shield back towards the needle holder.
4	Twist and pull the green needle shield straight off without touching the needle itself.
5	Perform phlebotomy as outlined in the venipuncture procedure.

continued

Kaiser Permanente
Medical Care Program
California Division – South

SCPMG Laboratory Systems
Pre-analytical Processing
Procedure

Eclipse® Blood Collection Needle, continued

6	Remove the needle from the arm when all tubes have been collected.
7	Firmly lock the safety shield in place by pressing it down with the thumb until you hear a click and inspect.
8	Dispose the needle and holder in a Sharps container. NOTE: Do not unscrew the needle from the holder.

Non-Controlled Documents The following non-controlled documents support this procedure.

- Becton Dickinson Eclipse Blood Collection Needle Package Insert

Rebecca Rosser
Pre-analytical Processing Work Group

Author

Signature Manifest

Document Number: SCPMG-PPP-0166

Revision: 01

Title: BD Eclipse Blood collection Needle

All dates and times are in Pacific Standard Time.

BD Eclipse Collection Needle

Initial Approval

Name/Signature	Title	Date	Meaning/Reason
Janice Wolf (K119893)	Operations Director	16 Aug 2017, 09:11:51 AM	Approved

Final Approval

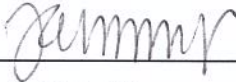
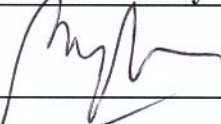
Name/Signature	Title	Date	Meaning/Reason
David Quam (P092597)	Rgnl Mg Admn-Pmg Executive	16 Aug 2017, 10:06:46 AM	Approved

Set Effective Date

Name/Signature	Title	Date	Meaning/Reason
Matthew Jones (F754627)	Systems Consultant		
Laura Perry (S533438)	Admin Spec II	17 Aug 2017, 04:14:11 PM	Approved

BD Eclipse® Blood Collection Needle

Reviewed and approved by (for Medical Center Area Approval Only):

SIGNATURE	DATE
	5.10.18
Name: <u>Janice M. Wolf</u> Operations Director, Area Laboratory	
	5/10/18
Name: <u>Sony Wirip, MD</u> CLIA Laboratory Director	
