Document History Page

Change	Changes Made to	Signature	Lab Operations	Loborotoni	Doto obones
type: New, Major, Minor etc.	Document – describe	Signature responsible person/date	Lab Operations Director Review/Date	Laboratory Medical Director Review/ Date	Date change implemented
Major	LCS2108-B assigned to Form MEDTOXScan Urine Drugs of Abuse Panel Manual Log for Meter#2 Name of form and from number designation appears in the footer section of the form Process or procedure associated with for is Profile®-V MEDTOXScan® Sample Porcessing for Drugs of Abuse Test System LCS 2108.	R.Castaneto 08-23-16	J.Wolf 08-26-16	S.Wirio M.D. 08-26-16	08-26-16
Major	Form updated to reflect Drug Confirmation for Triage Kit. Form LCS 2108 -C.	8-7-18	8-4-18	81816 June	8/13/13
			-		

Quest Confirmatory Testing for Positive Barb or Benzo Screens:

Update on Drug Confirmation for Triage Kit (80306C)

- Effective May 15, 2017, there is no need to confirm positive Barb, Benzo, Coc, Opiate, THC and TCA urine drug screen results from DRUG SCREEN, URINE, TRIAGE KIT [KPHC: 80306C/KRMS: 8410600]
 except for children under the age of 14 years and pregnant women or when requested by the provider.
- All positive screen results for Amphetamine, Methamphetamine and PCP will
 continue to be automatically confirmed by another alternative method, without
 an additional order from the provider. Please send all positive screen results to
 SWL for confirmation.
- There will be no changes to the testing procedure.
- There are no changes to the LRR codes (KP Health Connect Codes)
- Urine drug screen results are for medical purposes, Chain-of-Custody is not monitored.

Tech:

- For all other confirmatory testing, release results, reprint label and take to Separations. Reflexed tests go to RRL. Cerner will reflex the confirmatory testing unless Cerner is down.
- For children under the age of 14 years and pregnant women or when requested by the provider (See above update):

0	Reprint patient's label and place below. Give form and urine specimen to					
	accessioner.	DO NOT ALIQU	JOT SAMPLE.			
				6		

Accessioner:

Please accession test indicated by tech.

Drug Detected by Screen	Confirmatory Test	
Amphetamine or Methamphetamine	AMPHETAMINE, METHAMPHETAMINE, CONFIRM [82145D]	
PCP	PCP (PHENCYCLIDINE) URINE, CONFIRMATORY [80102ZF]	
Barbiturates 1	BARBITURATES, URINE, CONFIRMATION [82205M]	
Benzodiszepines ²	BENZODIAZEPINES, URINE, CONFIRMATION [80154U]	
Cocaine	COCAINE METABOLITE, URINE, CONFIRMATORY [80102H]	
Opiates ²	OPIATES, URINE CONFIRMATORY [80102F]	
THC ^{2,3}	CANNABINOIDS, URINE, CONFIRMATORY [80102A]	
TCA	CONFIRMATION NOT APPLICABLE (N/A)	