

## IRIS IQ200 AUTOVALIDATION SUSPENSION

**Purpose** This procedural document provides instructions on how to initiate and suspend autovalidation and autoverification in Iris IQ200 Urinalysis System.

**Policy** Autovalidation and autoverification should be suspended anytime that a component of the system or any of the algorithms is found to be out of order.

**Scope** All laboratory personnel who work on systems with autovalidation/autoverification capabilities have the access and right for this document usage whereby circumstances need them to for job duties.

**Workplace Safety** N/A

### Procedure:

#### Suspension of Autovalidation

Step	Action
1.	Suspend autovalidation if: <ul style="list-style-type: none"> <li>▪ QC results are unacceptable even after thorough investigation and corrective action</li> <li>▪ Instrument malfunction</li> <li>▪ Failed calibration</li> <li>▪ Reagent problem</li> <li>▪ Any other situation where instrument results are suspected to be adversely affected, e.g. high temperature in the lab</li> <li>▪ Consult lead CLS/Supervisor/Iris Hotline if unsure</li> </ul>
2.	To suspend autovalidation on IQ200 <ul style="list-style-type: none"> <li>▪ Log on to IQ200 computer as Manager</li> <li>▪ Click on Instrument</li> <li>▪ Go off line</li> <li>▪ Click on Settings</li> <li>▪ Click on Auto-Release</li> <li>▪ Uncheck Enable Auto-Release</li> <li>▪ Click on OK</li> </ul>
3.	Verify that autoverification is successfully turned off by running a previously known negative patient accessioned with a non-patient Health Connect test MRN and verify that results are not auto released. Document on daily checklist.

## IRIS IQ200 AUTOVALIDATION SUSPENSION

	Step	Action
<b>Re-initiation of Autovalidation</b>	1.	Re-initiate autovalidation if: <ul style="list-style-type: none"><li>▪ QC results are acceptable</li><li>▪ Instrument problem has been corrected</li><li>▪ Calibration has passed</li><li>▪ Reagent problem has been solved</li><li>▪ Any other situation has been corrected and instrument results are now acceptable</li><li>▪ Consult lead CLS/Supervisor/Iris Hotline if autovalidation/autoverification can be re-initiated</li></ul>
	2.	To re-initiate autovalidation on IQ200 <ul style="list-style-type: none"><li>▪ Log on to IQ200 computer as Manager</li><li>▪ Click on Instrument</li><li>▪ Go off line</li><li>▪ Click on Settings</li><li>▪ Click on Auto-Release</li><li>▪ Check the Enable Auto-Release box</li><li>▪ Click on OK</li></ul>
	3.	Verify that autoverification is successfully turned on by running a previously known negative patient accessioned with a non-patient Health Connect test MRN and verify that results are auto released. Document on the daily checklist.

### Non-controlled

**Documents**      IQ-200 Operators Manual, Set-up section, 300-4950 Rev D, 07/2004

---

**Author:**            Maaret Toprakci MS, MT(ASCP), Nanette Taft, CLS(ASCP)