

CLINICAL ALERT

Processing Trichomonas Update

- Dearborn Processing is to pour over any urine cup samples submitted for Trichomonas into the Urine Cobas tube. **Fill to between the two lines:**



- The processor will order the test and then deliver the Cobas Trich sample to the Trich Send out bucket.
- The decanter will then place all Trichomonas samples on a **room temp** packing list and send to **Royal Oak**.
- If the Trich and Chlamydia / Gonorrhea (ICNG) sample is shared- all tests will be placed on a packing list and sent to Royal Oak for testing.