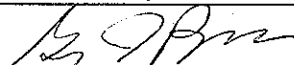
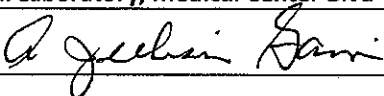
	Oxyhemoglobin and Total Hemoglobin Measurement Using the AVOXimeter 1000E Analyzer PRO-POCT-LAB-21	Dept:	Point of Care Testing
		Effective Date:	12/2003
		Revised Date:	03/2013
		Contact:	Angie Thayer, BSMT (ASCP) Clinical Laboratory Section on Point-of-Care Testing
Greg Pomper, MD, Clinical Laboratory Medical Director, Point-of-Care Testing		Date Approved	3/21/13
Signature/Date:  3/21/13			
A. Julian Garvin, MD, PhD CLIA Director, Clinical Laboratory, Medical Center Blvd			
Signature/Date: 			

I. General Procedure Statement:

The Clinical Laboratory at Wake Forest Baptist Medical Center (WFBMC) is responsible for oversight of explicitly identified non-waived laboratory testing performed in clinical areas by non-lab personnel. Specific testing sites have been identified and included under the CLIA certificate of the WFBMC Clinical Laboratories for a highly complex lab. Testing policies and procedures must meet all regulatory guidelines established by CLIA and other accrediting agencies, as applicable.

This is to ensure that all testing is performed according to manufacturer's recommendations and that each employee follows the same quality control and patient testing procedures. All AVOX testing should have a documented physician order.

The AVOX 1000E test device is used to measure the fractional oxyhemoglobin (%HbO₂), the total hemoglobin (THb) concentration, and the oxygen content in a sample of whole blood. Examples of situations where the physician may use the AVOX results are: 1) To help in identifying any possible shunts, 2) Verify oxygenation status of Cardiac Cath patients, and 3) Measurement of total hemoglobin (THb) in Cardiac Cath Lab patients.

A. Scope:

This document establishes procedures and guidelines for assuring quality of Point-of-Care Testing (POCT) results obtained from the AVOXimeter 1000E test device. Non-Waived POCT sites covered by the Clinical Laboratory CLIA certificate shall adhere to processes outlined in this document.

1. Responsible Department/Party/Parties:

- a. Procedure Owner:** Clinical Laboratory Point-of-Care Testing Manager/Coordinator
- b. Procedure:** Non-Waived POCT sites covered by the Clinical Laboratory CLIA certificate shall adhere to processes outlined in this document. Testing personnel have been identified as follows: Cardiac Cath Lab, Cardiovascular Specialists/Staff Nurses. If additional test sites are added in the future, they will also be required to adhere to processes outlined in this document.
- c. Supervision:** The Medical Director for Point-of-Care Testing shall supervise the person(s) performing activities outlined in this document.
- d. Implementation:** Each applicable POCT site manager is responsible for ensuring compliance with processes stated in this document.

II. Definitions:

- A. Point-of-Care Testing (POCT)** defined as tests designed to be used at or near the site where the patient is located, that do not require permanent dedicated space, and that are performed outside the physical facilities of the clinical laboratory.
- B. Clinical Laboratory Improvement Amendments (CLIA)**—United States federal regulatory standards that apply to all laboratory testing performed on humans.
- C. Quality Control (QC)**—processes to ensure the test system is performing as expected.
- D. Quality Assurance (QA)**—a system for ensuring a desired level of quality. The POCT program incorporates activities to monitor the quality of processes and the test system.
- E. Proficiency Testing (PT)**—Unknown samples sent to a lab/test site by a Centers for Medicare and Medicaid Services (CMS) approved PT program.
- F. Functional Oxygen Saturation** is the oxygenated hemoglobin divided by the sum of the oxygenated hemoglobin and the de-oxygenated hemoglobin. Carboxyhemoglobin, methemoglobin, sulfhemoglobin or other dyshemoglobins are not part of the equation and as a result can introduce error if they are present. The AVOX 1000E does not report functional hemoglobin.
- G. Fractional Oxygen Saturation (%HbO₂)** is the fraction of the total hemoglobin (THb) that is oxygenated. Fractional O₂Hb is sometimes referred to as FO₂Hb. Although the AVOX 1000E does not report the fractions of carboxy- and methemoglobin, they are measured and used in the measurement/calculation of % HbO₂.

III. Principle

Blood samples are injected into a sealed, single-use disposable cuvette that is inserted into the sample chamber. The AVOX 1000 E then illuminates the sample at multiple wavelengths, records the optical density of the sample at each of the wavelengths, and computes the results. Fractional Oxyhemoglobin (%HbO₂), Total Hemoglobin, and Oxygen Content in the sample are measured according to Beer's Law and the following calculations.

- A.** $\% \text{HbO}_2 = \text{O}_2\text{Hb} / (\text{Hb} + \text{O}_2\text{Hb} + \text{COHb} + \text{MetHb}) \times 100$
- B.** $\text{THb} = \text{O}_2\text{Hb} + \text{Hb} + \text{MetHb} + \text{COHb}$
- C.** $\text{O}_2 = 1.39 \times \text{THb} \times \% \text{O}_2\text{Hb} / 100.$

IV. Procedure

A. Equipment

1. AVOX 1000E Test Device
2. **SINGLE USE** AVOX Disposable Cuvettes (P/N C-100B)
 - a. Stored at room temperature (15-30°C) in tightly sealed bag with desiccant.
 - b. 1 required per sample
 - c. **DO NOT USE FOR PATIENT TESTING IF DESICCANT INDICATOR IS PINK!!! Patient results may be inaccurate.** See section, Sample Precautions/Limitations, for details to reverse moisture exposure.
 - d. **The CORRECT cuvette path length must be entered into the analyzer prior to patient or quality control (QC) testing.**
3. 5 cc syringe with blunt needle or safety needle
4. Gloves—ALWAYS wear gloves whenever collecting samples, testing QC or patient samples, or handling the AVOX analyzers.
5. Face Protective Equipment—is provided at user site
6. Tissues or 4X4 gauze
7. AVOX 1000E Quality Control (QC) filters (Two levels: yellow and orange) (**Analyzer specific**)
8. RNA Medical CC 527 Liquid QC Levels 1 and 3
 - a. Stored at 2-8 C until expiration date
 - b. May be stored at room temperature (up to 25C) for 12 months, provided the labeled expiration date is not exceeded.

- c. Avoid freezing and temperatures above 30C.
 - d. Call 1-800-533-6162 for order information.
9. Cuvettes and QC materials should be marked with the date they are received.

B. Reagents

There are no reagents with this system

C. Specimen Collection and Specifications

1. **Patient Preparation:** None required

2. **Specimen Type:** 50 µl of fresh whole blood-analyzed within 3 minutes of collection.

3. Instructions for Sample Collection

- a. Prior to collecting any blood sample, the patient's identity should be verified by using 2 patient identifiers, the name and medical record number, per WFBMC policies, PPB-WFBMC-14 and PPB-WFBMC-83.
- b. If testing is performed away from the patient's bedside, the blood sample should be identified at the AVOX test device so there is no question as to the sample identity. (Label syringe with patient ID sticker)
- c. Whole blood samples, which are obtained from an indwelling catheter, should be collected after sufficient waste has been drawn to clear the line (approximately 5 mL). Patient results may be compromised if catheter lines are not adequately cleared prior to sample collection.
- d. Testing site departmental guidelines should be followed for specimen collection procedures.
- e. In the **Cardiac Cath Lab** the physician will collect the blood sample and give to the Cath Lab staff to perform sample testing on the AVOX device. Sample testing should be performed within 3 minutes of collection.
- f. If testing will be delayed more than 3 minutes, the sample collection syringe should be heparinized.
 - (1.) Place a **small drop** of heparin into the syringe(s).
 - (2.) Push the plunger forward to remove any excess heparin. The syringe is now ready to collect the blood sample.
 - (3.) **Due to the small volume of blood, care should be taken to evacuate all excess heparin from the syringes. Results can be affected by too much heparin.**

4. Sample Precautions/Interferences/Sample Rejection Criteria

- a. Also, see section on Limitations of Procedure for additional information
- b. **Cuvettes exposed to moisture (desiccant indicator is pink)** can affect results and should not be used. The process can be reversed if the cuvettes are stored with a new desiccant pouch for 24 hours in a tightly sealed container.
- c. **Over-filled cuvettes** should be avoided.
- d. **Cuvettes should be inserted into AVOX analyzer within 30 seconds of filling.**
- e. **ONLY Fresh whole blood samples** should be used for testing.
- f. **Air contamination** will adversely affect results.
- g. **Excess heparin or saline** dilutes and falsely lowers the hemoglobin of the sample. Allowing the sample to be diluted with saline could possibly oxygenate the sample. Inaccurate results could be obtained.
- h. **Air Bubbles** can cause the %HbO₂ (oxyhemoglobin) fraction of the sample to be elevated by 1-2%. Always point the syringe and cuvette at a downward 45-degree angle to prevent oxygenation of the sample. Visible air bubbles should be evacuated before the sample is capped.
- i. **Delayed sample testing:** %HbO₂ measurements are most accurate if performed within 10 minutes of specimen collection. If samples are not tested within 10 minutes of collection, a new sample should be collected.
- j. **Clotted samples** should not be used for %HbO₂ monitoring on the AVOX device. If a sample will not be analyzed within 3 minutes, the syringe should be coated with heparin to prevent clotting.
- k. **Samples that have settled out and the red cells have separated from the plasma can give erroneous results. Samples should be well mixed prior to sample analysis.**

l. Specimens drawn with other anticoagulants besides sodium or lithium heparin **should NOT** be used for testing on the AVOX. Citrate is known to change the pH of the blood and cause errors in spectrophotometric measurements. Similarly, fluoride/oxalate should be avoided.

m. Lipid (fat) particles in a blood sample may distort the normal light scattering causing erratic readings.

5. Sample Handling

- a. All samples should be treated as biohazard substances
- b. Gloves should be worn when handling or collecting blood specimens and during sample analysis.
- c. When testing is completed, the sample syringe and cuvette should be discarded in an appropriate biohazard container.

D. Interfering Substances

Refer to above section, Sample Precautions, for interfering substances. Also refer to the AVOX Operator's Manual for additional details.

E. Sample Analysis for %HbO₂/THb

1. Review QC logs to ensure instrument is ready for use.
2. Gloves should be worn during sample collection and analysis.
3. Face shield protection is provided.
4. Turn on the AVOX by pressing the Enter/On key and confirm that the self-test was successful. The instrument indicates that it is ready to analyze samples by displaying the "READY SCREEN".
5. Verify that the correct cuvette path length is programmed in the AVOX. If not, refer to section Cuvette Calibration for details.
6. Open the sealed bag to remove cuvettes.
 - a. **Immediately reseal bag.**
 - b. Do not touch the sampling area (the clear surface) of the disposable cuvettes.
 - c. Handle cuvettes by black cap.
7. Collect the sample as defined in section Sample Collection.
8. Evacuate any air bubbles that may be in the sample.
9. If necessary, gently tap the syringe to knock any air bubbles loose***Immediately cap the sample to prevent air contamination***
10. Keep the syringe containing blood tightly sealed and roll syringe between palms to keep red blood cells and plasma well mixed.
11. Remove the cap from the end of the syringe.
12. Squirt out the first few drops of sample to check for clots and to get the well mixed portion of the sample.
13. Attach the cuvette to the Luer tip of the syringe, point the cuvette downward at a 45° angle, and observe the sample chamber of the cuvette.
14. Press the plunger gently and fill the cuvette. Stop injection of the sample as soon as blood reaches the air vent. Do not allow any air bubbles to adhere to the internal surface of the cuvette.
15. **Caution:**
 - a. Never force blood into the cuvettes. If the sample does not flow easily, discard the cuvette and use another one.
 - b. **When filling the cuvette, do not use excessive pressure or cause the vent patch to bulge outward.**
 - c. Cuvettes must be kept dry at all times. Verify that the desiccant indicator is NOT pink. **Do not use if the indicator is pink.** Before analyzing samples, take cuvettes from the Ziploc bag. Only take as many as needed for immediate use. Promptly re-seal the zip-lock bag containing cuvettes and desiccant.
16. Observe cuvette closely, make sure that the light path at the widest portion of the sample chamber is completely filled with blood and that blood reaches the air vent at the opposite end of the cuvette. Proper technique for filling cuvettes is illustrated in the AVOX Operator's / Service Manual.
17. Note: If any blood is on the exterior surface of the cuvette, wipe it off with gauze to avoid contamination of the instrument optics. If adequate sample allows, it is best to fill another cuvette and discard any cuvette that has a contaminated exterior.

18. With syringe still attached to the cuvette, hold the cuvette firmly by its black cap, and insert the cuvette into the sample chamber.
 - a. **Do not depress the plunger while the syringe is in the instrument.**
 - b. **The filled cuvette should be inserted in the AVOX analyzer within 30 seconds of fill.**
 - c. **DO NOT INJECT THE BLOOD INTO THE INSTRUMENT'S SAMPLE CHAMBER.**
19. Observe the LCD display. Do not disturb the AVOX. Results should display within 10 seconds.
20. Record the %HbO₂/THb results on the patient's departmental specific flow sheet and remove the sample from the sample port. Results should be documented with the date, time, sample site, testing personnel, and analyzer serial number.
21. After the AVOX reports the results of the sample, withdraw cuvette and dispose of it. Record the results before removing the cuvette from the analyzer. The results are not displayed once the sample is removed. **Do not leave the cuvette in the instrument after the result is displayed and recorded.**
22. Measure additional samples, as needed.
23. Discard used cuvettes and syringes in an appropriate biohazard container.
24. **Cuvettes should not be re-used.**
25. Cuvettes should be kept clean, dry and free of contamination.

F. Calculations: N/A

(See the AVOX Operator's Manual if analyzer is to be used for Hemodynamic Computations.)

G. Reporting Results

1. See Sample Analysis section.
2. Cath Lab staff will not always report THb results from the AVOX 1000E. If a physician requests THb results, the result will be documented on the departmental specific flow sheet or department electronic record.
3. Respond according to departmental specific guidelines.
4. **Reference Ranges (Normal Range)**
 - a. **% HbO₂**: There is no reference range for % HbO₂. The result is dependent on the anatomical site from which the sample is taken. For example, arterial blood is typically 97% saturated with oxygen (% HbO₂= 97%) whereas, venous blood contains less oxyhemoglobin.
 - b. **THb**: Varies by age. See Exhibit 1
5. **Critical Values:**
 - a. Critical values represent an emergency condition and must be reported immediately to the patient's physician. AVOX test results performed in the Cardiac Cath Lab are reported directly to the physician. WFBMC policy should be followed for read back of critical values.
 - b. There are no defined critical values for %HbO₂.
 - c. **THb Critical Values:** < or = 6.0 and > or = 20.0. If a critical THb value is reported by the AVOX, the physician must be notified and results documented, even if the physician did not request a THb.
 - d. Unexpected critical values should be verified by appropriate repeat testing. Analyzer and cuvette performance should be verified by performing liquid QC and daily QC filter checks.
6. **Reportable Range:**
 - a. %HbO₂: 0-100%
 - b. THb: 4-25g/dL
 - c. THb results <4 or >25 will give an ERROR on the AVOX analyzer. Results outside of this range should be reported as "Out of Instrument Range."
 - d. If values outside of this range are required, samples can be sent to the Clinical Lab for testing.

H. Clerical Errors

If an error is discovered in identifying a patient sample or in reporting the results, the patient's physician should be notified immediately and repeat testing performed. Documentation on the site-specific patient care flow sheet should be clear as to the action which was taken.

I. Limitations of Procedure

When questionable or unexpected results are obtained, proper instrument and cuvette performance should be verified by performing quality control checks (QC). Appropriate repeat testing by the Clinical Laboratory should be performed when AVOX results are in question.

1. Poor technique and improper instrument settings can cause erratic results.
2. Do not use the AVOX device for patient reporting if all QC values are not within acceptable limits. Follow appropriate troubleshooting procedures as outlined in the AVOX Operator's Manual.
3. Cuvettes with smudges, bubbles, fingerprints, or blood smears can cause erroneous or erratic results and should be avoided.
4. Do not remove the sample before a reading is completed.
5. DO NOT reuse cuvettes.
6. DO NOT force the sample into the cuvette. Obtain a new sample and cuvette, as necessary.
7. **Total hemoglobin readings may be affected if the proper cuvette optical path length is not correctly entered into the AVOX.**
8. **Total hemoglobin readings may be affected if the cuvettes are not stored in an air tight container with a desiccant pouch.**

J. Records

Quality control records, maintenance and troubleshooting records should be kept current and updated at all times. QC records are kept for 2 years by the Clinical Laboratory and AVOX user site. Maintenance and troubleshooting records are reviewed and stored for the life of the instrument, plus 2 years. Record retention will be adjusted, as necessary, to be in compliance with regulatory authorities. When analyzers are not used for patient testing, it is acceptable to document "not in use" (NIU) on the QC records. QC documentation must be complete with QC values or NIU and testing personnel initials.

V. Quality Control/Quality Assurance (QA) Procedures

*****GLOVES SHOULD BE WORN WHEN PERFORMING ANY MAINTENANCE OR QC PROCEDURE ON THE AVOX DEVICE.*****

A. Maintenance/Troubleshooting/QA Activities:

1. Daily:

The AVOX 1000E is checked for calibration accuracy and optics cleanliness with the use of Quality Control Filters. See Daily QC in section below.

2. Weekly:

The instrument and cuvettes are checked for accuracy and calibration stability. See section on Liquid Quality Control for details.

3. Semiannually: Analyzer Comparison and Linearity Verification

The AVOX 1000E is compared against the Clinical Lab's Co-Oximeter every 6 months and linearity is verified by testing a linearity kit. See Exhibit 2 for the procedure.

4. As Needed: Clean the analyzer exterior with WFBMC-approved disinfecting agent.

5. **Troubleshooting:** See the troubleshooting section of the AVOX Operator's /Service Manual for illustrations and details. All instrument errors should be logged on the QC log and corrective actions documented.

*****NOTE: After any analyzer maintenance, calibration, or repair and prior to patient use, both levels of QC filters and liquid QC must be tested to verify analyzer/cuvette performance.**

6. Calibration Verification:

Calibration is verified daily with the optical filters and also weekly with the liquid QC.

B. Daily Quality Control Procedures--Quick Cal Check and Test for Spilled Blood

1. Every day of patient use:

- a. The operation and calibration of the AVOX 1000E is verified every day of patient testing with the use of two optical filters supplied with every AVOX analyzer.
- b. Each filter is marked with the serial number of the instrument for which it is intended and the range of acceptable values. **Do not use the filters from another instrument.**

- c. These filters are treated just like a sample cuvette.
- d. The QC readings of the filters are recorded on the AVOX Daily QC Log.
- e. **Note:** In addition to confirming that the AVOX is calibrated, the filters also assess the status of the optical system. If blood has contaminated the optics, the readings from the QC filters will not fall within the acceptable ranges. Consult the AVOX Operator's /Service Manual for detailed instructions on how to disassemble the unit and clean the optics. Call ITC Technical Support at 1-800-631-5945 for assistance, if needed.

f. Procedure

- (1.) Verify that the serial numbers match between the QC filters and the AVOX analyzer.
- (2.) Wipe off the cuvette-shaped yellow and orange optical filters.
- (3.) Observe that the "READY" message is displayed and insert the QC filter into the AVOX.
- (4.) Wait until readings appear on the display.
- (5.) Record results on the provided logs *and review for acceptability*.
- (6.) Repeat with second QC filter.
- (7.) Record second filter results *and review for acceptability*.

g. If BOTH THb and BOTH %HbO2 readings are within specified ranges, the AVOX passes both the Quick Cal Check and the Test for Spilled Blood.

h. However, if any one of the four values is outside its specified range, the AVOX fails these tests and **can NOT be used for reporting patient results until the problem has been resolved.**

i. Remedial Action:

- (1.) Verify QC filters are clean and match the instrument used.
- (2.) The most likely cause of failure is blood or other debris obscuring the light detector.
- (3.) Repeat filter testing before referring to the AVOX Operator's Manual for optics cleaning.
- (4.) If acceptable results are not obtained with repeat testing, contact Clinical Engineering and/or review the AVOX Operator's /Service Manual for detailed instructions on how to disassemble the unit and clean the optics.
- (5.) Additional quality checks must be performed if the unit is disassembled. Consult ITC Technical Support and the Clinical Laboratory Point-of-Care Testing Coordinator for guidance.
- (4.) ITC Technical Support: 1-800-631-5945

*****ALL QC VALUES and TROUBLESHOOTING ACTIONS SHOULD BE RECORDED FOR AN ACCURATE AUDIT TRAIL*****

C. Liquid QC (Calibration Verification)

1. Two levels of Liquid controls are performed:
 - a. **Weekly on all AVOX analyzers** to verify calibration stability and the accuracy of the cuvettes.
 - b. On each **new shipment of cuvettes**
 - c. Whenever the **cuvette path length is changed** in an analyzer.
 - d. Whenever analyzer or cuvette **performance is in question**.
 - e. **Post analyzer repair or calibration--prior** to use for patient testing.
 - f. Liquid QC performance should be rotated among all users.
2. If this calibration verification fails (any of the QC values falls outside of acceptable range), re-calibration of the AVOX 1000E may be required. See below for Remedial Action.
3. **The AVOX analyzer and cuvettes cannot be used for patient sample testing until all quality control values are within acceptable limits.**
4. If all readings are **within** the specified ranges, the AVOX **passes** the liquid QC check.
5. The liquid QC should equilibrate to room temperature for at least 4 hours prior to testing.
6. Before use, hold the ampule at the top and bottom (with forefinger and thumb) and shake for 10 seconds to mix the solution. Restore the QC material to the bottom of the ampule. Let the ampule sit for about a minute to ensure there are no micro bubbles present.
7. Using finger protection, **CAREFULLY** break open the ampule of control solution and **SLOWLY** withdraw 0.2 to 0.3 ml into a 1 ml syringe. Use a safety needle or a blunt needle.
 - a. **NEVER manually re-cap needles!**

- b. **EXTREME CARE SHOULD BE USED IN OPENING THE QC AMPULES.**
- c. To avoid accidental sharps exposure, the ampule top should be covered with gauze, tissue, or a protective plastic sleeve when opening.
- 8. **Avoid getting air bubbles in the syringe or AVOX results may be adversely affected.**
- 9. If used, re-sheath the safety needle and carefully remove needle from the syringe. Discard the needle in a biohazard sharps container.
- 10. Immediately place an AVOX cuvette on the syringe.
- 11. Fill the cuvette according to illustrations in the AVOX Operator's/Service Manual.
- 12. Observe that the "READY" message is displayed and insert the Control-filled cuvette, with syringe attached, into the AVOX.
- 13. Wait until readings appear on the display.
- 14. Record results on the AVOX Liquid QC log and review for acceptability.
- 15. Repeat above steps with the second level of liquid control.
- 16. After the AVOX reports the results, withdraw the cuvette and dispose of it. Do not leave the cuvette in the instrument after results are displayed and recorded.
- 17. **DO NOT perform patient testing if all QC values are not within acceptable limits.**
- 18. *****All QC values should be recorded for an accurate audit record.*****
- 19. **Remedial Action**
 - If a QC result is outside of acceptable limits, verify the following:
 - a. Verify that the correct cuvette path length is programmed in the analyzer.
 - b. Check the desiccant indicator in the cuvette package.
 - c. Ensure liquid controls have been stored and handled appropriately.
 - d. Repeat the same level of liquid control using a new ampule and a new cuvette from the same box of cuvettes.
 - e. If repeat results are within acceptable limits, then the analyzer and cuvettes may be used for patient testing.
 - f. If repeat results are still outside of acceptable limits, perform troubleshooting procedures.
 - g. Contact ITC Technical Support at 1-800-631-5945 or call Clinical Engineering for assistance if the remedial action steps do not resolve the problem.

D. Calibration

1. Instrument Calibration

The AVOX is calibrated at the factory and ready for use. The AVOX employs highly stable state-of-the-art light sources. Factory tests and extensive clinical use indicate that the AVOX easily maintains its calibration readings for two or more years. The most frequent cause of error messages or erroneous reading is not loss of calibration but contamination of the optical detector by spilled blood or other debris. AVOX Systems does not recommend re-calibration unless indicated by calibration verification or QC failures. Calibration verification is performed using the daily QC filters and two levels of liquid controls.

2. Cuvette Calibration

*****Each time a new bag of cuvettes is opened, the AVOX cuvette path length must be updated and/or verified. Follow instructions below.*****

There are instructions inside the box lid as well.

- a. While the AVOX displays "READY", press the Main Menu key.
- b. Select Option 1 to reach the Calibration Menu and Enter.
- c. Select 3 for cuvette path length and Enter.
- d. Observe the number displayed. If it matches the new cuvette path length, press Enter and skip to step j.
- e. When prompted, enter the AVOX path length value from the bag of cuvettes by backspacing (use the ← key) over the current value shown and type in the new path length indicated on the bag of cuvettes.
- f. Press Enter.
- g. The next screen displayed "Value (s) OK? [Y/N]" will be confirmation of the result entered. Press Yes or No as needed to confirm or correct.
- h. Once the number entered is confirmed as correctly entered, the AVOX will store a new calibration constant in non-volatile memory and use it in subsequent analyses.

- i. The message "Cuvette Path length Calibration: Complete Press any key" is displayed.
- j. Press the Cancel key repeatedly to return to the sample analysis "READY" screen.
- k. *****NOTE: Prior to patient use, after entering a new cuvette path length in the AVOX analyzer, both levels of QC filters and liquid QC must be tested to verify analyzer/cuvette performance**

E. Semi-Annual Patient Comparison and Linearity Verification--See Exhibit 2 for details

1. The user site is responsible for performing and documenting analyzer comparison and linearity verification every 6 months on each AVOX analyzer.
2. Two patient samples and a linearity kit are tested semi-annually. The linearity kit is intended to challenge the full reportable range for both THb and %HbO2. A patient sample is tested to validate comparability of results with the Clinical Lab Cooximeter.
3. Results are documented and submitted to the Clinical Lab for review.
4. The Clinical Lab's CO-Ox instrument will be used as the reference method for %HbO2 and THb results.
5. If patient comparison results do not match or linearity verification fails, re-calibration of the AVOX 1000E is required. (Call ITC Technical Support at 1-800-631-5945 for assistance in re-calibration.)

VI. Proficiency Testing

The AVOX Point-of-Care Testing (POCT) sites at WFBMC participate in a CMS-approved proficiency testing program. Users should follow instructions provided with the survey samples. The proficiency samples are rotated among different users, and there is no communication between the Clinical Laboratory and POCT sites prior to the deadline for submission of results to the proficiency provider. If proficiency survey sample results fail, the problem is investigated and resolved as necessary. E.g. Re-training, instrument performance evaluation, or survey sample handling. All proficiency failures are reviewed by the Point-of-Care Testing Medical Director.

VII. Employee Training and Competency

Each user of the AVOX should be trained prior to using the device for patient testing and should maintain updated competency records. Competency is assessed yearly by observation, quality control performance, or written test. Other methods may be used to assess competency, per direction from regulatory authorities. New employees are reassessed at 6 months post-training and then annually, thereafter. Any user who fails to meet the competency requirements will need to be re-educated on the use of the analyzer. The Clinical Laboratory Point-of-Care Testing Coordinator and user site manager maintains training and competency records.

VIII. Product Information

Technical Support: 1-800-631-5945

IX. Review/Revision/Implementation:

- A. Review Cycle:** Each 2 years
 1. All new policies/procedures/guidelines and those that have major revision must be reviewed/signed by the CLIA Laboratory Medical Director.
 2. Review/sign-off can be completed by the designated section Medical Director in the following circumstances:
 - a. Biennial review
 - b. Minor document revisions
- B. Office of Record:** Clinical Laboratory, Section on Point-of-Care Testing

X. Related Policies/Procedures/Guidelines:

- A.** PPB-WFBMC-LAB-775-18, "Non-Waived Point of Care Testing (POCT) Quality Management Plan and Quality Control/Quality Assurance Procedures"
- B.** PPB-NCBH-95, "Understanding of Responsibilities Between Testing Sites and the Clinical Laboratory for Point-of-Care Testing"

XI. Attachments:

- A. Exhibit 1**
Total Hemoglobin Reference Ranges (Normal Values)

B. Exhibit 2

AVOX 1000E 6-Month Analyzer Comparison/Linearity Verification Procedure

XII. Revision Dates:

Effective: 12/1/03

Revised:

12/05

07/08

3/2013—procedure renumbered with this version-- numbering changed from PPB-NCBH-LAB-775-21 to PRO-POCT-LAB-21

XIII. References

A. AVOX 1000E Whole-Blood Oximeter Compliance Guide, A-Vox Systems, Inc., 28267 Ruffian Drive, Fair Oaks Ranch, Texas 78015. Rev 3.98


B. AVOX 1000E Operator's & Service Manual, Whole-Blood Oximeter, A-Vox Systems, Inc., 28267 Ruffian Drive, Fair Oaks Ranch, Texas 78015. Rev 4.98E

C. AVOX NCCLS Formatted procedure-Internet version- A-Vox Systems, Inc., 28267 Ruffian Drive, Fair Oaks Ranch, Texas 78015. 09/22/03

D. RNA Medical liquid QC CC527 package insert, RNA Medical, 7 Jackson Road, Devens, MA 01432 Rev B1190.

E. AVOX 1000E Operator's Manual, Whole-Blood Oximeter, International Technidyne Corporation, 8 Olsen Avenue, Edison, NJ 08820. AP1001 11/07

Reviewed: _____	Date: _____
Reviewed: _____	Date: _____
Reviewed: _____	Date: _____
Reviewed: _____	Date: _____
Reviewed: _____	Date: _____
Reviewed: _____	Date: _____

	<p>Oxyhemoglobin and Total Hemoglobin Measurement Using the AVOXimeter 1000E Analyzer PRO-POCT-LAB-21</p> <p>Exhibit 1 AVOX 1000E Total Hemoglobin Reference Ranges (Normal Values)</p>	Dept:	Point of Care Testing
		Effective Date:	12/2003
		Revised Date:	03/2013
		Contact:	Angie Thayer, BSMT (ASCP) Clinical Laboratory Section on Point-of-Care Testing
Greg Pomper, MD, Clinical Laboratory Medical Director, Point-of-Care Testing		Date Approved	3/25/13
Signature/Date: <i>Greg Pomper</i> 3/21/13			
A. Julian Garvin, MD, PhD CLIA Director, Clinical Laboratory, Medical Center Blvd			
Signature/Date: <i>A. Julian Garvin</i> 3/25/13			

Analyte	Units	Reference Range (Normal Range)
Hemoglobin	g/dL	18y 14-18
MALE		12y 13-16
		6y 11.5-15.5
		2y 11.5-13.5
		6m 10.5-13.5
		3m 9.5-13.5
		2m 9-14
		1m 10-18
		14D 12.5-20.5
		7d 13.5-21.5
		1d 14.5-22.5
		0d 13.5-19.5
Hemoglobin	g/dL	18y 12-16
FEMALE		12y 12-16
		6y 11.5-15.5
		2y-7d same as male
		1d 14.5-22.5
		0d 13.5-19.5

Effective: 12/03

Revised: 3/2013—no reference ranges were changed. Document format updated and document renamed/renumbered with this revision from Appendix 1 PPB NCBH LAB 775 21 to Exhibit 1 PRO-POCT-LAB-21

Reviewed: _____ Date: _____

Reviewed: _____ Date: _____


Reviewed: _____ Date: _____

Reviewed: _____ Date: _____

Reviewed: _____ Date: _____

Reviewed: _____ Date: _____

PRO POCT LAB 21 AVOX Exhibit 1 031213

	Oxyhemoglobin and Total Hemoglobin Measurement Using the AVOXimeter 1000E Analyzer PRO-POCT-LAB-21 Exhibit 2 AVOX 1000E 6-Month Analyzer Comparison/Linearity Verification Procedure	Dept:	Point of Care Testing
		Effective Date:	12/2003
		Revised Date:	03/2013
		Contact:	Angie Thayer, BSMT (ASCP) Clinical Laboratory Section on Point-of-Care Testing
Greg Pomper, MD, Clinical Laboratory Medical Director, Point-of-Care Testing		Date Approved	3/21/13
Signature/Date: <i>Greg Pomper 3/21/13</i>			
A. Julian Garvin, MD, PhD CLIA Director, Clinical Laboratory, Medical Center Blvd			
Signature/Date: <i>A Julian Garvin</i>			

I. General Procedure Statement:

Linearity verification is used to confirm that the calibration of the test system has remained stable for the full reportable range. Analyzer comparison is used to verify that results reported by the AVOX correlate to results reported by the Clinical Laboratory co-oximeter.

II. Principle

Every 6 months an arterial and venous blood specimen is compared against the Clinical Laboratory Co-oximeter analyzer to verify correlation between the two different testing methodologies. Also, a linearity kit is used to verify analyzer linearity for full analytical measurement range (AMR). This validation is performed on all AVOX analyzers in use.

III. Procedure

A. Equipment/Supplies

1. At least 1 cc of arterial blood
2. At least 1 cc of venous blood
3. RNA Medical Linearity Kit CVC 223
4. Syringes (at least 3 cc) with blunt or re-sheathable safety needles
5. Paper towels, tissues, or 4x4 gauze
6. AVOX Disposable Cuvettes
7. AVOX 1000E

B. Procedure--Patient Sample Comparison

1. Perform required daily optical QC filter checks and record results.
2. Obtain a fresh venous heparinized blood sample with the % HBO2 in the 50 - 75% range. If possible, try to get total hemoglobin (THB) in the upper end of the reportable range.
3. Obtain an arterial heparinized blood sample with the % HBO2 in the 85 - 100% range. If possible, try to get the total hemoglobin (THB) in the lower range of the reportable range.
4. Test both samples on all AVOX analyzers and then test specimens on the Clinical Laboratory co-oximeter. Keep samples well mixed and tightly capped.
5. Measure and Record the AVOX %HBO2 and THB results on both samples on the AVOX Patient Comparison Log.

6. Submit results to the Clinical Laboratory Point-of-Care Testing Coordinator for review.

C. Procedure--Linearity Verification

1. Follow linearity kit package insert for handling instructions.
2. Test all 5 levels on each AVOX in triplicate. It is acceptable to use the same sample for triplicate testing.
3. Document results and verify that all values are within package insert ranges, specifically listed for the AVOX 1000E

D. Acceptable Limits

1. **Patient split samples** must match the Clinical Lab +/- 1g/dL for THB and +/- 3%HBO2 saturation.
2. **Linearity readings** must fall within the package insert acceptable limits.
3. If any values are not within acceptable limits, patient testing should be suspended on the AVOX until resolution is made.

IV. Review/Revision/Implementation:

A. Review Cycle: Each 2 years

1. All new policies/procedures/guidelines and those that have major revision must be reviewed/signed by the CLIA Laboratory Medical Director.
2. Review/sign-off can be completed by the designated section Medical Director in the following circumstances:
 - a. Biennial review
 - b. Minor document revisions

B. Office of Record: Clinical Laboratory, Section on Point-of-Care Testing

Effective: 12/03

Revised:

4/04

12/05

07/08

3/2013—document renamed/renumbered with this revision from Appendix 2 PPB NCBH LAB 775 21 to Exhibit 2 PRO-POCT-LAB-21

Reviewed: _____ Date: _____

Reviewed: _____ Date: _____

Reviewed: _____ Date: _____

Reviewed: _____ Date: _____

PRO POCT LAB 21 AVOX Exhibit 2 031213