



IMMUCOR, INC. Norcross, GA 30071 USA

US LICENSE NO: 886
LOT NO: 41815

EXPIRES: 24 Oct 02

PANOCELL - 16 MASTER LIST

NAME Mike Blue SEX M
 NO. 2613 INSTITUTION _____
 PREGNANCY _____ TRANSFUSION _____
 BLOOD GROUP _____
 ANTIBODY IDENTITY A+B-E

TECH AT DATE 1-28-03

	Special Type	Donor	Rh - Hr					Kell						Duffy	Kidd	Lewis	P	MN	Luh-	X ₉	PATIENT'S TEST RESULTS										
			D	C	E	e	Y	K ₁	K ₂	K ₃	K ₄	K ₅	K ₆	F ₁	J ₁	J ₂	L ₁	L ₂	P ₁	M ₁		S ₁	S ₂	X ₁₀							
1		R1R1 B4775	+	+	0	0	0	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	1	IS	37°	AHCC	SEL
2		R1WR1 B3581	+	+	0	0	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2	-	-	-	2+
3		R2R2 C2348	+	+	0	0	0	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	3	-	1+	2+	2+
4		Ror D619	+	+	0	0	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	4	-	-	-	2+
5		r'r E413	0	+	+	0	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	5	-	-	-	2+
6	[a]	r'r F463	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	6	-	W	1+	2+
7	[a]	r'r G488	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	7	-	-	-	2+
8		r'r H660	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	8	-	-	-	2+
9		R1R1 B4299	+	+	0	0	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	9	-	-	-	2+
10	Bga(+)	R1R1 B1258	+	+	0	0	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	10	-	-	-	2+
11		R2R2 C2884	+	+	0	0	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	11	-	1+	2+	2+
12		r'r N1804	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	12	-	-	-	2+
13	[13]	r'r N2029	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	13	-	-	-	2+
14		r'r H367	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	14	-	-	-	2+
15		r'r V127	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	15	-	-	-	2+
16	Y(h+)	r'r N1676	0	0	+	+	+	0	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	16	-	-	-	2+
PC		PATIENTS' CELLS																									PC				

DIRECT ANTIGLOBULIN TEST		ELUATE RESULT
POLY	IgG C3	
LOT		
RESULT		

Some examples of anti U may react weakly with the S+ cells above.

All cells are positive for I, Ga, Ya, T_a, Va, Loc, Db and negative for Hg, Vw, D₉, and Wta except where noted. * indicates those antigens whose presence or absence may have been determined using only a single example of a specific antibody.

In those instances where a patient's serum is known to contain anti D, it may be desirable to perform antibody screening tests with D+ red cells. The panel cells whose vial numbers are set off by brackets (II) can be used together to form a three- or four-vial D negative antibody screening reagent. All bracketed cells must be used to construct a complete screening reagent.

PATIENT'S SERUM:	
REVERSE GROUPING CELLS	PANO-SCREEN LOT
A1	III
A2	
B	



Ortho-Clinical Diagnostics

a Johnson & Johnson company

PATIENT NAME: Lucy Boyd

PATIENT ID: 469833

DATE: 1-28-03

TECH: dt

CONCLUSION: Anti-K, Anti-Lea

Cell 4 may exhibit variable reactivity with different examples of anti-Xpa.

cannot R/O E or Kpa

Panel A

Reagent Red Blood Cells
0.8% Resolve® Panel A
Antigram® Antigen Profile

Lot No. 8RA146

Exp. Date 27 AUG 2002

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

Cell#	Rh-ir	Donor Number	Rh-ir				KELL				DUFFY			KIDD			LEWIS			MNS			P		Special Antigen Typing	Test Results			
			D	E	c	e	K	Kpa	Kpb	Js ^a	Js ^b	Fya	Fyb	Jka	Jkb	Xga	Lea	Leb	S	M	N	P1	Luth ^a	Luth ^b					
1	R1wR1	114033	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	TS 31	2+	
2	R1R1	113925	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
3	R2R2	114456	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
4	Ror	114450	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
5	rr	113792	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
6	rT	100431	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
7	rr	112491	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
8	rr	112223	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
9	rr	109523	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
10	rr	108700	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
11	R1wR1	106011	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	2+	2+		
	Mode of Reactivity		37°C/Antiglobulin																					Variable		Cold	Var.		

i antigen status may have been determined presumptively based on Rh-ir phenotype.

Additional Cells	Rh-ir					KELL					DUFFY			KIDD			LEWIS			MNS			P		LUTHERAN	Special Antigen Typing	Test Results			
Rh-ir	Donor Number	D	C	E	c	e	f	Cw	V	K	k	Kpa	Kpb	Js ^a	Js ^b	Fya	Fyb	Jka	Jkb	Xga	Lea	Leb	S	M	N	P1	Luth ^a	Luth ^b		



Ortho-Clinical Diagnostics

a Johnson & Johnson company

*cannot r/o α CW, V, Kp^a, JS, α L^a
α K only on heterozygous*

PATIENT NAME: Brown, Charlie

PATIENT ID: 10010

DATE: 1/28/03

TECH: *dt*

CONCLUSION: Anti-C

Panel A

Reagent Red Blood Cells
0.8% Resolve® Panel A
Antigram® Antigen Profile

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

Lot No. 8RA151 Exp. Date 14 JAN 2003

Cells	Rh-hr Donor Number	Rh-hr							KELL										DUFFY					KIDD					LEWIS					MNS					P		LUTHERAN		Special Antigen Typing	Test Results	
		C	D	E	c	e	f	i	C ^w	V	K	K ^a	K ^b	J ^s	J ^b	F ^y ^a	F ^y ^b	J ^k	J ^k	X ^g ^a	X ^g ^b	Le ^a	Le ^b	S	S	M	N	P ₁	Lu ^a	Lu ^b															
1	R1W1	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	IS 37°	AMK		AMK						
2	R1R1	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	St 3+	St 3+	St 3+						
3	R2R2	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	—	—	—	—					
4	Ror	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	—	—	—	—					
5	r'	0	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	St 3+	St 3+	St 3+	St 3+					
6	r*	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	—	—	—	—					
7	r	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	—	—	—	—					
8	r	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	—	—	—	—					
9	r	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	—	—	—	—					
10	r	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	—	—	—	—					
11	R1R1	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+		Cell#	—	St 3+	St 3+	St 3+	St 3+					

* 1 antigen status may have been determined presumptively based on Rh-hr phenotype.

Additional Cells				Rh-hr					KELL					DUFFY					KIDD					LEWIS					MNS					P		LUTHERAN		Special Antigen Typing	Test Results
Rh-hr	Donor Number	D	C	E	c	e	f	C ^w	V	K	K ^a	K ^b	J ^s	J ^b	F ^y ^a	F ^y ^b	J ^k	J ^k	X ^g ^a	X ^g ^b	Le ^a	Le ^b	S	S	M	N	P ₁	Lu ^a	Lu ^b										
Cells																																							



Ortho-Clinical Diagnostics

a Johnson & Johnson company

*Camacho de la V, Kp a JS, JKo, Lea, Lk²
 + K only heterozygous*

PATIENT NAME: Sammy Davis
 PATIENT ID: 114-55-66
 DATE: 1/28/03 TECH: LW
 CONCLUSION: Anti-C, -S

Lot No. 8RA151 Exp. Date 14 JAN 2003

Panel Reagent Red Blood Cells 0.8% Resolve® Panel A Antigram® Antigen Profile

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

Cell#	Rh-ir	Donor Number	Rh-ir			KELL			DUFFY			KIDD			LEWIS			MNS			P ₁	P ₂	Lutheran	Special Antigen Typing	Test Results				
			B	C	E	Kp ^a	Kp ^b	Jsa	Jsb	Fya	Fyb	Jka	Jkb	Xga	Lea	Leb	S	M	N										
1	R1wR1	115107	+	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				S-C+	3+			
2	R1R1	115117	+	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				S-C+	3+			
3	R2R2	43344	+	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				S-C+	3+			
4	Ror	114771	+	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				S-C+	3+			
5	rr	61360	0	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				S-C+	3+			
6	r'r	115112	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				S-C+	3+			
7	rr	113149	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				C-St	3+			
8	rr	114241	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				C-St	3+			
9	rr	111163	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				C-St	3+			
10	rr	114712	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				C-St	3+			
11	R1R1	115114	+	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+				C-S	3+			
Mode of Reactivity			37°C/Antiglobulin															Antiglobulin			Variable			Cold			Var.		

* antigen status may have been determined presumptively based on Rh-ir phenotype.

Cell#	Rh-ir	Donor Number	Rh-ir			KELL			DUFFY			KIDD			LEWIS			MNS			P ₁	P ₂	Lutheran	Special Antigen Typing	Test Results
			D	C	E	Kp ^a	Kp ^b	Jsa	Jsb	Fya	Fyb	Jka	Jkb	Xga	Lea	Leb	S	M	N						