



PANOCELL® - 16
MASTER LIST

IMMUCOR, INC. Norcross, GA 30071 USA

US LICENSE NO: 886
LOT NO: 43860
EXPIRES: 07 Nov 02

LICENSE NO. 686



a Johnson & Johnson Company

Harsen, New Jersey 08863

PATIENT NAME: Bethelle Maeve Weather Lot No. RA325 Exp. Date 09 JUL 2002

PATIENT ID: L4823A DATE: TECH:

CONCLUSION:

Panel A

A

Reagent Red Blood Cells
Resolve® Panel A
Antigram® Antigen Profile

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

| Cell# | Rh-hr | Donor Number | Test Results | | | | | | | | | | | | | | | | | | | | | | Anti-D | | |
|-------|-------|--------------------|--------------|---|---|---|---|---|----|---|---|----------------|----------------|-----|-----------------|-----------------|-----|-----|-----------------|-----------------|------------------|---|---|---|----------------|-----------------|-----------------|
| | | | D | C | E | c | s | i | C" | V | K | K ^a | K ^b | Jsa | Fy ^a | Fy ^b | Jka | Jkb | Xg ^a | Xg ^b | Lew ^a | S | M | N | P ₁ | Lu ^a | Lu ^b |
| 1 | R1wR1 | 100542 | + | 0 | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | + | 0 | + | 0 | + | 0 | + | 0 |
| 2 | R1R1 | 114311 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 |
| 3 | R2R2 | 110793 | + | 0 | + | 0 | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | + | 0 | + | 0 | 0 | 0 | + | 0 | 0 | 0 | |
| 4 | Rx | 112353 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | |
| 5 | rr | 114160 | 0 | + | 0 | + | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | 0 | |
| 6 | rr | 114317 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | + | 0 | + | 0 | 0 | 0 | 0 | |
| 7 | rr | 111526 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | |
| 8 | rr | 114697 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 9 | rr | 112029 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 10 | rr | 113545 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 11 | RzR1 | 114306 | + | 0 | + | 0 | + | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | |
| | | Patient Cells | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | Mode of Reactivity | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | 37°C/Antiglobulin | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | Antiglobulin | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | 37°C/Anlglobulin | | | | | | | | | | | | | | | | | | | | | | | | | |

* I antigen status may have been determined presumptively based on Rh-hr phenotype.

| Cell# | Rh-hr | Donor Number | Test Results | | | | | | | | | | | | | | | | | | | | | | | Cell# | |
|-------|-------|--------------|--------------|---|---|---|---|---|----|---|---|----------------|----------------|-----|-----------------|-----------------|-----|-----|-----------------|-----------------|------------------|---|---|---|----------------|-----------------|-----------------|
| | | | D | C | E | c | s | i | C" | V | K | K ^a | K ^b | Jsa | Fy ^a | Fy ^b | Jka | Jkb | Xg ^a | Xg ^b | Lew ^a | S | M | N | P ₁ | Lu ^a | Lu ^b |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | |



Ortho-Clinical Diagnostics, Inc.
a Johnson & Johnson company

Ranitan, New Jersey 08869

Cell 7 of this test is despatched with an A.

PATIENT NAME: Jack Nichols

PATIENT ID: 12345/26

PATIENT ID: 02/25/03

DATE: 02/25/03 TECH: JG

CONCLUSION:

Panel A

Reagent Red Blood Cells Resolve® Panel A Antigram® Antigen Profile

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment

13

Lot No. RA305

Exp. Date 19 FEB 2002

| Cell # | Donor Number | Fh-hr | | | | | | | | | | | | Anti-IgM | | | | | | | | | | | | Test Results | | | | | | | | | | | |
|--------------------|--------------|-------------------|---|---|---|---|---|----------------|---|---|---|-----------------|-----------------------------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|---|---|---|----------------|-----------------|-----------------|---------------|---------------|---------------|---------------|---------------|---------------|---------------|---------------|---------------|---------------|
| | | D | C | E | C | e | i | C ^m | V | K | K | Kp ^a | J ^s Kp ^b | Fy ^a | Fy ^b | Jk ^a | Jk ^b | Xg ^a | Xg ^b | Le ^a | Le ^b | S | M | N | P ₁ | Lu ^a | Lu ^b | Cell 7 Status |
| 1 | R1wR1 | + | 0 | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | + | 0 | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 |
| 2 | R1R1 | + | 0 | 0 | + | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | |
| 3 | R2R2 | + | 0 | + | 0 | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 |
| 4 | Ror | 112726 | + | 0 | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 5 | r7 | 113023 | 0 | + | 0 | + | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 6 | r7 | 112550 | 0 | 0 | + | 0 | + | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 7 | r7 | 111965 | 0 | 0 | + | 0 | + | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 8 | r7 | 113798 | 0 | 0 | 0 | + | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | |
| 9 | r7 | 111595 | 0 | 0 | 0 | + | 0 | 0 | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | | |
| 10 | r7 | 113820 | 0 | 0 | + | 0 | + | 0 | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | | |
| 11 | R1R1 | 111586 | + | 0 | 0 | + | 0 | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | | |
| Patient Cells | | 37°C/Antiglobulin | | | | | | | | | | | | Anti-IgG | | | | | | | | | | | | Variable | | | | | | Var. | | | | | |
| Mode of Reactivity | | 37°C/Antiglobulin | | | | | | | | | | | | Anti-IgM | | | | | | | | | | | | Variable | | | | | | Var. | | | | | |

* If antigen status may have been determined presumptively based on Rh-hr phenotype.

| Additional Cells | Fh-hr | | | | | | | | | | | | Anti-IgM | | | | | | | | | | | | Test Results | | | | | | | | | | | | |
|------------------|-------|---|---|---|---|---|----------------|---|---|---|-----------------|-----------------------------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|---|---|---|----------------|-----------------|-----------------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|
| | D | C | E | C | e | i | C ^m | V | K | K | Kp ^a | J ^s Kp ^b | Fy ^a | Fy ^b | Jk ^a | Jk ^b | Xg ^a | Xg ^b | Le ^a | Le ^b | S | M | N | P ₁ | Lu ^a | Lu ^b | Cell 7 |
| | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |



PANOCELL® - 16
MASTER LIST

IMMUCOR, INC. Norcross, GA 30092 USA

卷之三

US LICENSE NO: 886
LOT NO: 43060
EXPIRES: 07 Nov 02

NAME Donald Parker SEX M
NO. INSTITUTION
PREGNANCY TRANSFUSION
BLOOD GROUP ANTIBODY IDENTITY

| TECH | DATE | PATIENT'S TEST RESULTS | | | | | | | | | | | | | | | | | | | | | | |
|------|--------------|------------------------|-------|----|---|---|---|------|----|---|----|---|----------------|----------------|----------------|----------------|----------------|----------------|------------------|------------------|---|----------------|-----|-----|
| | | Rh - Hr | | | | | | Kell | | | | | | Duffy | | | | | | | | | | |
| TEST | Special Type | Donor | | D | C | c | E | e | r' | V | C- | K | K _y | K _x | J _y | J _x | F _y | F _x | J _y ' | J _x ' | L _y | L _x | CEP | CEP |
| 1 | | RzR1 | A2180 | + | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | + | 1 | - | - | - | |
| 2 | | R1wR1 | B2262 | + | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 2 | - | - | - | |
| 3 | | R2R2 | C2888 | + | + | 0 | 0 | 0 | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 3 | - | - | - | |
| 4 | Bg(a+) | RoI | D451 | + | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | 0 | - | |
| [5] | | r'r | E294 | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 6 | - | - | |
| [6] | Bg(a+) | r'r | F170 | 0 | 0 | + | + | 0 | 0 | + | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 6 | - | |
| [7] | | rr | G679 | 0 | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | 0 | + | 7 | |
| 8 | | rr | HB49 | 0 | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 8 | |
| 9 | | R1R1 | B1085 | + | + | 0 | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | + | 0 | - | |
| 10+ | | R2r | J744 | + | 0 | + | + | 0 | 0 | + | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 10 | - | |
| 11 | | R1R1 | B4627 | + | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | + | 0 | + | 11 | - | |
| [12] | | rr | N2035 | 0 | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 12 | |
| 13 | | rr | N2037 | 0 | 0 | + | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | + | 0 | + | 0 | 13 | |
| 14 | | rr | G684 | 0 | 0 | + | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | + | 0 | + | 14 | - | |
| 15 | VS+ | r'sr | E166 | 0 | w | + | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 15 | |
| 16 | | rr | N1760 | 0 | 0 | + | 0 | + | 0 | 0 | + | 0 | 0 | + | 0 | 0 | 0 | 0 | + | 0 | 0 | 0 | 16 | |
| PC | | PATIENT'S CELLS | | | | | | | | | | | | | | | | | | PC | DIRECT ANTIGLOBULIN TEST | | | |
| | | ELUATE RESULT | | | | | | | | | | | | | | | | | | I | Some examples of Rh and Kell factors weakly with the S-1 cells shown. | | | |
| | | POLY | IgG | C3 | | | | | | | | | | | | | | | | II | All cells are positive for IgG, IgM, IgA, Vic, Caa*, Dta and negative for IgG, Vic, Dta, and Wta except where indicated. * indicates these antigens whose presence or absence may have been determined using only a single example of a specific antibody. | | | |
| LOT | | | | | | | | | | | | | | | | | | | | III | In these bottles where a patient's serum is known to contain IgG and IgM, it may be desirable to perform antibody screening tests with Dated cells. The panels used here (I, II, III) are not all the bottles (III can be used elsewhere to form a screen and I, II and III together will give a more extensive antibody or antigen screen). Antibodies which have been | | | |

