



Ortho-Clinical Diagnostics, Inc.

a Johnson & Johnson company

Raritan, New Jersey 08869

PATIENT NAME: Beatrice Neeli weather Lot No. RA925 Exp. Date 09 JUL 2002

PATIENT ID: 4418237

DATE: 1-31-03

TECH: K.J

CONCLUSION: Anti-Fyb

Panel A

Reagent Red Blood Cells
Resolve® Panel A
Antigram® Antigen Profile

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

Cell#	Rh-ir	Donor Number	Rh-ir										KELL								DUFFY				KIDD				LEWIS				MNS				P		LUTHERAN		Special Antigen Typing	Test Results
			C	E	e	g	i	C ^w	V	K	Kp ^a	Kp ^b	Js ^a	Js ^b	Fy ^a	Fy ^b	Jk ^a	Jk ^b	Xg ^a	Xg ^b	Le ^a	Le ^b	S	s	M	N	P ₁	Lu ^a	Lu ^b													
1	R1WR1	108642	+	+	0	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	+	+	+	+							
2	R1R1	114311	+	+	0	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	+	+	+								
3	R2R2	110798	+	+	0	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	0	+	+	+	+								
4	Ror	112363	+	0	0	+	+	+	+	0	+	+	+	0	+	+	+	+	0	+	+	0	+	+	+	+	+	+	0	+	+	+	+	+	D(e ⁺)							
5	r ₁	114160	0	+	0	+	+	+	+	0	+	+	+	0	+	+	+	+	0	+	+	+	0	+	+	+	+	0	+	+	+	+	+	+	Ⓣ							
6	r ₁	114317	0	0	+	+	+	+	+	0	+	+	+	0	+	+	+	+	0	+	+	+	0	+	+	+	+	0	+	+	+	+	+	+	Ⓣ							
7	r ₁	111526	0	0	0	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	0	+	+	+	0	+	+	+	+	+	+	+	+	Ⓣ							
8	r ₁	114097	0	0	0	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	+	+	+	+	Ⓣ							
9	r ₁	112029	0	0	0	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	+	+	+	+	+							
10	r ₁	113545	0	0	0	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	+	+	+	+	+							
11	R2R1 Patient Cells	114306	+	+	0	+	+	+	+	0	+	+	0	+	+	+	+	+	0	+	+	0	+	+	+	+	0	+	+	+	+	+	+	+	+							
Mode of Reactivity			37°C/Antiglobulin										Antiglobulin								Variable				Cold		Var.															

R/O C.E.K. J₁ & R

* antigen status may have been determined presumptively based on Rh-ir phenotype.

Also R/O C^w, V, Kp^a, J₁^a, Lu^a



Ortho-Clinical Diagnostics, Inc.

a Johnson & Johnson company

Raritan, New Jersey 08839

Cell 7 of this lot is designated with an A.

Order to run cells that are Fyb negative and positive for S, Xk^a, k, F, and G.
 10 Cannot make out CW

PATIENT NAME: Jack Nichols

PATIENT ID: 105126

DATE: 02/05/03 TECH: By

CONCLUSION: anti-Fyb shows negative

Lot No. RA905 Exp. Date 19 FEB 2002

Panel

Reagent Red Blood Cells
 Resolve® Panel A

Antigram® Antigen Profile

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

A

Cell#	Rh-ir	Donor Number	Rh-ir											KELL										DUFFY			KIDD			LEWIS			MNS			P			LUTHERAN		Special Antigen Typing	Test Results
			B	C	E	e	i	Cw	Y	K	k	Kp ^a	Kp ^b	Jsb	Jsa	Fyb	Fya	Jkb	Jka	Xk ^a	Xk ^b	Lgb	Lga	S	s	M	N	F ₁	F ₂	Lub	Lua											
1	R1W1	113818	+	+	0	0	+	0	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+									
2	R1R1	108347	+	+	0	0	+	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+								
3	R2R2	42222	+	+	0	+	0	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+							
4	Ror	112726	+	+	0	0	+	+	+	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+							
5	r ^r	113023	0	+	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+							
6	r ^r	112550	0	0	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+							
7	r ^r	111965	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+							
8	r ^r	113798	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+							
9	r ^r	111595	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+						
10	r ^r	113820	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+						
11	R1R1	111536	+	+	0	0	+	0	0	0	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+	+						

* 1 antigen status may have been determined presumptively based on Rh-ir phenotype.

Cell#	Rh-ir	Donor Number	Rh-ir				KELL			DUFFY			KIDD			LEWIS			MNS			P			LUTHERAN		Special Antigen Typing	Test Results															
			D	C	E	e	f	Cw	V	K	K	Kp ^a	Kp ^b	Jsa	Jsb	Fya	Fyb	Jka	Jkb	Xka	Xkb	Lga	Lgb	S	s	M			N	F ₁	F ₂	Lua	Lub										



Ortho-Clinical Diagnostics
 a Johnson & Johnson company

PATIENT NAME: Bernice Schwarz
 PATIENT ID: 032390 TECH: PS
 DATE: 1-29-03
 CONCLUSION: anti D anti C
cannot r/o anti Cu
& anti E - hetero only

Panel A
 Reagent Red Blood Cells
 0.8% Resolve® Panel A
 Antigram® Antigen Profile

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

Lot No. 8RA151

Exp. Date 14 JAN 2003

Cell#	Rh-hr	Donor Number	Rh-hr											KELL					DUFFY		KIDD		LEWIS		MNS		P		Special Antigen Typing	Test Results			
			D	C	E	c	e	f	Cw	V	K	Kp ^a	Kp ^b	Js ^a	Js ^b	Fy ^a	Fy ^b	Jk ^a	Jk ^b	Xg ^a	Le ^a	Le ^b	S	s	M	N	P ₁	Lu ^a			Lu ^b		
1	R1WR1	115107	+	+	0	0	+	0	+	0	+	0	+	+	+	0	+	+	0	0	0	+	+	0	0	0	0	0	+	+		37 RKE	6E
2	R1R1	115117	+	+	0	0	+	0	+	0	+	0	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		4+	4+
3	R2R2	43344	+	+	0	0	+	0	+	0	+	0	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		3+	4+
4	Roc	114771	+	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	0	0	0	+	+	+	+	+	+	+	+		3+	4+
5	r'	61360	0	+	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		3+	4+	
6	r'	115112	0	0	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		3+	4+	
7	r'	113149	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		3+	4+	
8	r'	114241	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		3+	4+	
9	r'	111163	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		3+	4+	
10	r'	114712	0	0	0	+	+	+	+	0	0	+	+	+	+	+	+	+	+	0	0	0	+	+	+	+	+	+	+		3+	4+	
11	R1R1 Patient Cells	115114	+	+	0	0	+	+	0	0	+	+	+	+	+	+	+	+	+	0	0	+	+	+	+	+	+	+	+		3+	4+	
Mode of Reactivity			37°C/AntiIgG											Antiglobulin					Variable		Cold		Var.										

* If antigen status may have been determined presumptively based on Rh-hr phenotype.

Cell#	Rh-hr	Donor Number	Rh-hr											KELL					DUFFY		KIDD		LEWIS		MNS		P		Special Antigen Typing	Test Results				
			D	C	E	c	e	f	Cw	V	K	k	Kp ^a	Kp ^b	Js ^a	Js ^b	Fy ^a	Fy ^b	Jk ^a	Jk ^b	Xg ^a	Le ^a	Le ^b	S	s	M	N	P ₁			Lu ^a	Lu ^b		