

22

PATIENT NAME: Anne Weethy
PATIENT ID: 159645

Lot No. 8RA144 Exp. Date 02 JUL 2002

Ortho-Clinical Diagnostics
a Johnson & Johnson company

Panel Reagent Red Blood Cells
0.8% Resolve® Panel A
Antigram® Antigen Profile

TECH: A

DATE: _____
CONCLUSION: _____

Shaded columns indicate those antigens which are destroyed or depressed by enzyme treatment.

Cell#	Rh-hr										KELL				DUFFY				LEWIS				MNS				P		SPECIAL ANTIGEN TYPING		TEST RESULTS					
	D	C	E	c	e	f	C ^w	V	K	k	Kp ^a	Kp ^b	Js ^a	Js ^b	Fy ^a	Fy ^b	Jk ^a	Jk ^b	Xg ^a	Xg ^b	Le ^a	Le ^b	S	S	M	N	P ₁	Lu ^a	Lu ^b	Cell#	37°	AMC	CL	AMC	CL	
1	+	0	0	+	0	+	0	0	0	+	0	+	0	+	+	0	+	0	+	0	+	0	+	+	+	+	+	+	0	0	3+	3+	AMC	CL	AMC	CL
2	+	0	0	+	0	+	0	0	0	+	0	+	0	+	+	0	+	0	+	0	+	0	+	+	+	+	+	+	0	0	3+	3+	3+	3+	3+	3+
3	+	0	0	+	0	+	0	0	0	+	0	+	0	+	+	0	+	0	+	0	+	0	+	+	+	+	+	+	0	0	0	0	0	0	0	0
4	+	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	+	0	0	0	0	0	0	0	0
5	0	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	0	0	0	0	0	0	0	0	0
6	0	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	0	0	0	0	0	0	0	0	0
7	0	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	0	0	0	0	0	0	0	0	0
8	0	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	0	0	0	0	0	0	0	0	0
9	0	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	0	0	0	0	0	0	0	0	0
10	0	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	0	0	0	0	0	0	0	0	0
11	+	0	0	+	0	+	0	0	0	+	0	+	0	+	0	0	+	0	+	0	+	0	+	+	+	+	+	+	0	0	0	0	0	0	0	0
Mode of Reactivity	37°C/Antiglobulin										Antiglobulin				Variable				Cold		Var.															

* I antigen status may have been determined presumptively based on Rh-hr phenotype.

Cell#	Rh-hr										KELL				DUFFY				LEWIS				P		SPECIAL ANTIGEN TYPING		TEST RESULTS										
	D	C	E	c	e	f	C ^w	V	K	k	Kp ^a	Kp ^b	Js ^a	Js ^b	Fy ^a	Fy ^b	Jk ^a	Jk ^b	Xg ^a	Xg ^b	Le ^a	Le ^b	S	S	M	N	P ₁	Lu ^a	Lu ^b	Cell#	37°	AMC	CL	AMC	CL		

