



SOFTBANK - Select patient.

LN: CKABO FN: LESSON MN:

MR: WX0000190 M: Ab: Ag: ADH: Tx:

SS#: - - Sex: N Age: TR: ABO.Rh: AB POS

Adm: 11/30/23 Attn by: FADEYI EMMANUEL Disch.date: //

Loc: MC CC 04 HEMATOLOGY ONCOLOGY

Bill#: WX000001142 Diag:

SOFTBANK (Active)

Patient Requirements		
Bone Marrow Protocol	11/30/23 12:27	by 7716
Irradiated products ONLY	11/30/23 12:20	by 7716
Note to Tech	11/30/23 12:29	by 7716
Transfuse Group B RBC's***	11/30/23 12:27	by 7716
Transfuse Plasma AB	11/30/23 12:28	by 7716
Transfuse Plt's AB or Wash A/B/O	11/30/23 12:28	by 7716
Antibody screen: NEG	11/30/23 12:19	Ord#: E73000000
Antibodies:		
CHECK UNIT ABORH AGAINST PCW	(<CKAB)	11/30/23 12:29 by BT

WHEN PATIENT'S ARE TO RECEIVE AN INCOMPATIBLE ALLOGENEIC STEMCELL TRANSPLANT, TRANSFUSED RC'S AND BLD PRODUCTS NEED TO BE COMPATIBLE WITH BOTH DONOR AND RECIPIENT. THAT IS WHY WE PUT ALL THE ABOVE COMMENTS IN THE PATIENT'S PCW.....**ESPECIALLY THE CKAB ANTIBODY CODE!**

SOFTBANK - Patient Product Order

LN: CKABO FN: LESSON MN:

MR: WX0000190 M: Ab: Ag: ADH: Tx:

SS#: - - Loc: MC CC 04 HEMATOLOGY ONCOLOGY Age: ABO: AB POS

Bill#: WX000001142 Diag: Aux Order#:

Ord#: Type Specific: Electronic Xmatch: Outdate: //

Prod: Units(ord.sel): 0 0 Reason: //

Req: // 00.00 by: Ward:

XM: Instr.: BBID:

SOFTBANK - Batch unit selection

Unit Number	Product	Main Pr.	Ext...	ABO	Rh	Expiration date	Vo...	Auto...	Status
W201223390110	IRC_E0332	IRC		B	P	12/12/2023	300		Selecte
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		

SOFTBANK (Active)

PATIENT has antibody: <CKAB. UNIT antigen CKABO not registered. Issue?

YOU WILL SEE THE FLAG DEMONSTRATED HERE WHEN YOU ARE SELECTING A UNIT.....BASICALLY IT'S TELLING YOU THE UNIT IS NOT "SCREENED" FOR THE "CKABO ANTIGEN". AT THIS POINT YOU WANT TO ASSURE YOU ARE SELECTING THE APPROPRIATE BLOOD TYPE. IT DOES NOT FLAG WITH AN EXCEPTION AT THIS TIME.

SOFTBANK - Patient Product Order

LN: CKABO FN: LESSON MN:

MR: Wx00000190 M: Ab: Ag: ADH: Tx:

SS#: - - Loc: MC CC 04 HEMATOLOGY ONCOLOG Age: ABO: AB POS

Bill#: Wx0000001142 Diag: Aux Order#:

Ord#: E73000000 Type Specific: Electronic Xmatch: Outdate: 12/03/23 23:59

Prod: IRC Units(ord.sel): 1 0 Reason: 07 //

Req: 11/30/23 12:20 by: 7716 Ward: 247

XM: XMIS Instr: BBID:

SOFTBANK - Batch unit selection

Unit Number	Product	Main Pr.	Ext...	ABO	Rh	Expiration date	Vo...	Auto...	Status
W201223390110	RC_E0336	RC		B	P	12/12/2023	300		Available
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		
						//	0		

SOFTBANK (Active)

PATIENT has antibody: <CKAB UNIT antigen CKABO: not registered. Select?

Yes No

AT THE TIME OF ISSUE – YOU WILL SEE THE SAME EXCEPTION BOX COMES UP AGAIN – REMINDING YOU THAT YOU SHOULD HAVE CHECKED THE UNIT TYPE AGAINST THE PCW.

SOFTBANK - Patient Product Order

LN: CKABO FN: LESSON MN:

MR: Wx00000190 M: Ab: Ag: ADH: Tx:

SS#: - - Loc: MC CC 04 HEMATOLOGY ONCOLOG Age: ABO: AB POS

Bill#: Wx0000001142 Diag: Aux Order#:

Ord#: E73000000 Type Specific: Electronic Xmatch: Outdate: 12/03/23 23:59

Prod: IRC Units(ord.sel): 1 1 Reason: 07 //

Req: 11/30/23 12:20 by: 7716 Ward: 247

XM: XMIS Instr: BBID:

SOFTBANK (Active) - Exception occurred: XAGNT - Issued with Not Tested Antigens

```

Unit: W201223390110 IRC_E0332 B POS Patient: AB POS
Pat: CKABO LESSON
MR: Wx00000190 Order#: E73000000
Unit not tested for: CKABO
Unit issued by BT
----- Comment entered when exception record is generated
11/30/23 12:42 by 37
  
```

ONCE YOU HAVE COMPLETED THE ISSUE PROCESS THIS EXCEPTION WILL APPEAR

SOFTBANK - Patient Product Order

LN: CKABO FN: LESSON MN:

MR: WX00000190 M: Ab: Ag: ADH: Tx:

SS#: -- Loc: MC CC D4 HEMATOLOGY ONCOLOG Age: ABO AB POS

Bill#: WX000001142 Diag: Aux Order#:

Ord#: E7300000 Type Specific: Electronic Xmatch: Outdate: 12/03/23 23:59

Prod: IRC Units(ord.sel): 1 1 Reason: 07 //

Req: 11/30/23 12:20 by: 7716 Ward: 247

XM: XMIS Instr: BBID:

SOFTBANK (Active) - Exception occurred: XAGNT - Issued with Not Tested Antigens

```

Unit: W201223390110 IRC_E0332 B POS Patient AB POS
Pat: CKABO LESSON
MR#: WX00000190 Order#: E73000000
Unit not tested for: CKABO
Unit issued by BT
----- Comment entered when exception record is generated:
11/30/23 12:42 by BT:
GIVING 3 RED CELLS PER PCW

```

IN THIS EXAMPLE THE EXCEPTION HAS BEEN ANSWERED WITH A BRIEF COMMENT STATING WHAT BLOOD TYPE WAS GIVEN AND WHY.

Prod: IRC	Unit:
Req: 11/30/23 12:20	by: 77
XM: XMIS	Instr:

SOFTBANK - Exception occurred: XAGNT - Issued with Not Tested Antigens

Unit: W201223390110 IRC_E0332 B POS
 Pat: CKABO LESSON
 MR#: WX00000190 Order#: E73000000
 Unit not tested for: CKABO
 Unit issued by BT
 ----- Comment entered when exception record is generated:
 @11/30/23 12:42 by BT:

HMLYS	HEMOLYZED
ICTRC	ICTERIC
IERRNO	Irradiation not needed
MBINC	Mother/Baby ABO/Rh Incompatibility
MERGE	Merged files in SCC under new #
MFABO	MF ABO Explanation
MFRH	MF RH explanation
MTP	Massive Transfusion Protocol
NO2AB	No 2nd ABORH collected giving group O
OBNOT	Patient is NOT an OB patient
OBPT	Patient is OB. to be sent to PCW
PCWC	RC ABORH given per PCW
PTRGO	Patient received multiple group O RBCs
PTRHN	Patient received Rh negative RBC
PTRHP	Patient has received Rh positive RBC
RECT	Receipt testing complete and acceptable
RVHTR	HP -Critical Alert, Suspected HTR phon
RSV	Released please receipt and discarded

IN THIS EXAMPLE THE TECH OPTED TO USE THE CANNED COMMENT CREATED SPECIFICALLY FOR THESE INSTANCES

SOFTBANK - Patient Product Order

LN: CKABO FN: LESSON MN:

MR: WX00000190 M: Ab: Ag: ADH: Tx:

SS#: .. Loc: MC CC 04 HEMATOLOGY ONCOLOG Age: ABO: AB POS

Bill#: WX000001142 Diag: Aux Order#:

Ord#: E73000000 Type Specific: Electronic Xmatch: Outdate: 12/03/23 23:59

Prod: IRC Units(ord.sel): 1 1 Reason: 07 //

Req: 11/30/23 12:20 by: 7716 Ward: 247

XM: XMIS Instr: BBID:

SOFTBANK (Active) - Exception occurred: XAGNT - Issued with Not Tested Antigens

Unit: W201223390110 IRC_E0332 B POS Patient: AB POS
 Pat: CKABO LESSON
 MR#: WX00000190 Order#: E73000000
 Unit not tested for: CKABO
 Unit issued by BT
 ----- Comment entered when exception record is generated:
 @11/30/23 12:42 by BT:
 PCW checked. See PCW give this type of Red cells:

ONCE THE SPECIFIC CANNED COMMENT IS CHOSEN IT WILL AUTO FILL THE EXCEPTION BOX AS SEEN ABOVE

SOFTBANK - Patient Product Order

LN: CKABO FN: LESSON MN:

MR: WX00000190 M: Ab: Ag: ADH: Tx:

SS#: .. Loc: MC CC 04 HEMATOLOGY ONCOLOG Age: ABO: AB POS

Bill#: WX000001142 Diag: Aux Order#:

Ord#: E73000000 Type Specific: Electronic Xmatch: Outdate: 12/03/23 23:59

Prod: IRC Units(ord.sel): 1 1 Reason: 07 //

Req: 11/30/23 12:20 by: 7716 Ward: 247

XM: XMIS Instr: BBID:

SOFTBANK (Active) - Exception occurred: XAGNT - Issued with Not Tested Antigens

Unit: W201223390110 IRC_E0332 B POS Patient: AB POS
 Pat: CKABO LESSON
 MR#: WX00000190 Order#: E73000000
 Unit not tested for: CKABO
 Unit issued by BT
 ----- Comment entered when exception record is generated:
 @11/30/23 12:42 by BT:
 PCW checked. See PCW give this type of Red cells: B POS

HERE THE TECH ADDED THE CORRECT ABO TYPE OF BLOOD/BLOOD PRODUCT TO GIVE PER THE PCW.

Problem occurred: 11/15/23 22:29 Released: 11/22/23 14:05 by: BT
 Problem class: XAGNT Issued with Not Tested Antigens Site: WFBMC

SOFTBANK (Active) - Exception comment view

Unit: W121623451338 IRC_E0392 O POS Patient: A POS
 Pat: HILL, AMANDA N.
 MR#: 4690199 Order#: 951501577
 Unit not tested for: CKABO.
 Unit issued by: [REDACTED]

----- Comment entered when exception record is generated:
 @11/15/23 22:29 by [REDACTED]
 CKABO

----- Comment entered when exception record is released:
 @11/22/23 14:05 by BT:
 OK

WRONG

WHAT'S OK?

SOFTBANK - Exception Report

Class: XAGNT Issued with Not Tested Antigens Site: [REDACTED] Not released: [X]
 Date range, from: 11/01/23 00:00 to: 11/30/23 11:00 Released: [X]

SOFTBANK

Problem occurred: 11/24/23 02:28 Released: 11/30/23 11:23 by: BT
 Problem class: XAGNT Issued with Not Tested Antigens Site: WFBMC

SOFTBANK (Active) - Exception comment view

Unit: W201223435506 IRC_E0668 O POS Patient: A POS
 Pat: HILL, AMANDA N.
 MR#: 4690199 Order#: 952303552
 Unit not tested for: CKABO.
 Unit issued by: [REDACTED]

----- Comment entered when exception record is generated:
 @11/24/23 02:28 by [REDACTED]
 PCW checked. Per PCW give this type of Red cells: [REDACTED]

----- Comment entered when exception record is released:
 @11/30/23 11:21 by BT:
 OK

WRONG

USED CANNED COMMENT – BUT LEFT BLANK

SOFTBANK - Exception Report

Class: XAGNT Issued with Not Tested Antigens Site: [REDACTED] Not released: [X]
 Date range, from: 11/01/23 00:00 to: 11/30/23 11:00 Released: [X]

SOFTBANK

Problem occurred: 11/29/23 10:36 Released: 11/30/23 11:23 by: BT
 Problem class: XAGNT Issued with Not Tested Antigens Site: WFBMC

SOFTBANK (Active) - Exception comment view

Unit: W202929277155 IRC_E0332 O NEG Patient: AB POS
 Pat: DAVIS, CHRISTOPHER M.
 MR#: 1108643 Order#: 952800463
 Unit not tested for: CKABO.
 Unit issued by: [REDACTED]

----- Comment entered when exception record is generated:
 @11/29/23 10:36 by [REDACTED]
 Patient has received a BMT

----- Comment entered when exception record is released:
 @11/30/23 11:22 by BT:
 OK

WRONG

THIS DOESN'T TELL ANYONE ANYTHING ABOUT WHAT BLOOD TYPE IS TO BE GIVEN PER THE PCW OR IF YOU ARE ACTUALLY GIVING THE CORRECT TYPE

SOFTBANK - Exception Report

Class: XAGNT Issued with Not Tested Antigens Site: Not released:

Date range, from: 11/01/23 00:00 to: 11/30/23 11:00 Released:

SOFTBANK

Problem occurred: 11/29/23 10:35 Released: 11/30/23 11:23 by: BT

Problem class: XAGNT Issued with Not Tested Antigens Site: WFBMC

SOFTBANK (Active) - Exception comment view

Unit: W201223405333 IRC_E0332 O POS Patient: O POS
 Pat: ROBERSON, GARY
 MR#: 5255200 Order#: 952701156
 Unit not tested for: CKABO.
 Unit issued by: [REDACTED]
 ----- Comment entered when exception record is generated:
 @11/29/23 10:35 by [REDACTED]
 O PER PCW
 ----- Comment entered when exception record is released:
 @11/30/23 11:22 by BT:
 OK

ACCEPTABLE

CANNED COMMENT NOT USED BUT ACKNOWLEDGED APPROPRIATELY THAT THE BLOOD TYPE GIVEN WAS PER THE PCW

SOFTBANK - Exception Report

Class: XAGNT Issued with Not Tested Antigens Site: Not released:

Date range, from: 11/01/23 00:00 to: 11/30/23 11:00 Released:

SOFTBANK

Problem occurred: 11/02/23 13:11 Released: 11/06/23 09:44 by: CSW

Problem class: XAGNT Issued with Not Tested Antigens Site: WFBMC

SOFTBANK (Active) - Exception comment view

Unit: W201223343980 IRC_E0332 A NEG Patient: A POS
 Pat: CAMPBELL, PAULA J.
 MR#: 1373380 Order#: 943000997
 Unit not tested for: CKABO.
 Unit issued by: [REDACTED]
 ----- Comment entered when exception record is generated:
 @11/02/23 13:11 by [REDACTED]
 PCW checked. Per PCW give this type of Red cells:A NEG RBCS
 ----- Comment entered when exception record is released:
 @11/06/23 09:44 by CSW:
 OK

ACCEPTABLE

CANNED COMMENT USED AND FILLED IN APPROPRIATELY