

## i1000/i2000 Refrigerated Calibrators

Test	Abbott Calibrator Name	Preparation	Open Stability	Storage Temp After Opening	Number of Levels
BHCG	BHCG Cals	Ready to Use	Until Exp Date	2-8° C	6
Free PSA	Free PSA Cals	Ready to Use	Until Exp Date	2-8° C	2
Total PSA	Total PSA Cals	Ready to Use	Until Exp Date	2-8° C	2
Free T4	FT4 Cals	Ready to Use	Until Exp Date	2-8° C	6
TSH	TSH Cals	Ready to Use	Until Exp Date	2-8° C	2
Vitamin B12	B12 Cals	Ready to Use	30 Days	2-8° C	6
Ferritin	Ferritin Cals	Ready to Use	Until Exp Date	2-8° C	2
BNP	BNP Cals	Ready to Use	Until Exp Date	2-8° C	6
Free T3	FT3 Cals	Ready to Use	Until Exp Date	2-8° C	2
Troponin	Stat Troponin-I Cals	Ready to Use	Until Exp Date	2-8° C	6
Homocysteine	HCY Cals	Ready to Use	Until Exp Date	2-8° C	6
Myoglobin	Stat Myoglobin Cals	Ready to Use	Until Exp Date	2-8° C	6
CA 19-9	CA 19-9xr Cals	Ready to Use	Until Exp Date	2-8° C	6
Estradiol	Estradiol Cals	Ready to Use	Until Exp Date	2-8° C *Sensitive to Light*	6
FSH	FSH Cals	Ready to Use	Until Exp Date	2-8° C	2
LH	LH Cals	Ready to Use	120 Days	2-8° C	6
Insulin	Insulin Cals	Ready to Use	Until Exp Date	2-8° C	6
AFP Tumor Marker	AFP Cals	Ready to Use	Until Exp Date	2-8° C	6
CA 125	CA 125 II Cals	Ready to Use	Until Exp Date	2-8° C	6
CA 15-3	CA 15-3 Cals	Ready to Use	Until Exp Date	2-8° C	6
C-Peptide	C-Pep Cals	Ready to Use	Until Exp Date	2-8° C	6
DHEA-S	DHEA-S Cals	Ready to Use	Until Exp Date	2-8° C	6
CEA	CEA Cals	Ready to Use	Until Exp Date	2-8° C	2
Vitamin D 25-OH	VitD 25OH Cals	Ready to Use	Until Exp Date	2-8° C	6