
TEG®6s Maintenance

RC.CH.CSL.TEG.PR.002r00

Principle

The TEG®6s Haemostasis system is indicated for in vitro diagnostic use to provide quantitative indications of the hemostasis state of a venous blood sample. Daily, weekly, and monthly maintenance are performed to assure that the instrument is functioning optimally.

Supplies

1. Alcohol wipes
 2. Water
-

Procedure

- A. Daily Maintenance
Turn the analyzer off and on
 1. Find the toggle switch at the back on the analyzer.
 2. Flip the switch off (down).
 3. Wait one minute.
 4. Flip the switch back on (up).
 5. The analyzer will go through its system checks.
- B. Weekly Maintenance
Clean outside of analyzer with alcohol.
- C. Monthly Maintenance:
Clean the fan filter with water.
 1. Remove the fan cover at the back of the instrument.
 2. Remove the fan filter.

TEG® 6s Maintenance

3. Clean filter with water.
4. When dry, put filter and fan cover back on.

References

Haemonetics® TEG® 6s Operators Aide.
TEG® 6s User manual

Attachments

Authorized Reviewers

Not posted online. Printed copies of this document are not considered up-to-date. Please verify current version date with master document.

Clinical Pathology: *Chemistry*
BEAUMONT LABORATORY Royal Oak

DATE: 07/09/2019 RC.CH.CSL.TEG.PR.002r00

TEG® 6s Maintenance

Document Control

Location of Master: Master electronic file stored on the Beaumont Laboratory server under S:/ClinPathChem/Automated Chemistry/Document Control Library/NEW INDEX AND TEMPLATE/CSL/TEG

Number of Controlled Copies posted for educational purposes:

Number of circulating Controlled Copies:

Location of circulating Controlled Copies:

Document History

| | | Revision # | | Related Documents |
|---|-----------|---------------|--------------|----------------------|
| Signature | Date | | | Reviewed/ |
| Prepared by: Kelly Walewski C (ASCP)cm | 6/21/2019 | | | |
| Approved by: Kelly Walewski C(ASCP)cm | 6/21/2019 | | | |
| | | | | |
| Reviewed by: (Signature) | Date | Re visi | Modification | Related |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |

Not posted online. Printed copies of this document are not considered up-to-date. Please verify current version date with master document.

Clinical Pathology: Chemistry

BEAUMONT LABORATORY Royal Oak

DATE: 07/09/2019 RC.CH.CSL.TEG.PR.002r00

TEG® 6s Maintenance

Not posted online. Printed copies of this document are not considered up-to-date. Please verify current version date with master document.

Clinical Pathology: *Chemistry*

BEAUMONT LABORATORY Royal Oak

DATE: 07/09/2019 RC.CH.CSL.TEG.PR.002r00