

## Platelet Factor 4 IgG Assay

### Appendix D

#### HPF4 Read on the ELx-800 Plate Reader

1. Select the Gen5 2.07 program
2. Select Experiments > Create using existing protocol
3. Select protocol (HPF4 or HPF4 Confirm) > Open
4. Click Read New (green play button) > Select wells to be read
5. Place plate in carrier (on ELx-800 Reader) > Click OK
6. A Save As box will appear at the end of run. Identify run using the month.day.year (Example: 12.19.16) and add "confirm" to the end of a confirmatory run (Example 12.19.16confirm) > Click OK
7. Select Printer Icon to print your results
8. Close program
9. Report data from column indicating "Mean"

***Printed copies of this document are not considered up-to-date. Please verify current version date with online document.***