

Calibrator	Abbott Name	Preparation	Open Stability (Days)	Storage Temp after opening	Number of Levels	Requires Set Point Change with New LOT	Assays Calibrated
ICT Serum Cal	ICT Low/High	Ready to Use	7	2-8° C	2	N	CI-C K-C Na-C
ICT Urine Cal	ICTU Low/High	Ready to Use	7	2-8° C	2	N	CI-CU K-CU Na-CU
Multiconstituent Cal	MCC	Ready to Use	7	2-8° C	2	Y	AlbG CaC CaCU Chol CreaC CreaCU Glu GluCU Fe-PI LactA MAG MAGU Phos PhosU TP Trig UA Urea UreaU
BHBT Cal	BHOB	Ready to Use	Until Exp Date	2-8° C	1	N	BHBT
Bilirubin Cal	Bil	Ready to Use	7	2-8° C	2	Y	BiID BiIT
CO2 Cal	CO2C	Ready to Use	30	2-8° C	2	Y	CO2C
Lipid Multiconstituent Cal	LIPDMCC	ADD 1mL H2O	7	2-8° C	1	Y	UHDLD LDL
Urine/ CSF Protein Cal	Upro	Ready to Use	Until Exp Date	2-8° C	5	N	Upro
Ammonia Cal (included with Reagent)	Amm	Ready to Use	120	2-8° C	1	N	Amm
Lipase Cal	Lipase	ADD 3mL H2O	28	2-8° C	1	Y	Lip
Specific Protein Cal	SP	Ready to Use	30	2-8° C	5	Y	C3 C4 Hapt IgA IgG IgM TRF
Microalbumin Cal	uAlbCal	Ready to Use	6 months	2-8° C	5	N	uAlb
Prealbumin Cal	PreAlb	Ready to Use	30	2-8° C	5	N	Palb
CRP Cal Set *	32CRP	Ready to Use	90	2-8° C	5	N	CRP32 * See below for details on CRP Calibration
CRP Cal Set *	16CRP	Ready to Use	90	2-8° C	5	N	CRP16 * See below for details on CRP Calibration
Quantia Protein Cal	PROT	Ready to Use	Until Exp Date	2-8° C	5	Y	A1AT
Plasma Protein Cal	PPCS3	Ready to Use	30	2-8° C	1	Y	Cerul
Quantia IgE	IgE	Ready to Use	Until Exp Date	2-8° C	5	N	IgE
Rheumatoid Factor Cal	RF	Ready to Use	90	2-8° C	5	N	RF
Quantia ASO Cal	ASO	ADD 1mL H2O	15	2-8° C	1	N	ASO
Ethanol Cal	EtoH	Ready to Use	Until Exp Date	2-8° C	2	N	ETOH
B2 Microglobulin Cal	B2M	Ready to Use	Until Exp Date	2-8° C	1	Y	B2M
Quantia Lp(a) Cal	Lpa	ADD 1 mL H2O	15	2-8° C	5	Y	Lpa
Internal Water							AlkP ALT Amy AmyU AST CK GGT LD
TDM Multiconstituent Cal	TDM MCC	Ready to Use	60	2-8° C	6	Y	AMIK CARB DIG GENT PHENO PHENY THEO VALP VANCO
Acetaminophen Cal	ACET	Ready to Use	Until Exp Date	2-8° C	1	N	ACETA
Salicylate Cal		Ready to Use	Until Exp Date	2-8° C	1	N	SALIC
Tobramycin Cal	TOBRA	Ready to Use	Until Exp Date	2-8° C	6	N	TOBRA
Clin Chem Cal	CCC-S	ADD 3mL H2O***	2 days #	2-8° C	1	Y	LITHI CRENZ #Can be frozen for 14 days *** Wait 30 Minutes
DOA Neg	DOAQL1	Ready to Use	Until Exp Date	2-8° C	1	N	Amphet Barb Benzo Cannab Cocaine Methadone Opiate PCP
DOA MC 1	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Amphet Benzo
DOA MC 2	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Barb Benzo Cocaine Methadone
DOA MC 3	See below	Ready to Use	Until Exp Date	2-8° C	1	N	PCP Benzo
DOA MC 4	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Benzo
THC 50	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Cannab
Opiate 300	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Opiate
Neg Cal 0	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Fentanyl
Fentanyl Urine Cal 1	See below	Ready to Use	60 days	2-8° C	1	N	Fentanyl
Neg Cal Ref	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Oxycodone
Oxycodone Urine Calibrator	See below	Ready to Use	Until Exp Date	2-8° C	1	N	Oxycodone

CRP Details for Calibration

	CAL 1	CAL 2	CAL 3	CAL 4	CAL 5	CAL 6
CRP 16 (hsCRP	2.5 Bright Yellow	5.0 White	10 Light Yellow	20 Green	80 Pink	160 Red
CRP 32 (CRP	5 White	10 Light Yellow	20 Green	40 Blue	160 Red	320 Brown

hsCRP calibrator contains only the 2.5 calibrator with the Bright Yellow Lid. This calibrator is combined with the CRP calibrators to complete the calibrator set. Once in use the calibrators are all combined in the CRP Calibrator Box.

Drugs of Abuse Details

Analyte	Calibrators		Abbott Name Negative	Abbott Name for Cut off Cal
Amphetamines	DOA Neg	DOA Cal #1	DOAQL1	DOAQL 2
Barbituates	DOA Neg	DOA Cal #2	DOAQ 1	DOAQ 2
Benzodiazepine	DOA Neg	DOA Cal #1,2,3,4	DOASQ1	DOASQ2,3,4,5
Cannabinoids	DOA Neg	THC 50	THCQ1	THCQ2
Cocaine	DOA Neg	DOA Cal #2	DOAQ1	DOAQ2
Opiates	DOA Neg	Opiate 300	OPQ1	OPQ2
Methadone	DOA Neg	DOA Cal # 2	DOAQ1	DOAQ2
PCP	DOA Neg	DOA Cal #3	DOAQ1	DOAQ3
Fentanyl	Neg Cal 0	Fentanyl Urine Cal 1	Immunalysis Negative 0	Immunalysis Fentanyl Urine Cal 1
Oxycodone	Neg Cal	Oxycodone Urine Cal 100	Immunalysis Negative Reference Cal	Immunalysis Oxycodone Urine Calibrator 100