

ATTACHMENT B- ENTERING LOT NUMBERS

- A. **System Ready** display, press keypad next to [Menus].
 - a. From Menu display, press [1] to select Run Control option.
 - b. Use the numeric keypad to enter the numeric portion of the lot number of the test cartridge being used.
 - c. To modify the lot number use the [<] key to erase it, then enter the correct number. Press keypad next to [Enter].

- B. **PRINT CONTROL RESULTS:**
 - a. From **System Ready** display, press keypad next to [Menus].
 - b. From Menu display, press [4] to select *Print Log* option.
 - c. From Print Log display, press [2] to select *Control Results* option.
 - d. Use numeric keypad to enter the number of control results to be printed. Press keypad next to [Enter]. The system will print the number of results entered, starting with the most recent control performed.