

# Beaumont

Origination 1/21/2022  
Last 12/28/2023  
Approved  
Effective 12/28/2023  
Last Revised 12/28/2023  
Next Review 12/27/2025

Document Kelly Walewski:  
Contact Supv, Laboratory  
Area Laboratory-  
Chemistry  
Applicability Royal Oak

## Automated Chemistry Contaminated Specimen Alert - Royal Oak

Document Type: Procedure

### I. PURPOSE AND OBJECTIVE:

- A. This document details how the laboratory handles samples which may be contaminated when the specimens are drawn by nursing and/or phlebotomy.

### II. POLICY:

- A. Chemistry technologists and technicians are expected to recognize and investigate abnormal results that suggest a sample has been contaminated with an IV solution (e.g. Dextrose, Total Parenteral Nutrition (TPN), Saline, IV medication etc.) or that has been improperly collected with EDTA. They are expected to take the following appropriate action.

### III. SPECIMEN HANDLING:

- A. The technologist or technician will retrieve the questionable specimen.
- B. Repeat testing.
- C. Contact the patient's nurse to deliver a scripted message with results. The specimen may be contaminated with IV solution (see scripting below).
- D. Results are released with comment and an alert to interpret with caution.
- E. The patient's nurse will be notified that all specimens drawn at this time will need to be re-ordered and re-collected.
- F. Critical results are reported with documentation in the LIS.
- G. It is the clinical team's or physician's decision to accept the results as clinically appropriate or

to phone the Lab and request test cancellation.

H. Specimens with obvious EDTA contamination are canceled.

## IV. PROCEDURE:

- A. When the laboratory technologist/technician suspects that a specimen is contaminated with TPN, Saline, Dextrose or other IV fluid, or that a specimen contains EDTA, he/she should:
1. Review the abnormal Chemistry result(s) and result(s) trend(s) in the LIS Chart Review together with a Supervisor, Lead Tech, or second Technologist. Suspected contamination is a judgment call.
  2. Retrieve the specimen and repeat suspect result(s), preferably on a second analyzer.
  3. "Verify" or "Repeat" any criticals (with exception of Troponin and blood gas pH/pCO<sub>2</sub>/pO<sub>2</sub>).
  4. Retrieve collection date, time, collector information.
  5. Contact the Nursing Unit to speak with the nurse who is taking care of the patient.
  6. Initiate the appropriate scripted conversation to verify B#, release non-critical results, release critical results or cancel results with obvious EDTA contamination.
  7. Per procedure, request read back for any critical result reported.
  8. Request nurse's employee ID # for documentation purposes.
  9. Document results in LIS with appropriately selected, scripted text comment and ID # of nurse alerted.
    - a. NOTE: Scripted text result comment for suspected contamination is attached to the first reported chemistry analyte result (e.g. sodium) of the affected panel or group of tests.
  10. Notify the patient's nurse and request that all tests drawn at this time be reordered in the Hospital Information System (HIS) and recollected.
    - a. Ask Order to be placed with STAT priority, unless original order included tests (e.g. Special Chemistry) that are only available as routine. In this case, a "mixed priority" may be necessary.
  11. For Outreach or Outpatient cases (e.g. Nursing Home, Home Care, Dialysis) with suspected specimen IV contamination, Chemistry Technologist will:
    - a. Report results with suspected contamination comment.
    - b. Complete documentation in LIS for any critical result associated with suspected contaminated specimen.
    - c. Phone Customer Services with patient's Name and medical record number (MRN) to explain situation. They are to place the call to client to deliver results and scripted message.
    - d. Cancel specimens with obvious EDTA contamination.
      - i. Notify Customer Services per standard protocol to request that client recollect specimen.

12. Should Nursing or Customer Services instruct the Technologist to “just cancel the results and we will redraw them”, the Technologist will state our procedure is to release results with a statement indicating suspicion of IV contamination.
13. If it is decided that the sample was contaminated with IV fluid, the physician or nurse may call the Lab (x11155) to remove the result.
  - a. Lab can still retrieve original results from the LIS audit trail (Chart Review), if further investigation is warranted.
  - b. Attach appropriate selected LIS cancellation comment “Possible specimen contamination with IV fluid. Canceled per request of \_\_\_\_\_”.
14. EDTA specimen contamination will affect certain chemistries (e.g. K, Ca, Alkaline Phosphatase, Mg, CK). When EDTA contamination is suspected:
  - a. Cancel all Automated Chemistry tests on the affected specimen as well as Coagulation and Special Testing tests drawn at the same time.
  - b. Enter the reason for cancellation as “results suggestive of EDTA contamination.”
  - c. The K will be increased, and the Ca, Alkaline Phosphatase, Mg and CK will be decreased.
  - d. If a Phlebotomist draw is involved, immediately notify the phlebotomy supervisor.
  - e. Request that the Nursing Unit place an order in the HIS for specimen recollection.
  - f. If EDTA contaminated specimen(s) are from Outpatient or Outreach patients, contact Customer Services to arrange for specimen recollection.

#### **B. Scripting - Abnormal Chemistry Results with Possible IV Contamination**

##### **1. Technologist:**

*This is \_\_\_\_\_ from the chemistry lab. May I please speak to the nurse taking care of \_\_\_\_\_(patient first and last name) in room \_\_\_\_\_(number).*

##### **2. Technologist to nurse (non-critical result):**

- a. *This is \_\_\_\_\_ from the chemistry lab. I'm calling about \_\_\_\_\_(patient's first and last name). May I have the patient's MRN \_\_\_\_\_(Wait for response). Thank you.*  
*Until the last blood draw on \_\_\_\_\_(date/time of draw), \_\_\_\_\_(electrolyte/ glucose/etc) results were running in the \_\_\_\_\_(normal/low/high) range and now the result is significantly different. I'm concerned that the sample may be collected from the wrong patient, or contaminated with an IV fluid, IV medication or TPN solution.*  
*Will you please verify the B# ? (wait for verification). Thank you.*
- b. (If Registered Nurse (RN) unwilling to retrieve B#, just proceed. If mismatch, state there is a B# discrepancy and you will need to cancel the labs and follow with a variance report.)  
*Results are being released into \_\_\_\_\_(patient's first and last name) chart*

*with a comment indicating possible specimen contamination with IV fluid and an alert to interpret results with caution. Appropriate specimens drawn at this time for (state all Chemistry, Hematology/Coagulation, Special Testing tests on order) must be re-ordered.*

*May I please have your employee ID number for documentation purposes. Thank you.*

**3. Technologist to nurse (critical result):**

- a. *This is \_\_\_\_ from the chemistry lab. I'm calling about \_\_\_\_ (patient's first and last name). May I have the patient's MRN (wait for response). Thank you.*

*Until the last blood draw on \_\_\_\_ (date/time of draw), (electrolyte/glucose/ etc) results were running in the normal/low/high range and now the \_\_\_\_ result is CRITICAL.*

*I'm concerned that the sample may be collected from the wrong patient, or contaminated with an IV fluid, IV medication or TPN solution.*

*Will you please verify the B#? (wait for verification). Thank you.*

- b. *(If RN unwilling to retrieve B#, just proceed. If mismatch, state there is a B# discrepancy and you will need to cancel the labs and follow with a variance report).*

*Results are being released into \_\_\_\_ (patient's first and last name) chart with a comment indicating possible specimen contamination with IV fluid and an alert to interpret results with caution. Appropriate specimens drawn at this time for (state all Chemistry, Hematology/Coagulation, Special Testing tests on order) must be re-ordered.*

*However, since the result(s) fall(s) into the critical range for our laboratory, I need to document that \_\_\_\_ (test) is CRITICAL at \_\_\_\_ (result).*

*Would you please read that back to me.*

*May I please have your employee ID # for documentation purposes. Thank you.*

**4. If the nurse says: "Just cancel the results and I'll redraw them".**

*Our lab policy is to release the results with a statement indicating suspicion of IV contamination. The clinical team or physician can then decide whether the results are appropriate. You may contact the Lab to cancel the results for this test if desired.*

*May I have your employee ID number for documentation purposes? (If not previously given)*

**5. IV Contamination Comment for LIS:**

*"Possible specimen contamination with IV fluid, IV medication or TPN solution. Interpret results with caution. Appropriate samples drawn at this time must be re-ordered. Please contact the lab to cancel this test if desired. Nurse # \_\_\_\_ alerted".*

**6. Scripting - Abnormal Chemistry Results with EDTA Contamination**

*This is \_\_\_\_ from the chemistry lab. I'm calling to cancel the \_\_\_\_ (test, panel, etc.) on \_\_\_\_ (patient first and last name), MRN \_\_\_\_\_. The SST specimen collected \_\_\_\_ (date/time) appears to be grossly contaminated with EDTA, as the K+ is significantly increased and the Calcium is significantly decreased. May I please have your ID# for documentation purposes? Thank you.*

## Approval Signatures

Step Description	Approver	Date
CLIA Directors	Ann Marie Blenc: System Med Dir, Hematopath	12/28/2023
System Medical Director	Caitlin Schein: Staff Physician	12/21/2023
Medical Director	Subhashree Mallika Krishnan: Staff Physician	12/18/2023
Technical Director	Qian Sun: Tech Dir, Clin Chemistry, Path	12/18/2023
Technical Director	Nga Yeung Tang: Tech Dir, Clin Chemistry, Path	12/15/2023
Policy and Forms Steering Committee Approval (if needed)	Kelly Walewski: Supv, Laboratory	12/15/2023
Lab Manager	Leah Korodan: Mgr, Division Laboratory	12/15/2023
	Kelly Walewski: Supv, Laboratory	12/15/2023