
PROSTATE CORES

RA.SP.PR.GR.BX.13.r01

-
- I. Purpose**
To provide a procedure for the gross examination of a prostate core biopsy specimen.
- II. Principle**
To submit all tissue cores so that a diagnosis can be made microscopically by a pathologist.
- III. Equipment**
1. Ruler
 2. Forceps
 3. Lens paper
- IV. Special Safety Precautions**
1. PPE should be worn.
 2. FORMALIN is a known carcinogen.
- V. Supplies and Reagents**
1. 10% NEUTRAL BUFFERED FORMALIN (pH range 6.9-7.2)
 2. Hematoxylin
- VI. Quality Control**
All tissue should be submitted.
- VII. Limitations/Notes**
The following may influence the validity of test results:
1. The specimen should be fixed in formalin.

PROSTATE CORES

VIII. Procedure

1. The specimen will arrive in a jar of formalin and usually received as numerous parts.
2. Using forceps, carefully remove all cores from the container.
3. Dictate the number, the size range (smallest length to largest length in cm.), the color, and the consistency of the cores.
4. Submit all cores by wrapping them in lens paper with no more than three cores per cassette.
5. Ink all cores with hematoxylin.
6. Verify the correct protocol is selected. Cases with under 15 parts (A-N) should have serial sections X6- 1, 3, and 5 H&E; 2, 4, and 6 unstained. Cases with 15 parts or more (saturation biopsies) should have just three H&E levels.
7. Load the cassette or cassettes on the next small processor to allow for proper fixation.

IX. References

1. Hruban RH, Westra, WH, Phelps, PH, & Isacson, C: Surgical Pathology Dissection An Illustrated Guide, New York, NY, Springer-Verlag Inc., 1996.
2. Lester, SC: Manual of Surgical Pathology, New York, NY, Churchill Livingstone, 2001.

X. Authorized Reviewers

1. Medical Director, Anatomic Pathology
2. Chief, Surgical Pathology

XI. Document Control

Location of Master: Master electronic file stored on the Beaumont Laboratory server under S:\AP_Grossing_Manual

Number of Controlled Copies posted for educational purposes: 0

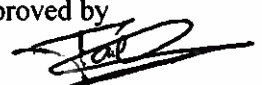
Number of circulating Controlled Copies: 1

Location of circulating Controlled Copies: Master Grossing Manual located in Surgical Pathology

Printed copies of this document are not considered up-to-date. Please verify current version date with online document.

PROSTATE CORES

Document History

Signature	Date	Revision #		Related Documents Reviewed/ Updated
Prepared by: <i>Anne Tranchida PA(ASCP)</i>	8/2/2007	r00		
Approved by: <i>Ali-Reza Armin, MD</i>	12/5/2007	r00		
Reviewed by: (Signature)	Date	Revision #	Modification	Related Documents Reviewed/ Updated
<i>Ali-Reza Armin, MD</i>	12/10/2008	r00		
<i>Anne R. Tranchida, PA(ASCP)</i>	10/20/2009	r00		
<i>Ali-Reza Armin, MD</i>	10/20/2011	r00		
<i>Ali-Reza Armin, MD</i>	4/2/2013	r00		
<i>Mitval B. Amin, MD</i>	2/14/2015	r00		
<i>Zhenhong H. Qu, MD</i>	3/12/2015	r00		
<i>Kurt Bernacki, MD</i>	10/27/2017	r00		
<i>Kurt Bernacki, MD</i>	10/21/2019	r00		
<i>Kurt Bernacki, MD</i>	10/20/2021	r00		
<i>Jawwad Arshad, MD</i>	3/20/2023	r00		
Revised by: <i>Heather Genson, HTL(ASCP)^{CM} PA^{CM}</i> 	8/22/2025	r01	Added steps to include the protocol for saturation biopsies	
Approved by 	9/19/25	r01		

Printed copies of this document are not considered up-to-date. Please verify current version date with online document.