

Daily Rack QC

Doctors Medical Center
 1441 Florida Avenue
 Modesto CA 95350
 Transfusion Service Department

Reviewed By: _____

Month/Year: _____

Date _____

BLOOD BANK			Date	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
			CLS's Initials																
			Daily Review: Initials																
Tube #	Reagents	QC Reagent	Expected Results																
1	Anti-A	CorQC Rgt Cells	2-4+	IS															
2	Anti-B	CorQC Rgt Cells	2-4+	IS															
3	Anti-A,B	CorQC Rgt Cells	2-4+	IS															
4	Mono Ctrl	CorQC Rgt Cells	0	IS															
5	Anti - D	CorQC Rgt Cells	2-4+	IS															
6	Anti - D	A1 Cells	0	IS/AHG	/	/	/	/	/	/	/	/	/	/	/	/	/	/	/
7	A1 Cells	CorQC Anti-Serum	2-4+	IS															
8	A2 Cells	CorQC Anti-Serum	2-4+	IS															
9	B Cells	CorQC Anti-Serum	2-4+	IS															
10	SCI	CorQC Anti-Serum	1-3+	AHG															
11	SCII	CorQC Anti-Serum	1-3+	AHG															
12	SCIII	CorQC Anti-Serum	1-3+	AHG															
13	SCI	Anti - A	0	IS/AHG	/	/	/	/	/	/	/	/	/	/	/	/	/	/	/
14	Check Cells M	Anti-IgG	1-4+	IS															
15	Check Cells P	Anti - IgG, -C3b,-C3d	1-4+	IS															
Reagent Rack - Appearance, Performance, Arrangement and Rgt Lot #s				S/U															

Reagent	Lot #	Exp. Date	Start Date	New Lot #	Exp Date	Start Date
Anti -A						
Anti -B						
Anti -A,B						
Anti -D						
Mono Ctrl						
A1 Cells						
A2 Cells						
B Cells						
Screen Cells						
LISS						
PEG						
Anti-IgG (AHG)						
Polyspecific AHG						
Check Cells						
CorQC Cells						
CorQC Serum						

Comments? Corrective Actions:

IS - Immediate Spin AHG - Antihuman Globulin
 Each CLS's will review the previous day's QC.
 Write your Initials in the "Daily Review" column
 All Reagents are Manufactured by Immucor
 P - Polyspecific
 S - Satisfactory
 M- Monospecific
 U - Unsatisfactory

Daily Rack QC

Doctors Medical Center
 1441 Florida Avenue
 Modesto CA 95350
 Transfusion Service Department

Reviewed By: _____

Month/Year: _____

Date _____

BLOOD BANK			Date	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
			CLS's Initials																
Tube #	Reagents	QC Reagent	Daily Review: Initials																
			Expected Results																
1	Anti-A	CorQC Rgt Cells	2-4+	IS															
2	Anti-B	CorQC Rgt Cells	2-4+	IS															
3	Anti-A,B	CorQC Rgt Cells	2-4+	IS															
4	Mono Ctrl	CorQC Rgt Cells	0	IS															
5	Anti - D	CorQC Rgt Cells	2-4+	IS															
6	Anti - D	A1 Cells	0	IS/AHG	/	/	/	/	/	/	/	/	/	/	/	/	/	/	/
7	A1 Cells	CorQC Anti-Serum	2-4+	IS															
8	A2 Cells	CorQC Anti-Serum	2-4+	IS															
9	B Cells	CorQC Anti-Serum	2-4+	IS															
10	SCI	CorQC Anti-Serum	1-3+	AHG															
11	SCII	CorQC Anti-Serum	1-3+	AHG															
12	SCIII	CorQC Anti-Serum	1-3+	AHG															
13	SCI	Anti - A	0	IS/AHG	/	/	/	/	/	/	/	/	/	/	/	/	/	/	/
14	Check Cells M	Anti-IgG	1-4+	IS															
15	Check Cells P	Anti - IgG, -C3b,-C3d	1-4+	IS															
Reagent Rack - Appearance, Performance, Arrangement and Rgt Lot #s				S/U															

Reagent	Lot #	Exp. Date	Start Date	New Lot #	Exp Date	Start Date
Anti-A						
Anti-B						
Anti-A,B						
Anti-D						
Mono Ctrl						
A1 Cells						
A2 Cells						
B Cells						
Screen Cells						
LISS						
PEG						
Anti-IgG (AHG)						
Polyspecific AHG						
Check Cells						
CorQC Cells						
CorQC Serum						

Comments? Corrective Actions:

IS - Immediate Spin AHG - Antihuman Globulin
 Each CLS's will review the previous day's QC.
 Write your Initials in the "Daily Review" column
 All Reagents are Manufactured by Immucor
 P - Polyspecific
 S - Satisfactory
 M- Monospecific
 U - Unsatisfactory