

Sysmex Reagents

WNR lysercell and fluoroWNR Feb 2022

| Reagent | Discrete mode | |
|----------------------|-----------------|-----------------------|
| | CBC | CBC+WDF+WPC+RET+PLT-F |
| Total reagent volume | Approx. 29.9 mL | Approx. 62.8 mL |
| CELLPACK DCL | Approx. 27.9 mL | Approx. 53.7 mL |
| SULFOLYSER | Approx. 0.5 mL | Approx. 0.5 mL |
| Lysercell WNR | Approx. 1.5 mL | Approx. 2.5 mL |
| Fluorocell WNR | Approx. 20 µL | Approx. 20 µL |
| Lysercell WDF | - | Approx. 1.5 mL |
| Fluorocell WDF | - | Approx. 20 µL |
| Lysercell WPC | - | Approx. 1.5 mL |
| Fluorocell WPC | - | Approx. 20 µL |
| CELLPACK DFL | - | Approx. 3.0 mL |
| Fluorocell RET | - | Approx. 20 µL |
| Fluorocell PLT | - | Approx. 20 µL |

Lysercell WNR + Fluorocell WNR = NRBC channel + Basophils

WNR Lysercell™ WNR

Lyasing reagent for hematology analyzers



5 L
WNR-210A

AW341307-U

Mfg. Date 2020-03-24
Exp. Date 2021-03-24
Lot No. AM0007


Reagent Code




HEAD02338ECPG0710UELR00E



Remove center portion at the dashed lines.

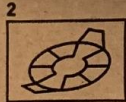


Lift up (*) tabs.

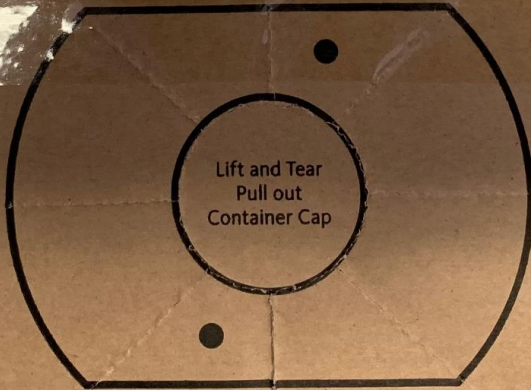

Lift and Tear
Pull out
Container Cap



Pull container cap through tabs.

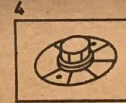

Replace (*) tabs under container cap.

Store in a dust free area

Lysercell WNR

5L per box

3333 sample per box

Store 2 - 35°C

Use 15 – 30°C

On-Board 90 days



WNR Lysercell™ WNR

REF ZPPBL121531

5 L

Storage and shelf life after first opening:
1. Once opened, this reagent should be used within 60 days.
2. Do not freeze.

Components:
Organic quaternary ammonium salts 0.20 %
Nonionic surfactant 0.10 %

Mfg. Date 2020-03-24
Exp. Date 2021-03-24
Lot No. AM0007

+2°C +35°C

AD515324-F



EAN/JAN



8 887843 000242

GSI:128

(0126856763333) (01210324) (0144507)



Sysmex Corporation

1-5-1 Wakinohama-Kaigandori,
Chuo-ku, Kobe 651-0073, Japan

Manufacturing site:

Sysmex Asia Pacific Pte Ltd.
19 Jalan Tukang Singapore 619265

MADE IN SINGAPORE

Sysmex is Registered in U.S. Pat. & Tm. Off.





www.sysmex-europe.com/ifu



EC REP
Sysmex Europe GmbH
Bornbarch 1,
22848 Norderstedt, Germany



Sysmex Corporation
1-5-1 Wakinohama-Kaigandori,
Chuo-ku, Kobe 651-0073, Japan
MADE IN JAPAN



H302
P301+P312
P501

WNR Fluorocell™ WNR
フルオロセル™ WNR



Lysercell WNR

2 x 82 ml cassette per box

20ul per test

4000 tests per cassette,
8000 tests per box

Store 2 - 35°C

Use 15 – 30°C

On-Board 90 days (44 tests per day)

