



Bar Code Blood Band

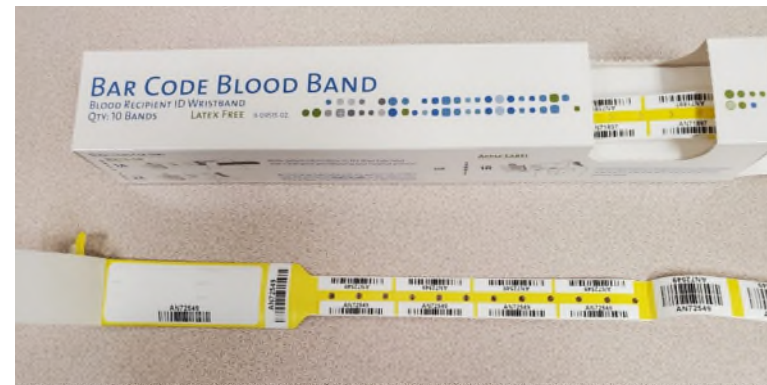

Directions for use for the new Bar Code Blood Band.

1. Remove one band from box and PRINT patient information on the draw tube label (larger section) with ball point pen (non-bleed pen) – or lift self-laminating shield from patient info area and place pre-printed patient label.

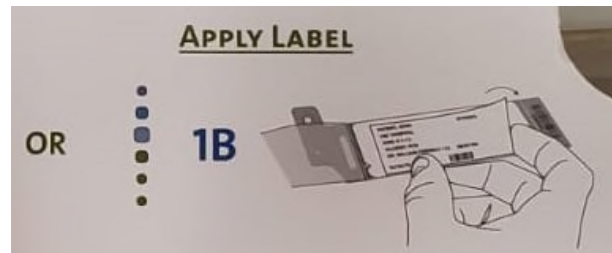

2. If written –starting right to left: remove draw tube label with patient info from wristband (Right to Left) so the carbon copy remains.



3. Apply patient information label to blood tube – ensure V-notch is aligned for visibility and specimen identification



Bar Code Blood Band

4. On a flat surface, peel away white liner from clear cover flap and press clear cover flap down firmly from left to right several times.



5. Separate draw tube tail from Bar Code Blood Band at the perforation; then remove the liner on the back of the tail of labels and attach to blood draw tube.



6. Apply Bar Code Blood Band to patient allowing one finger space between wrist and wristband closure.

