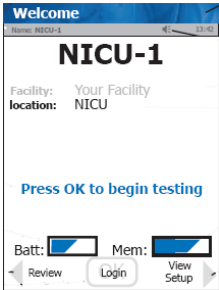
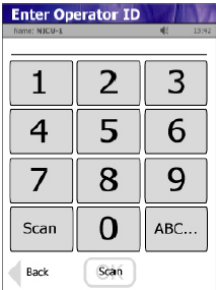


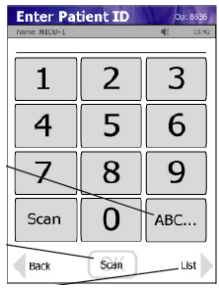

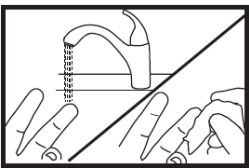
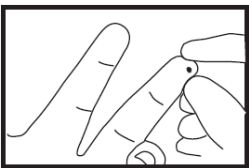
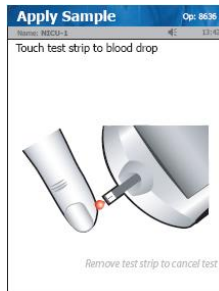
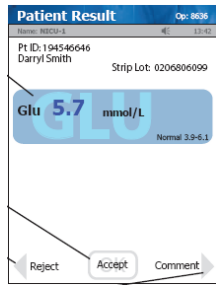





Nova Biomedical Glucometer
Quick Operating Guide AID
(Refer to POC20201)
POCT Manual
Page 1 of 1

Document Number:
POC20202
Issue Date:07-Jul-14
Revision Date: 07-Jul-16

 <p>1. From Home Screen, Press Login.</p>	 <p>2. Enter or scan Operator ID and press Accept.</p>	 <p>3. From Patient Test screen press Accept.</p>	 <p>4. Scan Strip lot no. and press Accept.</p>
 <p>5. Enter Patient ID and press Accept.</p>	 <p>6. Insert Test Strip into Meter.</p>	 <p>7. Wash patient's hand thoroughly, or use an alcohol wipe to clean puncture site. Ensure site is thoroughly dry.</p>	 <p>8. Use safety lancet to puncture finger. Gently squeeze finger to form a blood drop. Wipe away the first drop of blood.</p>
 <p>9. Touch strip to blood drop. Result will appear within 6 seconds.</p>	<p>10. WARNING!</p> <p>The test strip must fill completely upon touching the blood droplet. If the test strip does not fill completely, do not touch the blood droplet a second time. Discard the test strip and repeat the test with a new test strip.</p>	 <p>11. To accept result, press Accept. To reject result, press Reject.</p>	 <p>12. To review other results, press Review from Patient test screen.</p>

NOTE: This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: POC20202NovaBiomedicalGlucometerQuickOperatingGuideAID.doc

PRINT DATE: 7 July 2014