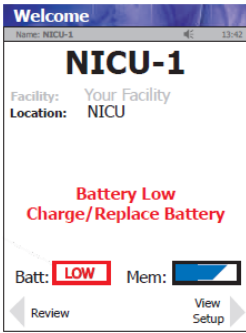
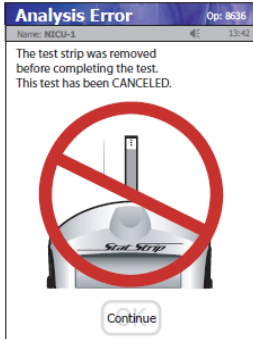
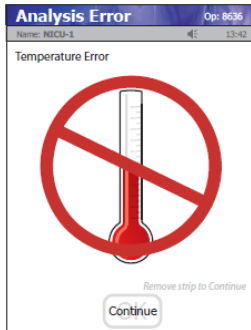

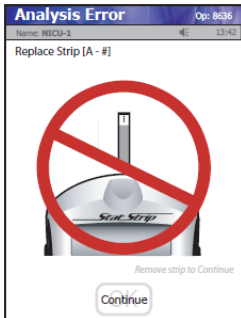
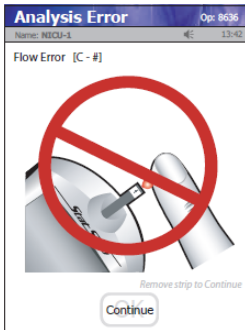
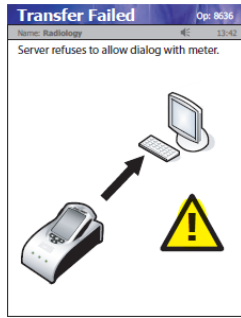





<p>1. Low Battery</p>  <p>Change battery and place meter into charging / docking station.</p>	<p>1. Test Strip Removed</p>  <p>Test has been cancelled</p>	<p>2. Temperature</p>  <p>Meter will only work in a temperature range of 15° - 40° C</p>	<p>3. Bad sample</p>  <p>Insert new strip and repeat test.</p>
<p>4. Strip Rejected</p>  <p>Occurs after test strip insertion or during analysis. Insert another strip and repeat the test.</p>	<p>5. Flow Error</p>  <p>Either insufficient sample or the sample was applied incorrectly. Repeat test with a new strip.</p>	<p>6. Transfer Failed</p>  <p>Meter is unable to connect to the transfer computer. Check network settings, network status, or contact your administrator for assistance.</p>	<p>7. Transfer Failed</p>  <p>Meter was removed before data transfer was complete. Re-dock the meter.</p>

PROBLEMS? Try restarting the instrument by removing the battery and putting it back in
Still problems? Call the laboratory and ask to speak to the Point of Care Testing Coordinator

NOTE: This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.