

Document Name: Urine Culture

Approved By:

Jennifer G. Daley Bernier, A/Manager, Laboratory Services

Status: **APPROVED**

## PURPOSE:

To determine the presence or absence of bacterial pathogens in urine specimens.

## SAMPLE INFORMATION:

Early-morning specimens are preferable – allowing urine to remain in the bladder for at least 4 hours will decrease the number of false – negative results.

<b>Type</b>	The urine specimens submitted to the laboratory for culture are either: <ul style="list-style-type: none"> <li>• Fresh urine collected in sterile container.</li> <li>• Fresh urine collected in urine transport tube.</li> </ul>	
<b>Source</b>	Voided Urines (non-sterile)	<ul style="list-style-type: none"> <li>• Midstream urine (MSU)</li> <li>• Neonatal bagged urine</li> <li>• Indwelling catheter (Foley) urine</li> <li>• Ileal conduit urine</li> </ul>
	Aseptically collected Urines (sterile)	<ul style="list-style-type: none"> <li>• Straight or "in and out" catheter</li> <li>• Nephrostomy urine</li> <li>• Cystoscopy urine</li> <li>• Suprapubic bladder aspirate</li> </ul>
<b>Stability</b>	<ul style="list-style-type: none"> <li>• Fresh urine collected in sterile container is acceptable for 24 hours if refrigerated.</li> <li>• Fresh urine collected in urine transport tube is acceptable for 72 hours (refrigeration not necessary).</li> </ul>	
<b>Storage Requirements</b>	<ul style="list-style-type: none"> <li>• Fresh urine without preservative should be refrigerated until processing.</li> <li>• Fresh urine collected in urine transport tube can be kept at room temperature.</li> </ul>	
<b>Criteria for rejection and follow up action</b>	<ul style="list-style-type: none"> <li>• Unrefrigerated fresh urine received longer than 2 hours after collection.</li> <li>• Refrigerated fresh urine specimens received longer than 24 hours after collection.</li> <li>• 24 hour urine collections.</li> <li>• Duplicate specimens obtained with same collection method within 48 hours.</li> <li>• Foley catheter tips.</li> <li>• Urine from the bag of a catheterized patient.</li> <li>• Specimens in leaking container and unlabeled specimens.</li> </ul>	

**NOTE:** This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

<b>Document Name: Urine Culture</b>	<b>Document Number: MIC31800</b>	
	<b>Version No: 3.0</b>	<b>Page: 2 of 7</b>
	<b>Effective: 28 April, 2017</b>	

**REAGENTS and/or MEDIA:**

- Blood Agar and MacConkey Agar
- Identification reagents: catalase, oxidase, spot indole, PYR, tube coagulase, etc.

**SUPPLIES:**

- 0.001 mL loops
- Vitek 2 Compact and supplies
- Biosafety cabinet
- 35° ambient air incubator

**SPECIAL SAFETY PRECAUTIONS:**

Containment Level 2 facilities, equipment, and operational practices for work involving infectious or potentially infectious materials or cultures.

- Lab gown must be worn when performing activities with potential pathogens.
- Gloves must be worn when direct skin contact with infected materials is unavoidable.
- Eye protection must be used where there is a known or potential risk of exposure to splashes.
- All procedures that may produce aerosols, or involve high concentrations or large volumes should be conducted in a biological safety cabinet (BSC).
- The use of needles, syringes, and other sharp objects should be strictly limited.

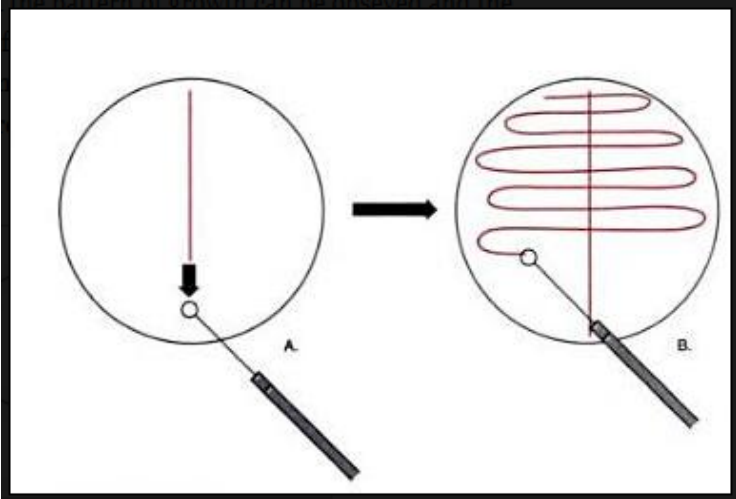
All patient specimens are assumed to be potentially infectious. Universal precautions must be followed. Since viable micro-organisms are used, all cultures must be handled with appropriate precautions. All equipment in contact with cultures should be decontaminated by appropriate methods.

**QUALITY CONTROL:**

Refer to Quality Control procedures as required for individual identification and susceptibility procedures used in evaluating and testing.

<b>NOTE:</b> This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.	
FILENAME: MIC31800.3UrineCulturePRO.doc	Print Date: 4/28/2017 4:44:00 PM

**PROCEDURE INSTRUCTIONS:**

Step	Action	
Follow the steps in the table below to set-up the culture for a urine specimen		
1	<ul style="list-style-type: none"> <li>• Hold a 0.001 mL loop vertically and immerse just below the surface of a well-mixed urine specimen.</li> <li>• Deliver a loopful of urine onto appropriate agar plates.</li> <li>• Make a straight line down the center of the plate.</li> <li>• Streak the urine by making a series of passes at 90° angles through the inoculum.</li> </ul> <div style="text-align: center;">  </div> <p>***Note: the same loop can be used for each plate per patient sample. The loop must be re-dipped for every plate.</p>	
2	If:	Then:
	<u>Voided Urines (non-sterile):</u> <ul style="list-style-type: none"> <li>• Midstream urine (MSU)</li> <li>• Neonatal bagged urine</li> <li>• Indwelling catheter (Foley) urine</li> <li>• Ileal conduit urine</li> </ul>	<ul style="list-style-type: none"> <li>• Inoculate BAP and MAC</li> <li>• Incubate plates for <b>18-24 hours</b> at 35° in the ambient air incubator</li> </ul>
	<u>Aseptically collected Urines (sterile):</u> <ul style="list-style-type: none"> <li>• Straight, intermittent or "in and out" catheter</li> <li>• Nephrostomy urine</li> <li>• Cystoscopy urine</li> </ul>	<ul style="list-style-type: none"> <li>• Inoculate BAP and MAC</li> <li>• Incubate plates for <b>48 hours</b> at 35° in the ambient air incubator</li> </ul>

**NOTE:** This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

- Suprapubic bladder aspirate

**INTERPRETATION OF RESULTS:**

**Reporting Results:**

- Using a 0.001 mL loop, 1 colony equals 1 X E6 cfu/L.
- Determine the colony count and extent of the work-up required for each morphotype on the plates.

**Type of Urine:** Non-sterile urine

Colony count	1 uropathogen	2 uropathogens	3 or more uropathogens
≤ 10 (≤ 10 X E6 cfu/L)	No workup; Report: <b>“No Significant Growth”</b>	No workup; Report: <b>“No Significant Growth”</b>	No workup; Report: <b>“No Significant Growth”</b>
11-99 (11-99 X E6 cfu/L)	ID and susceptibility testing	ID and susceptibility testing on both	No workup; Report: <b>“}CON1 Mixed Culture, Repeat”</b>
≥ 100 (≥ 100 X E6 cfu/L)	ID and susceptibility testing	ID and susceptibility testing on both	No workup; Report: <b>“}CON1 Mixed Culture, Repeat”</b>
Uropathogens >10 XE6 cfu/L and other isolates ≤10 XE6 cfu/L	ID and susceptibility on isolate >10 XE6 cfu/L and ignore isolates ≤ 10 XE6 cfu/L	ID and susceptibility on both isolates >10 XE6 cfu/L and ignore isolates ≤10 XE6 cfu/L	No workup; Report: <b>“}CON1 Mixed Culture, Repeat”</b>
>10 XE6 cfu/L Normal skin/urogenital flora	No workup; Report: <b>“Usual Urogenital Flora”</b>	No workup; Report: <b>“Usual Urogenital Flora”</b>	No workup; Report: <b>“Usual Urogenital Flora”</b>

**Type of Urine:** Sterile urine

Colony Count	Any number of morphotypes
Any growth	Perform ID and susceptibility testing
No growth after 48 hour incubation	<b>“}NG2D”</b>

**NOTE:** This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

**List of Uropathogens and Non-Uropathogens:**

Uropathogens	Non-uropathogens (normal skin/urogenital flora)
Enterobacteriaceae	<i>Lactobacillus</i>
<i>Pseudomonas aeruginosa</i>	Diphtheroids (not <i>C.urealyticum</i> )
Other gram negative bacilli	Viridans Streptococci ( not <i>A.urinae</i> )
<i>Enterococcus</i> species	Bacillus species
<i>Streptococcus pyogenes</i>	<i>Neisseria spp.</i>
<i>Streptococcus agalactiae</i>	
Yeast	
<i>Aerococcus urinae</i> *	
<i>Corynebacterium urealyticum</i>	
<i>Staphylococcus aureus</i>	
<i>Staphylococcus saprophyticus</i>	
Other coagulase negative staphylococcus	

\* Considered a uropathogen only if colony count is 10 times greater than that of all other microbiota.

ORGANISM	BIOCHEMICAL TESTS & EXPECTED RESULTS	ID & SENS
E.coli	LF, flat, deep pink Spot Indole: Positive Oxidase: Negative PYR: Negative	Vitek AST- N213
Enterobacteriaceae	LF or NLF Spot Indole: Negative/Positive Oxidase: Negative PYR: Positive/Negative	Vitek GN & AST-N213
P.aeruginosa	NLF, typical smell/morphology Spot Indole: Negative Oxidase: Positive	Vitek GN & AST-N213
Non-fermenters	NLF Oxidase: Positive/Negative	Vitek GN & AST-N213
Group A Strep	Beta hemolysis GAS latex: Positive	Add comment: "This organism is generally susceptible to beta lactam antibiotics"
Group B Strep	Small or no zone of beta hemolysis GBS latex: Positive	Add comment: "This organism is generally susceptible to beta-lactam antibiotics. This organism is resistant to TMP-SMX".
S.aureus	Catalase Positive, Slide Coagulase Positive, Tube Coagulase Positive	Vitek AST-GP67
S. saprophyticus	Catalase Positive, Slide Coagulase Negative, Novobiocin Resistant	Add comment: "This organism is generally susceptible to TMP-SMX, nitrofurantoin and urinary quinolones"
Enterococcus	Catalase Negative PYR Positive	Vitek AST-GP67
Yeast	Wet prep = yeast	Vitek YST ID card:
Aerococcus urinae	Resembles viridans Streptococcus Gram stain: GPC, tetrads Catalase: Negative PYR: Negative	Vitek GP Perform susceptibility testing as per DynaLIFE ASTM
Diphtheroides	Catalase positive <ul style="list-style-type: none"> <li>Rule out uropathogenic Corynebacterium urealyticum (urea+) if numbers are significant.</li> </ul>	Urea

**NOTE:** This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

**Limitations:**

- A mixed culture in an uncomplicated outpatient population likely indicates contamination.
- For uncomplicated UTI, culture is usually not indicated.
- False-negative results may be due to interfering substances, diluted urine, low urine pH and subjective interpretation of the criteria for further workup of the culture.

**REFERENCES:**

Clinical Microbiology Procedures Handbook, 4<sup>th</sup> edition, ASM Press, 2016  
 Jorgensen J.H., Pfaller M.A., Carroll K.C., Funke G., Landry M.L., Richter S.S., Warnock D.W. 2015. Manual of Clinical Microbiology, 11<sup>th</sup> edition, ASM Press, Washington, D.C.  
 J Obs Gyn Can 2004; 26(9):826-32

**REVISION HISTORY:**

REVISION	DATE	Description of Change	REQUESTED BY
1.0	24-Nov-10	Initial Release	M-L Dufresne
2.0	23-Dec-16	Updated to new template; Procedure updated to remove UriCult; Computer details changed to reflect practice for SoftMic SCC SoftComputer.	L. Steven
3.0	28 Apr 2017	Updated number; Changed Logo	JGD Bernier

 Invalid signature

X 

---

Jennifer G. Daley Bernier, R.T. (CSMLS)  
 A/ Manager, Laboratory Services  
 Signed by: Jennifer G. Daley Bernier

**NOTE:** This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.