

**Document Name:**

ABO Confirmation (Patient Retype) Workflow

**Approved By:**

Jennifer G. Daley Bernier, A/ Manager, Laboratory Services

**Status:** **APPROVED**

## **ABO & Rh Confirmation Testing - Change in workflow**

**ABO & RH testing will now need to be done by two different technologists.**

### For Routine Testing

- The 1000-1800 Bloodbanker (or covering MLT) will perform the CONF test and the Antibody screen
- The sample will be put in the ABORH to be Done rack in the Top of the Blood Bank fridge
- The 1500-2300 Evening Technologist will at some point between 1500-1700 Do any ABORH testing that is in the rack in the Top of the Blood Bank fridge
- Any routine samples the evening person receives will be processed same as the day shift – the ABORH testing will be put in the rack for the 1000-1800 person to perform the following day
- When done – samples filed as usual in bottom of fridge

### For STAT Testing

- When there is more than one technologist available – grab to do stat ABORH testing
- When there is only one technologist working/available
  - o perform both CONF and ABORH testing and result
  - o put sample in the ABORH to be Done rack.
  - o 2<sup>nd</sup> technologist to order 2<sup>nd</sup> CONF test and perform testing.

**NOTE:** This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.