

# **Stanton Territorial Hospital**

P.O. Box 10, 550 Byrne Road YELLOWKNIFE NT X1A 2N1

### **Document Name:**

Rejection Criteria Job Aid

## Distribution:

Microbiology Specimen Processing Manual

**Document Number:** 

MIC10261

Date Issued:

15 August 2018

	CANCELLATION DUE TO LABELLING ISSUES	
Specimen unlabeled	"Specimen not labeled. Not suitable for processing" *Excluding irretrievable specimens*	XUNL
Specimen mislabeled	"Specimen mislabeled. Not suitable for processing" *Excluding irretrievable specimens*	XMIS
Discrepancy between requisition and	"Specimen container label does not match patient identification on requisition. Not suitable for	XUML
specimen	processing" *Excluding irretrievable specimens*	
	CANCELLATION DUE TO SPECIMEN COLLECTION ISSUES	
Leaking container	"Specimen leaked in transport. No testing was performed. Please resubmit for testing"	XLIT
	*Excluding irretrievable specimens*	ALII
Insufficient quantity received	"Insufficient quantity for testing. Please resubmit if still indicated"	XIQ
	*For irretrievable specimens, phone physician to prioritize sample received*	
Duplicate specimens within 24 hours:	"Duplicate specimen collected on same day. Previous specimen will be tested"	
urine, sputum, stool, MRSA/VRE, GBS,	*Excluding urine or sputum specimens collected by a different method or MRSA/VRE collected	XDUP
throat and BV	from a different location*	
	CANCELLATION DUE TO AGE OF SPECIMEN	
Orange top urine, > 24 hours old	"Specimen received 24 hours after collection. Not suitable for processing. Please repeat collection if	X24H
	still indicated"	
Water samples, > 48 hours old	"Specimen received 48 hours after collection. Not suitable for processing. Please repeat collection if	X48H
	still indicated" *Endotoxin water is rejected if > 24 hours old*	
Blue top urine, > 72 hours old	"Specimen received 3 days after collection. Not suitable for processing. Please repeat collection if still	X72H
	indicated"	71
Stool culture, >72 hours old	"Specimen received 3 days after collection. Not suitable for processing. Please repeat collection if still	X72H
		\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\
otool culture, 712 flours old	indicated"	
Sputum culture, > 72 hours old	"Specimen received 3 days after collection. Not suitable for processing. Please repeat collection if still	X72H

NOTE: This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC10261RejectionCriteriaJobAidAID

PRINT DATE: 3/7/2019 10:50:00 AM



# **Stanton Territorial Hospital**

P.O. Box 10, 550 Byrne Road YELLOWKNIFE NT X1A 2N1

### **Document Name:**

Rejection Criteria Job Aid

## Distribution:

Microbiology Specimen Processing Manual

**Document Number:** 

MIC10261

Date Issued:

15 August 2018

	CANCELLATION OF STOOL SAMPLES	
Stool on patient hospitalized >3 days	"This specimen was not cultured for community acquired enteric pathogens because the patient has	XSTL
	been hospitalized for 3 or more days. Clostridium difficile toxin assay is recommended"	
Stool not received in EPT and >2 hours	"Specimen not received in Enteric Transport Medium. Unsuitable for C&S"	XNEP
old		
	CANCELLATION OF C.DIFF SAMPLES	
C.diff 3 days old, unless frozen	"Specimen received 3 days after collection. Not suitable for processing. Please repeat collection if still	X72H
	indicated"	X/ZII
C.diff on formed stool	"Formed stool received. Analysis not performed"	XCDT
Positive C.diff within 7 days	"Testing not indicated. Patients with previous positive test should not have repeat testing for cure,	XCDP
	unless they again come symptomatic after completion of therapy"	
Neg. C.diff within 7 days	"Testing not indicated. A single negative test is sufficient to rule out C.difficile"	XCDN
Rectal swab received for C.diff	"Rectal swabs unsuitable for C.difficile testing. Please resubmit a stool specimen"	XCDS
C.diff requested for child <1 year old	"Testing for C.difficile should not be performed in children under 1 year old"	X<12
CAN	NCELLATION OF HAY RIVER AND FORT SMITH VAGINAL SWABS	
Genital culture ordered but no	"Routine genital cultures will not be performed unless testing is requested AND relevant clinical history	VOEN
significant clinical information provided	is provided."	XGEN
Yeast culture ordered but no significant clinical information provided	"Routine yeast cultures will no longer be performed unless there is a clinical history of recurrent vaginal	XYST
	candidiasis, treatment failure and/or immunosuppression, particularly HIV infection. Microscopic	
	examinations will continue for the detection of yeast and, if seen, will be reported."	

NOTE: This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC10261RejectionCriteriaJobAidAID

PRINT DATE: 3/7/2019 10:50:00 AM