



Order standing order in LIS:

- Patient Last Name: **TEST**
- Patient First Name: **MICRO STERILITY**

- **CXENV** will be ordered
- **Collect, receive and plate**
- **MICRO OE Comments:** KB or Vitek Dispensette

Label 2 THIO broths with specimen labels

Dispense 3mL from each dispensette into corresponding THIO broth

Read each broth daily for 3 days

THIO broth is cloudy (indicates growth)

Record: **"Cloudy-AUTOCLAVE dispenser ^CLAVE"**

Report: **"Cloudy-AUTOCLAVE dispenser"** and finalize

Discard saline bottle and autoclave dispensette.

THIO broth is clear (indicates no growth)

Record: **"No Growth"** in LIS daily for 3 days

Report: **"No Growth After 3 Days"** and finalize

NOTE: This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.