

Stanton Territorial Hospital

P.O. Box 10, 550 Byrne Road YELLOWKNIFE NT X1A 2N1

Maintenance Record - Vitek 2

Document Name:

Maintenance Record - Vitek 2

Distribution:

Microbiology Instrumentation Manual

Document Number:

MIC70310

Date Issued:

17 Nov 2017

Month: _____ Year: ____

	1	2	3	4	5	6	7	8	9	1 0	1	1 2	1 3	1 4	1 5	1 6	1 7	1 8	1 9	2	2 1	2	2	2 4	2 5	2 6	2 7	2 8	2 9	3 0	3 1
Daily Maintenance			,	,															,												
Empty waste tray																															
Check status																															
Zero DensiCHEK																															
Check printer paper																															
Monthly Maintenance																															
Task	С	Date Tech			h	Task									Da	ate	Т	ech	ı	1				Task					e	Tech	
Clean boats						Clean base pan												(Clean keyboard												
Clean carousels				Clean drip pan															(Change pipette tips											
Clean cassettes				Clean vacuum seal															(Change saline bag											
Clean optics				Clean vacuum cham															1	Disp/pipette volume test											
Optics diagnostic test				Clean Smart Carrie									ation						9	Sterilize dispensette											
Clean waste tray		Clean computer and									nd	scanner															•				
DensiCHEK plus Calibra	tio	n																													
Calibration Standard	Reference Range										Lot #					ŧ .			Expiry			Result				Dat	е	Te	ch		
0.5				(0.44	4 to 0.56																									
2.0		1.85 to 2.15																													
3.0		2.79 to 3.21																													
Corrective action	ı: _													R	evie	ewe	d by	/: _													
NOTE: This is a CO electronic version prio			D do	ocum	ent f	or int	terna	l use	only	. An	y do	cume	ents a	appea	aring	in pa	aper	form	are ı	not co	ontro	lled a	and s	hould	d be	chec	ked	again	st		
FILENAME: MIC7031	0Maiı	ntena	inceF	Recor	d-Vit	ek2F	OR.c	locx												PRIN	IT DA	ATE:	6/14/	2021	12:0	4:00	PM				