

PROGRAM Standard Operating Procedure – Laboratory Services	
Title: MIC53300 – Motility Test	Policy Number:
Program Name: Laboratory Services	
Applicable Domain: Lab, DI and Pharmacy Services	
Additional Domain(s):	
Effective Date:	Next Review Date:
Issuing Authority: Director of Health Services	Date Approved:
Accreditation Canada Applicable Standard: N/A	

GUIDING PRINCIPLE:

The motility test is used to detect the presence of flagella by bacteria microscopically.

PURPOSE/RATIONALE:

This standard operating procedure describes how to perform the motility test.

SCOPE/APPLICABILITY:

This procedure applies to Medical Laboratory Technologists (MLTs) performing the motility test.

SAMPLE INFORMATION:

Type	Few, fresh, well isolated colonies
-------------	------------------------------------

REAGENTS and/or MEDIA:

- Thioglycollate broth (THIO)

SUPPLIES:

- Disposable loops
- Wooden sticks
- Sterile pipettes
- Glass microscope slides
- Cover slips
- Sterile saline

EQUIPMENT:

- 35° ambient air incubator

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

SPECIAL SAFETY PRECAUTIONS:

Containment Level 2 facilities, equipment, and operational practices for work involving infectious or potential infectious materials or cultures.

- Ensure that appropriate hand hygiene practices be used.
- Lab gown must be worn when performing activities with potential pathogens.
- Gloves must be worn when direct skin contact with infected materials is unavoidable.
- Eye protection must be used when there is a known or potential risk of exposure of splashes.
- All procedures that may produce aerosols, or involve high concentrations or large volumes should be conducted in a biological safety cabinet (BSC).
- The use of needles, syringes and other sharp objects should be strictly limited.

All patient specimens are assumed to be potentially infectious. Routine Practices must be followed. Since viable micro-organisms are used, all cultures must be handled with appropriate precautions. All equipment in contact with cultures should be decontaminated by appropriate methods.

QUALITY CONTROL:

- Quality control performed as tested:
 - Positive: *Escherichia coli* ATCC 25922
 - Negative: *Klebsiella pneumoniae* ATCC 700603
- A TQC order is automatically generated when test is ordered to record the QC results

PROCEDURE INSTRUCTIONS:

Step	Action
Performing the motility test	
1	Use fresh growth from an agar plate and suspend isolated colonies into the Thioglycollate broth. Use a light inoculum (not visibly turbid) NOTE: It is acceptable to suspend the organism in a small amount of saline for an initial wet mount, but follow with incubation of Thioglycollate broth if the result is negative.
2	Incubate the Thioglycollate broth as follows: <ul style="list-style-type: none">• <i>Listeria</i> and <i>Enterococcus</i> spp. for 2 to 4 hours at room temperature• Non-fermenting, Gram-negative rods for 24 hours at room temperature• Other organisms, including <i>Bacillus</i> spp. for 2 to 24 hours at 35°C
3	While wearing gloves, place a small drop of the broth on the center of a microscope slide. Add a coverslip and allow the organism to "settle" for a minute.
4	Move the condenser out of the path of illumination and observe under high power magnification (40X).

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

INTERPRETATION OF RESULTS:

IF	THEN
Directional and purposeful motility	Motility Test = Positive
No directional movement observed	Motility Test = Negative

LIMITATIONS:

1. *Bacillus* spp. is best tested directly from a fresh plate. If a fresh plate is not available, inoculate a plate and incubate for 4 hours and then perform wet mount.
2. *Bacillus* spp. should be motile. Lack of motility could indicate *Bacillus anthracis*.
3. *Enterococcus casseliflavus* and *Enterococcus gallinarum* are motile.
4. *Listeria* organisms are motile at room temperature but not at 35°C, with tumbling motility.
5. *Yersinia enterocolitica* is motile at room temperature but not at 35°C.
6. *Campylobacter* spp. will display darting motility.
7. Excessive heat on microscope can affect the results.

REFERENCES:

1. Clinical Microbiology Procedures Handbook, 4th edition, ASM Press, 2016

APPROVAL:

Date

REVISION HISTORY:

REVISION	DATE	Description of Change	REQUESTED BY
1.0	05 Apr 19	Initial Release	L. Steven
2.0	30 Aug 21	Procedure reviewed and added to NTHSSA policy template	L. Steven

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.