

PROGRAM Standard Operating Procedure – Laboratory Services	
Title: MIC53500 – CAMP/Reverse CAMP Test	Policy Number:
Program Name: Laboratory Services	
Applicable Domain: Lab, DI and Pharmacy Services	
Additional Domain(s):	
Effective Date:	Effective Date:
Issuing Authority: Director, Health Services	Date Approved:
Accreditation Canada Applicable Standard: N/A	

GUIDING PRINCIPLE:

The CAMP test is used in the identification of *Streptococcus agalactiae* and many Gram-positive rods, including *Listeria monocytogenes*. The reverse CAMP test is used for the identification of *Arcanobacterium haemolyticum*.

PURPOSE/RATIONALE:

This standard operating procedure describes how to perform the CAMP/Reverse CAMP test.

SCOPE/APPLICABILITY:

This procedure applies to Medical Laboratory Technologists (MLTs) performing the CAMP/Reverse CAMP test.

SAMPLE INFORMATION:

Type	One, well isolated colony
------	---------------------------

REAGENTS and/or MEDIA:

- Blood agar (BA)

SUPPLIES:

- QC organism *Staphylococcus aureus* ATCC 25923
- QC organism *Streptococcus agalactiae* ATCC 12386
- QC organism *Streptococcus pyogenes* ATCC 19615
- Disposable loops

EQUIPMENT:

- 35° ambient air and 37° CO₂ incubators

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

SPECIAL SAFETY PRECAUTIONS:

Containment Level 2 facilities, equipment, and operational practices for work involving infectious or potential infectious materials or cultures.

- Ensure that appropriate hand hygiene practices be used.
- Lab gown must be worn when performing activities with potential pathogens.
- Gloves must be worn when direct skin contact with infected materials is unavoidable.
- Eye protection must be used when there is a known or potential risk of exposure of splashes.
- All procedures that may produce aerosols or involve high concentrations or large volumes should be conducted in a biological safety cabinet (BSC).
- The use of needles, syringes and other sharp objects should be strictly limited.


All patient specimens are assumed to be potentially infectious. Routine Practices must be followed. Since viable micro-organisms are used, all cultures must be handled with appropriate precautions. All equipment in contact with cultures should be decontaminated by appropriate methods.

QUALITY CONTROL:

- Quality control performed as tested:
 - Positive: *Streptococcus agalactiae* ATCC 12386
 - Negative: *Streptococcus pyogenes* ATCC 19615
- A TQC order is automatically generated when test is ordered to record the QC results

PROCEDURE INSTRUCTIONS:

Step	Action
Performing the CAMP test	
1	Streak <i>Staphylococcus aureus</i> ATCC 25923 in a straight line across the center of a Blood agar plate.
2	Streak the unknown organism in the same manner perpendicular to but NOT touching the <i>Staphylococcus</i> streak.
3	Streak the positive control organism parallel to and approximately 2.5 cm from the unknown organism.
4	Streak the negative control organism in the same way on the opposite side of the <i>Staphylococcus</i> streak.
5	Label the identification of each streak on the back of the agar plate.
6	Incubate the plate overnight at 35° in the CO ₂ incubator. NOTE: Refrigeration may enhance the reaction after incubation.

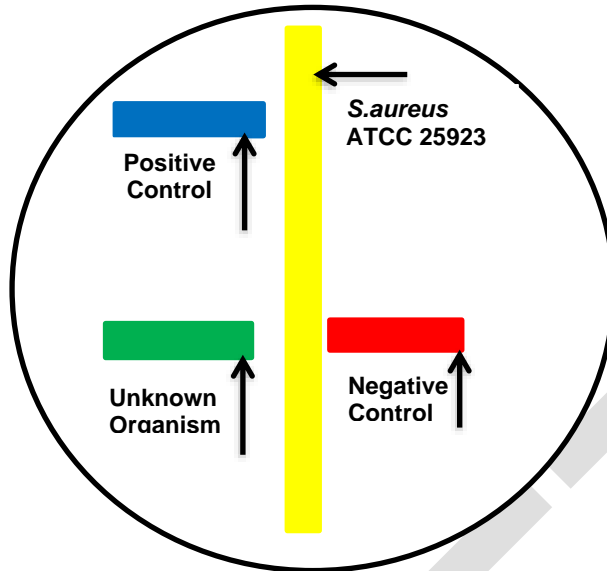

INTERPRETATION OF RESULTS:

IF	THEN
Distinct arrowhead of hemolysis at the intersection of the <i>Staphylococcus</i>	CAMP Test = Positive
No enhanced hemolysis at the intersection of the <i>Staphylococcus</i>	CAMP Test = Negative
Distinct arrow of no hemolysis at the intersection of the two haemolytic organisms	Reverse CAMP Test = Positive

PROCEDURE NOTES:

Gram-positive bacilli that are CAMP test positive:

- *Rhodococcus equi*
- *Listeria monocytogenes*
- *Propionibacterium avidum/granulosum*
- *Actinomyces neuii*
- *Turicella otitidis*
- *Corynebacterium glucuronolyticum*
- *Corynebacterium colyaeae*
- *Corynebacterium imitans*
- Some strains of *Corynebacterium striatum* and *Corynebacterium afermentans* group

Organisms that are reverse CAMP test positive:

- *Corynebacterium pseudotuberculosis*
- *Corynebacterium ulcerans*
- *Arcanobacterium haemolyticum*
- *Clostridium perfringens*

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

LIMITATIONS:

1. The test is 98% sensitive in detecting *Streptococcus agalactiae*. Isolates with a negative CAMP test may still be *Streptococcus agalactiae* and require further testing.
2. Increased nonspecific hemolysis at the intersections (a *matchstick* effect) may be seen with other Streptococci, but only *Streptococcus agalactiae* produces a definite arrowhead.
3. *Streptococcus pyogenes* can give a reaction that may be interpreted as positive, but it is PYR positive.
4. The CAMP Test separates *Listeria monocytogenes* from most other *Listeria* species.
5. If the agar is too thin or hemolysed, the reaction may be very weak.

REFERENCES:

1. Clinical Microbiology Procedures Handbook, 4th edition, ASM Press, 2016

APPROVAL:

Date

REVISION HISTORY:

REVISION	DATE	Description of Change	REQUESTED BY
1.0	05 Apr 19	Initial Release	L. Steven
2.0	30 Jun 21	Procedure reviewed and added to NTHSSA policy template	L. Steven

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.