Issuing Authority: Director of Health Services Next Review Date: Type: Laboratory Services Program SOP

Policy Number: Date Approved:

PROGRAM Standard Operating Procedure – Laboratory Services				
Policy Number:				
Program Name: Laboratory Services				
Applicable Domain: Lab, DI and Pharmacy Services				
Additional Domain(s):				
Next Review Date:				
Date Approved:				
Accreditation Canada Applicable Standard:				

GUIDING PRINCIPLE:

Organisms that cannot be identified or require further susceptibility testing need to be referred to *Dyna*LIFE (DL). TDG regulations need to be followed including paperwork and packaging.

PURPOSE/RATIONALE:

To ensure microbiology Category B specimens are sent for referral testing to *Dyna*LIFE appropriately.

SCOPE/APPLICABILITY:

This procedure applies to Medical Laboratory Technologists (MLTs) and Medical Laboratory Assistants (MLAs) packaging Category B samples being referred to *Dyna*LIFE.

SAMPLE INFORMATION:

The following category B specimens need to be sent to DL for referral testing:

- Any organism that cannot be identified
- Any susceptibility testing that cannot be performed at Stanton microbiology laboratory or that requires confirmation

SUPPLIES:

- Copan Transystem Culture swab transport system (with or without charcoal)
- DynaLIFE Referred-In Isolate Submission Form
- Category B box
- Category B package supplies
- DL RG2 Biosafety fax coversheet

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

Policy Number: Page 1 of 5


Issuing Authority: Director of Health Services


Next Review Date:

Type: Laboratory Services Program SOP Policy Number:

Date Approved:

PROCEDURE INSTRUCTIONS:

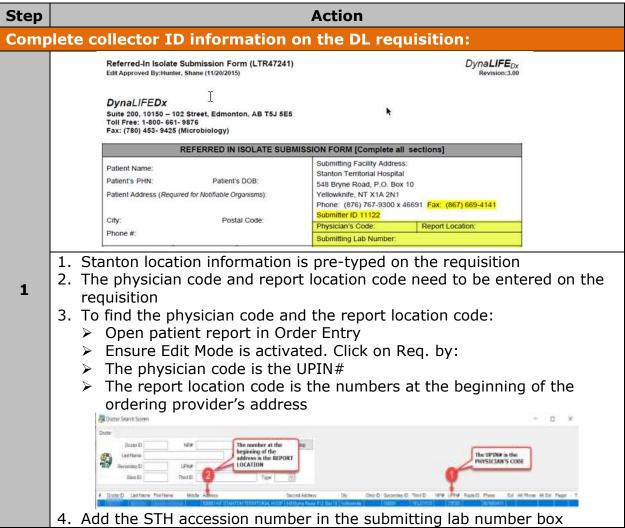

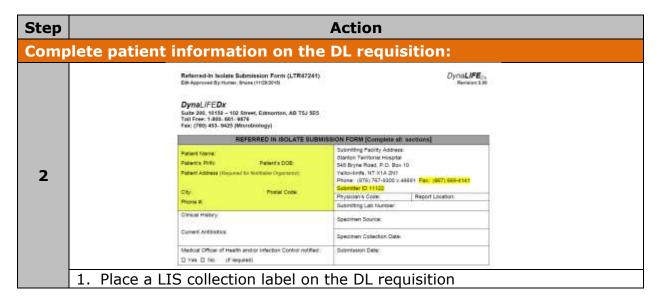

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.


Policy Number: Page 2 of 5


Issuing Authority: Director of Health Services

Next Review Date:

Type: Laboratory Services Program SOP

Policy Number: Date Approved:

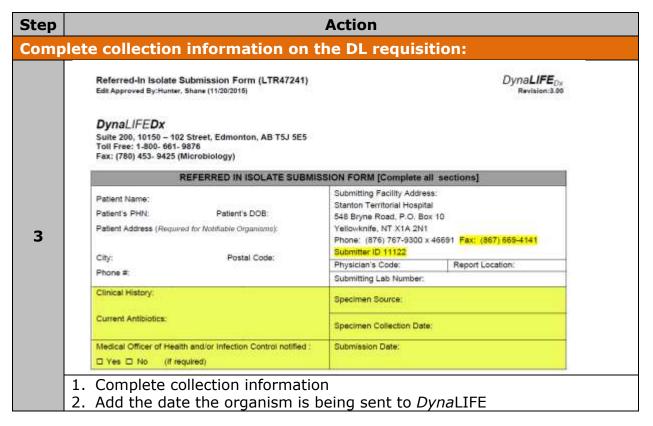

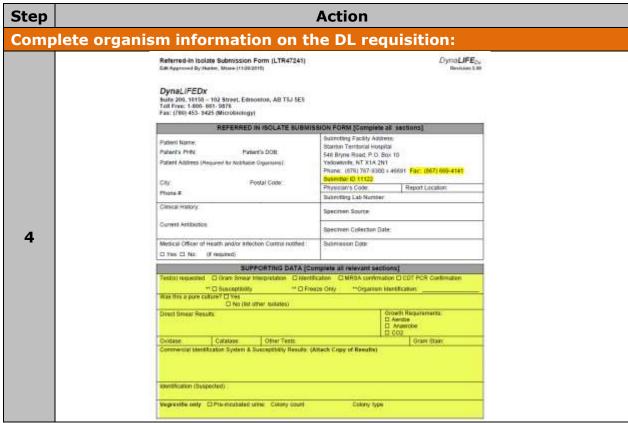

Disclaimer Message: This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

Issuing Authority: Director of Health Services

Next Review Date:

Type: Laboratory Services Program SOP

Policy Number: Date Approved:

Step	Action							
Complete highlighted areas on the DL RG2 coversheet:								
5	Stanton Territorial Hospital – Microbiology Laboratory 548 Byrne Road, PO Box 10 - Yellowknife, NT X1A 2N1 Phone: 867 767-9300 [ext.46691] Fax: 867 669-4141 Tec DynaLile Dx Microbiology Laboratory Fax: 1 (780) 453 9425 Phone: 1 (800) 661 9876 Date:							
	Rec Transfer of Human Pathogens (L-R2-09731-22-BK-00) CC: att. biosafety@gav.nt.ca							
	STH Microbiology laboratory is transferring a package containing a Risk Group 2 Human pathogen to your facility. The sample is to be directed to the							
	Name of STH employee sending sample. Signature							

Step	Action				
Order a referred test in result entry screen:					
6	 Select Add Test Choose appropriate reference code: ?REFD- DynaLIFE Select Ok Add a period (.) in the results line Final report the test line 				

Step	Action							
Comp	plete Category B send-out process:							
7	 Print off any Vitek results for organism being referred Submit organism on a labelled agar plate sealed with parafilm or on a labelled C&S swab Photocopy all papers and staple together. Hole punch photocopy stack and place in the <i>Dyna</i>LIFE and ProvLab Pending Referrals binder. Ensure the report is filed under the correct tab (<i>Dyna</i>LIFE, Cat B) Place a requisiton label on the DL requisition and scan into SoftMedia Pack specimen up according to TDG Category B regulations, send original paperwork with specimen Email completed coversheet to email locations listed Place Category B box in the Category B overpack box in the core lab specimen receiving area for shipment to <i>Dyna</i>LIFE 							

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

Policy Number: Date Approved: Page 4 of 5

Title: MIC36400-Referral of Category B Specimens to DL

Issuing Authority: Director of Health Services

Next Review Date:

Type: Laboratory Services Program SOP
Policy Number:
Date Approved:

REFERENCES:

1. DynaLIFE DX Referred-In Isolate Submission Form (LTR47241), revision 3.00

APPROVA	AL:		
Date			

REVISION HISTORY:

REVISION	DATE	Description of Change	REQUESTED BY
1.0	28 Apr 17	Initial Release	L. Steven
2.0	25 Feb 19	Procedure reviewed	L. Steven
3.0	30 Apr 21	Procedure reviewed and added to NTHSSA policy template	L. Steven
	_		

ref

Disclaimer Message: This is a **CONTROLLED** document for internal use only. Any documents appearing in paper form are not controlled and should be checked against the electronic file version prior to use.

Policy Number: Date Approved: Page 5 of 5