



Stanton Territorial Hospital  
P.O. Box 10, 550 Byrne Road  
YELLOWKNIFE NT X1A 2N1

**Document Name:**  
Dispensette Sterility Job Aid

**Distribution:**  
Microbiology Quality Control Manual

**Document Number:**  
MIC60051

**Date Issued:**  
March 19, 2021

**Uncontrolled When Printed**

Order standing order in LIS:

- Patient Last Name: **TEST**
- Patient First Name: **MICRO STERILITY**

- Enter **DUMD** in the Req. by box and order the test **CXENV**
- Collect, receive and plate order
- In Current Antibiotic Therapy select NA for not applicable
- In Micro OE comment, state source of saline-VITEK or GAS Dispensette

Label 2 THIO broths with  
specimen labels

Dispense 3mL of saline from  
each dispensette into their  
corresponding THIO broth

Incubate the broths in the  
Dispensette Sterility rack in the  
O<sub>2</sub> incubator

Read broth daily for 3 days

THIO broth is cloudy  
(indicates growth)

Record: "Cloudy-  
AUTOCLAVE  
dispenser ^CLAVE"  
in LIS

Report: "Cloudy-  
AUTOCLAVE  
dispenser"  
and finalize

Discard saline bottle  
and autoclave  
dispensette.

THIO broth is clear  
(indicates no  
growth)

Record: "No  
Growth" in LIS daily  
for 3 days

Report: "No Growth  
After 3 Days"  
and finalize

**NOTE:** This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC60051-Dispensette Sterility Test Job Aid

Print Date: 4/17/2025 6:20 PM