



Stanton Territorial Hospital
P.O. Box 10, 550 Byrne Road
YELLOWKNIFE NT X1A 2N1

Document Name:
Dispensette Sterility Job Aid

Distribution:
Microbiology Quality Control Manual

Document Number:
MIC60051

Date Issued:
March 19, 2021

Uncontrolled When Printed

Order standing order in LIS:

- Patient Last Name: **TEST**
- Patient First Name: **MICRO STERILITY**

- Enter **DUMD** in the Req. by box and order the test **CXENV**
- Collect, receive and plate order
- In Current Antibiotic Therapy select NA for not applicable
- In Micro OE comment, state source of saline-VITEK or GAS Dispensette

Label 2 THIO broths with
specimen labels

Dispense 3mL of saline from
each dispensette into their
corresponding THIO broth

Incubate the broths in the
Dispensette Sterility rack in the
O₂ incubator

Read broth daily for 3 days

THIO broth is cloudy
(indicates growth)

Record: "Cloudy-
AUTOCLAVE
dispenser ^CLAVE"
in LIS

Report: "Cloudy-
AUTOCLAVE
dispenser"
and finalize

Discard saline bottle
and autoclave
dispensette.

THIO broth is clear
(indicates no
growth)

Record: "No
Growth" in LIS daily
for 3 days

Report: "No Growth
After 3 Days"
and finalize

NOTE: This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC60051-Dispensette Sterility Test Job Aid

Print Date: 4/17/2025 6:20 PM