



Stanton Territorial Hospital
P.O. Box 10, 550 Byrne Road
YELLOWKNIFE NT X1A 2N1

Document Name:
Maintenance Record-VITEK 2

Distribution:
Microbiology Instrumentation Manual

Document Number:
MIC70110

Date Issued:
17 Nov 2017

Maintenance Record – VITEK 2

Month:

Year:

1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
---	---	---	---	---	---	---	---	---	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----

Daily Maintenance

Empty waste tray																														
Check status																														
Zero DENSICHEK																														
Check printer paper																														
Initials																														

Monthly Maintenance

Task	Date	Tech	Task	Date	Tech	Task	Date	Tech
Clean boats			Clean vacuum seal			Change saline bag		
Clean carousels			Clean vacuum chamber			Change pipette tips		
Clean cassettes			Clean optics			Optical Diagnostic Test		
Clean waste tray			Clean VITEK 2 exterior			Dispenser/Pipettor Diagnostic Test		
Clean base pan			Clean VITEK 2 computer					
Clean drip pan			Clean DENSICHEK					

VITEK DENSICHEK Calibration

Calibration	Reference Range	Lot #	Result	Date	Tech
0.0	0.00				
0.5	0.39 - 0.61				
2.0	1.81 - 2.19				
3.0	2.75 - 3.25				

Corrective action: _____ Reviewed by: _____

NOTE: This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC70110-Maintenance Record-VITEK 2

PRINT DATE: 5/15/2025 1:45:00 PM