

Document Name:

Suspect High Risk Organism Workup Job Aid

Distribution:

Microbiology Bacteriology Manual

Document Number: MIC40110

Date Issued:

November 5, 2024

WHEN TO SUSPECT HIGH BIOSAFETY RISK AGENTS

A. Presumptive diagnosis provided:

• **SUSPECT** relevant agents listed in this procedure

B. Gram stain results:

- Small, gram-negative bacilli or coccobacilli from sterile sites
- **SUSPECT** Brucella spp. and Francisella spp.

C. Gram stain results:

- Gram-negative diplococci from sterile sites
- SUSPECT Neisseria meningitidis

NOTE: *Neisseria meningitidis* is a Risk Group 2 organism, but given the potential for serious infection, culture should be treated like a Risk Group 3 organism

D. Culture results:

- Slow-growing, gram-negative bacilli/coccobacilli from all sites
- **SUSPECT** Brucella spp., Francisella spp., Yersinia pestis, Burkholderia pseuodomallei or Burkholderia mallei

E. Culture results:

- Rapid-growing, non-hemolytic colonies with ground-glass appearance often exhibiting comma-shaped protrusions from colony edge ("Medusa head" colonies)
- SUSPECT Bacillus anthracis

NOTE: This is a controlled document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC40110-Suspect High Risk Organism Workup Job Aid Print Date: 11/5/2024 2:01 PM