
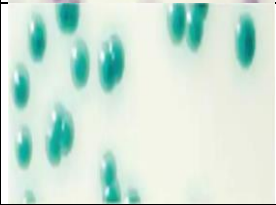

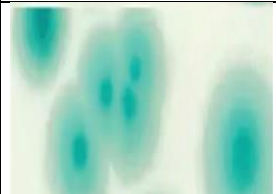
	<b>Stanton Territorial Hospital</b> P.O. Box 10, 550 Byrne Road YELLOWKNIFE NT X1A 2N1	<b>Document Name:</b> CandiSelect Agar Job Aid  <b>Distribution:</b> Microbiology Culture Manual	<b>Document Number:</b> MIC32210  <b>Date Issued:</b> March 31, 2025
---	--	--	--

Colour	Morphology	Picture	Suspected ID	Next steps
<b>Pink/ purple</b>	Appear as pink to purple colonies		<b><i>Candida albicans</i></b>	<ul style="list-style-type: none"> <li>Non-Sterile Site: No further testing</li> <li>Sterile Site: Perform wet prep → YSTID</li> </ul>
<b>Intense turquoise</b>	Appear as intense turquoise, spherical colonies with smooth morphotype		<b><i>Candida tropicalis</i></b>	<ul style="list-style-type: none"> <li>Perform wet prep → YSTID</li> </ul>
<b>Pale turquoise</b>	Appear as pale turquoise colonies that are flat and shiny with smooth morphotype, and they most commonly present a characteristic so-called "fish-eye" color pattern		<b><i>Candida glabrata</i></b>	<ul style="list-style-type: none"> <li>Perform wet prep → YSTID</li> </ul>
<b>Turquoise/ blue</b>	Appear as turquoise-blue colonies, with a rough morphotype, a dry appearance, and an irregular outline		<b><i>Candida krusei</i></b>	<ul style="list-style-type: none"> <li>Perform wet prep → YSTID</li> </ul>

**NOTE:** This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC32210-CandiSelect Agar Job Aid

PRINT DATE:3/22/2025 4:57 PM



**Stanton Territorial Hospital**  
P.O. Box 10, 550 Byrne Road  
YELLOWKNIFE NT X1A 2N1

**Document Name:**  
CandiSelect Agar Job Aid

**Distribution:**  
Microbiology Culture Manual

**Document Number:**  
MIC32210

**Date Issued:**  
March 31, 2025



**NOTE:** This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.

FILENAME: MIC32210-CandiSelect Agar Job Aid

PRINT DATE:3/22/2025 4:57 PM