



**Stanton Territorial Hospital**  
 P.O. Box 10, 550 Byrne Road  
 YELLOWKNIFE NT X1A 2N1

**Document Name:**  
 Microbiology Specimen Quality Comments Job Aid

**Distribution:**  
 Microbiology Specimen Processing Manual

**Document Number:**  
 MIC10130

**Date Issued:**  
 December 10, 2019

SPECIMEN TYPE	WHEN TO ADD	SPECIMEN QUALITY CODE
<ul style="list-style-type: none"> <li>Bacterial vaginosis screen</li> <li>Blood culture</li> <li>Body fluid culture</li> <li>Genital culture (upper/lower/IUD)</li> <li>Gonorrhoeae culture</li> <li><i>Trichomonas</i> rapid test</li> </ul>	Specimen >24 hours old	DELAY
<ul style="list-style-type: none"> <li>Ear culture and Eye culture</li> <li>Group B screen</li> <li>MRSA/MRO/VRE screen</li> <li>Nose culture</li> <li>Oral/Yeast culture</li> <li>Tip culture</li> <li>Throat culture</li> <li>Wound culture (deep/superficial)</li> </ul>	Specimen >48 hours old	DELAY
<ul style="list-style-type: none"> <li>Blue top or grey top urine</li> <li>Sputum</li> </ul>	Specimen >72 hours old	DELAY
Blood culture	Bottle received cold	COLD
Blood culture	Bottle frozen	FROZE
Blood culture	Bottle expired	BLDEX
Body fluid	Received on swab	SWBFL
Body fluid	Sample not spun	NOSPI
Specimen leaked	Specimen leaked	LEAK
Irretrievable specimen not labelled	Irretrievable specimen not labelled	IRR
GC culture	Swab for GC does not contain charcoal	NGON
GC culture	Vaginal swab received for <i>N. gonorrhoeae</i>	VAGGC
Bacterial vaginosis screen swab	BV swab > 5 days old	SLIDE
VRE	No fecal matter on VRE swab	VRE

**NOTE:** This is a CONTROLLED document for internal use only. Any documents appearing in paper form are not controlled and should be checked against electronic version prior to use.