

TRAINING UPDATE


Lab Location: SGAH & WAH
Department: Core

Date Distributed: 9/19/2013
Due Date: 10/1/2013
Implementation: 10/1/2013

DESCRIPTION OF PROCEDURE REVISION

Name of procedure:
ADVIA Centaur CP Maintenance Log AG.F182.002
Description of change(s):
<p>Log revised to add:</p> <ul style="list-style-type: none">• Prompt to verify 'Assay calibration dates checked'• Space to document that calibration was performed <p>This revised form will be implemented on October 1, 2013</p>

Document your compliance with this training update by taking the quiz in the MTS system.

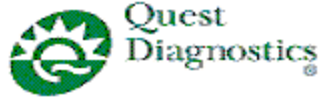

- Shady Grove Adventist Hospital
- Washington Adventist Hospital

ADVIA Centaur CP Maintenance Log (Page 1)

Month: _____

Year: _____


Instrument Serial Number: _____

	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Daily Tasks																																
Perform Automated Daily Cleaning																																
Aspirate Probe Bubble Detector Cal.																																
BNP QC Day Shift: Levels 1 & 3																																
BNP QC Eve Shift: Level 2																																
BNP QC Night Shift: Level 1 or 3																																
iPTH QC run as needed																																
Assay calibration dates checked																																
Tech																																
Calibration History:	BNP (every 28 days): <input type="checkbox"/> Done (check box if calibration is done) Tech: _____ Date: _____														iPTH (every 14 days): <input type="checkbox"/> Done (check box if calibration is done) Tech: _____ Date: _____ Tech: _____ Date: _____																	
Weekly Tasks																																
Clean Incubation Ring Cover																																
Clean exterior of reagent probe																																
Clean exterior of waste probe																																
Clean Probe Rails																																
Prepare a new CSC (cleaning solution concentrate)																																
Perform automated weekly Cleaning																																
Tech / Date																																

Monthly	Tech / Date
Maintain System Database	
Perform Automated Monthly Cleaning	

Bimonthly	Tech / Date
Clean Wash 1 container	

Weekly review:	Weekly review:	Weekly review:
Weekly review:	Weekly review:	Monthly review:

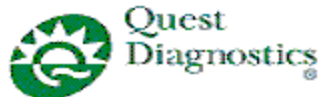

- Shady Grove Adventist Hospital
- Washington Adventist Hospital

ADVIA Centaur CP Maintenance Log (Page 2)

Month: _____

Year: _____

Instrument Serial Number: _____

As Needed Tasks	Reason for performing this task
Clean Acid / Base Compartment	
Clean Solid Waste Drawer and Liner	
Clean Sample Compartment	
Clean Sample Compartment Shutter	
Clean Sample Racks	
Clean Reagent Compartment	
Clean Reagent Compartment Shutter	
Clean System Exterior	
Clean WorkStation	
Data Base Maintenance	
Automated System Prime	
Tech	
Date	

Comments: _____

Weekly review:	Weekly review:	Weekly review:
Weekly review:	Weekly review:	Monthly review: