

## TRAINING UPDATE

**Lab Location:** SGAH and WAH      **Date Implemented:** 12.17.2013  
**Department:** Blood Bank      **Due Date:** 12.31.2013

### DESCRIPTION OF PROCEDURE REVISION

<b>Name of procedure:</b>
---------------------------

Immucor Manual Capture Equipment Maintenance Record
---

<b>Description of change(s):</b>
----------------------------------

Added lines at the bottom of the form to document in use lots of PBS and DIH <sub>2</sub> O.
--

- While updating the supply receipt log, I noted that SGAH and WAH were tracking new lots of saline differently. Saline is difficult to track, because we don't necessarily unpack it ourselves.
- To make the process easier, we will document saline in use on the equipment logs (Echo PBS log, cell washer log, and now manual capture log).

Month/Year:

Shady Grove Adventist Hospital, Rockville, Maryland  
 Washington Adventist Hospital, Takoma Park, Maryland

### Immucor Manual Capture Equipment Maintenance Record

#### Immucor CSW 100 Maintenance Record

Serial Number:																																
<b>Daily</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Prime to remove air bubbles																																
Check dispense and aspirate needles																																
Inspect tubing and manifold needles																																
Residual saline verification check (4-8 µL)																																
Fill PBS and DI water reservoirs																																
Empty waste container																																
<b>Weekly</b>																																
Backflush Reservoir Filters																																
<b>Monthly</b>																																
Replace exhaust filters (cotton balls)																																
Verification of fill check (21.6 - 26.4 mL)																																
<b>As Needed</b>																																
Clean manifold needles																																
Replace manifold needles																																
Flush and/or stylus manifold																																
Change tubing																																
Decontaminate																																

#### Immucor Incubator P2 Maintenance Record

Serial Number:																																
<b>Daily</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Clean the incubator with isopropyl alcohol																																
Verify the incubator temperature (36-38°C):																																
Left Incubator Temperature																																
Right Incubator Temperature																																
Serial Number of Temperature Strip																																
Expiration Date of Temperature Strip																																
Perform Timer Verification Semi-Annually; Document on Timer Calibration Record Form. Due Date:																																
	Date Added															Lot Number															Expiration Date	