

TRAINING UPDATE

Lab Location: SGAH & WAH
Department: Micro

Date Distributed: 1/5/2015
Due Date: 1/31/2015
Implementation: 1/5/2015

DESCRIPTION OF REVISION

Name of procedure:
Plating Chart for Media by Source and Test Code AG.F191.2
Description of change(s):
<p>Change to group B strep cultures - Due to a new Best Practice Team SOP we will no longer be using LIM broth for these. Send swab to Chantilly. No plating required.</p> <p>This FORM was implemented on January 5, 2015</p>

Document your compliance with this training update by taking the quiz in the MTS system.



- Germantown Emergency Center
- Shady Grove Adventist Hospital
- Washington Adventist Hospital

PLATING CHART FOR MEDIA BY SOURCE AND TEST CODE

Culture	Test Code	Blood	MAC	CHOC	Martin/ Lewis	ParaPak C&S vial	CNA	Ana Blood	Thio	Gram Stain	V agar	Misc	Send specimen to Chantilly
Stool Culture, Enteric Profile X <i>Salmonella, Shigella,</i> <i>Campylobacter,</i> <i>Aeromonas,</i> <i>Plesiomonas,</i> <i>Yersinia & Vibrio</i>	15292					X							Yes
Stool, <i>E. coli</i> O157	4221					X							Yes
Respiratory Culture	769	X	X	X			X						Yes
Eye Culture	76950	X	X	X			X						Yes
Ear Culture	76951	X	X	X			X						Yes
Throat Culture	5870	X		X									Yes
Group A Strep Only Culture	6470	X											Yes
Genital Culture	778		X	X			X				X		Yes
GBS Screen	14537											Send swab to CHY	Yes
GC Screen	657				X								Yes
Urine Culture	775	X	X									0.001 mL loop	Yes
Urine, sterile source	2983	X (0.01 + 0.001)	X (0.01 + 0.001)									0.001 mL and 0.01 mL loops	Yes
CSF Culture	127350	X	X	X			X		X	X			NO
Sterile body Fluid Culture	1273	X	X	X			X		X	X			NO
Wound Culture	783 / 78351	X	X	X			X						Yes
Tissue Culture	78350	X	X	X			X		X	X			NO
Surgical Culture	78353	X	X	X			X		X				NO
Prosthetic Joint Culture	78355	X	X	X			X		X				NO
Anaerobic Culture	15871	X						X				ana BBE/LKV biplate	depends on source
VRE Screen	8557						X						Yes
MDRO Screen	78354		X									KPC Chromagar	Yes
MRSA Screen	MRSAS											Spectra	NO
<i>Staph. aureus</i> Nasal Screen	900942	X					X						Yes
IV Catheter tip	78352	X											NO
Type 1 Lab water	4120											1 mL Standard Methods Agar	Yes



- Germantown Emergency Center
- Shady Grove Adventist Hospital
- Washington Adventist Hospital

Culture	Test Code	Blood	MAC	CHOC	Martin/ Lewis	ParaPak C&S vial	CNA	Ana Blood	Thio	Gram Stain	V agar	Misc	Send specimen to Chantilly
Environmental Culture Dialysis water /Dialysate Physical / Resp Ther Pharmacy glove tips Mold settle plate Bacterial settle plate Pharmacy hood surface Infection Control	6320											1 mL on T-soy agar Swab in T-soy broth TSA plate SAB TSA plate Swab in T-soy broth Varies with sample, Contact microbiologist	Yes, if available
Positive Blood Culture	18100 AER 18101 ANA	X X	X X	X X			X X	 X				SAB if yeast seen on Gram stain	NO