

## TRAINING UPDATE

**Lab Location:** GEC  
**Department:** Core

**Date Distributed:** 7/23/2015  
**Due Date:** 8/1/2015  
**Implementation:** 8/1/2015

### DESCRIPTION OF REVISION

<b>Name of procedure:</b>
<b>Dimension Xpand Maintenance Log AG.F179.2</b>
<b>Description of change(s):</b>
<p><b>Remove line to document replacing chlorine tablet (this info is captured on the Millipore log)</b></p> <p><b>This FORM will be implemented on August 1, 2015</b></p>

Document your compliance with this training update by taking the quiz in the MTS system.

## DIMENSION XPAND MAINTENANCE LOG

S/N \_\_\_\_\_

MONTH/YEAR \_\_\_\_\_

		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
		DAILY																															
	Verify Cuvette Temp. (36.8° - 37.2° C)																																
	Verify Reagent Temp. (2° - 8° C)																																
	Verify HM Temp. (42° - 44° C)																																
	Verify System Check																																
	Cut Waste Film																																
	Record Millipore Readings																																
	Daily QC Performed and Verified By / Include Initials																																
WEEKLY		Date and Initials						Date and Initials						Date and Initials						Date and Initials													
	Clean Outside of R2 and HM Wash Probes																																
	Clean Windows																																
MONTHLY		Date and Initials					Date and Initials					Date and Initials					Date and Initials					Date and Initials											
	Replace IMT Sensor Run Dilution Check & Condition Sensor (LOT#)																																
MONTHLY		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
	IMT System Clean																																
	Replace IMT Pump Tubing																																
	Replace / Clean Air Filters																																
	Replace HM Pump Heads																																
	Stylette HM Wash Probes																																
AS NEEDED																																	
	Clean Sample Probe & Drain																																
	Replace Sample Probe*																																
	Replace R1 Probe*																																
	Replace R2 Probe*																																
	Change Source Lamp																																
	Replace Chlorine Tablet																																
	Verify Calibration for the following:																																
	<b>CRP (QC ± 1.5sd)</b>																																

\* Replace before or at 30,000 cycles.

Weekly review:	Weekly review:	Weekly review:
Weekly review:	Weekly review:	Monthly review: