

TRAINING UPDATE

Lab Location: GEC
Department: Core

Date Distributed: 11/13/2015
Due Date: 11/30/2015
Implementation: 12/1/2015

DESCRIPTION OF PROCEDURE REVISION

Name of procedure:

Tip Sheet for Pochi-100 AG.F336.0

Description of change(s):

New form with instructions for resulting patients and QC from Pochi

This FORM will be implemented on December 1, 2015

Document your compliance with this training update by taking the quiz in the MTS system.

Tip Sheet for Pochi-100

Resulting Patients in Sunquest

Function: **MEM**

Worksheet: **GHMAX**

Test: **CBCND**

ACCEPT (A) OR MODIFY (M): **M**

WBC: **HMXG**
GP100

Change the method code on each analyte to **GP100** with the exception of RDW and MPV. Skip past these using the enter key.

To result patients:

Enter the ACCN number and enter the patient results.
For RDW and MPV type **HIDE**, otherwise these will stay pending.

Because you are performing manual entry, before accepting modify and recheck all results for clerical errors.

Resulting QC in Unity Real Time

QC must be entered in Unity Real Time. The three (3) levels of Sysmex poCH-100i controls (Low, Normal and High) are under the folder GEC Xpand 1. Enter QC values accordingly.