

Laboratories at Adventist Healthcare Washington Adventist Hospital, Shady Grove Medical Center and Germantown Emergency Center

Date: February 10, 2016

INTERNAL LABORATORY ALERT

Subject: Liquicheck Specialty Immunoassay Stability Change for PTH

The Laboratory is informing you of the following test change:

Effective date:	February 10, 2016
Test Names / Codes:	Quality Control: Bio-Rad Liquichek Specialty Immunoassay Control Test Name: Intact PTH, Intra Operative IPTH Test Code: ITPTH, IOIPTH Testing Facility: SGMC & WAH Centaurs Reference Range: 11.0 – 80.0 pg/mL
Control Storage & Stability :	 Unopened Quality Control stored at -20 to -70°C is stable until the expiration date. Thawed Opened and Unopened Quality Control stored at 2 to 8°C will only be stable for 7 days.
Contacts:	Rob SanLuis, Zanetta Morrow and Julie Negado

G:\AHC_Lab\Clinical Folders\Communication_General Lab\Lab Alerts