

TRAINING UPDATE

Lab Location: SGMC & WAH
Department: Core

Date Distributed: 2/2/2017
Due Date: 2/28/2017
Implementation: 3/1/2017

DESCRIPTION OF REVISION

Name of procedure:
Dimension Vista Maintenance Log AG.F246.2
Description of change(s):
Removed “Run System Check” under Weekly maintenance (<i>SOP revised last year to delete performing this task but log was not updated</i>) Updated QD logo and SG facility name This revised FORM will be implemented on March 1, 2017

Document your compliance with this training update by taking the quiz in the MTS system.

Dimension Vista Maintenance Log

Month: _____

Year: _____

Instrument Serial Number: _____

	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Daily																															
Modules Temperature: within acceptable range Yes (Y) or No (N)*																															
Probe Test all Vistas: Sample Probe 1, IMT Probe, Reagent Arms 1 & 2, Reagent Probe, Aliquotter and Aliquotter Purge & Vent Valves Pass (P) or Fail (F)*																															
Probe Test (continued) Vista 3 (SGMC) only: Sample Probes 2 & 3, Reagent Arms 3, 4 & 5 Pass (P) or Fail (F)*																															
Clean Sample & STAT Lane area Disinfect the Bench (√)																															
Tech Initial																															

* Document the correction action that was taken to resolve the issue

Weekly

Clean Base Plate (√)					
Inspect IMT Sample Port (√)					
Tech Initial / Date					

Monthly

Clean Sample/Reagent Drain	Tech Initial / Date
Replace Air Filters	
Clean Waste Chute	
Inspect Peristaltic Pump Tubing	
Clean Flex Inserts	
Clean Aliquot Probe Tip	

Comments:

Weekly review:	Weekly review:	Weekly review:
Weekly review:	Weekly review:	Monthly review: