

## TRAINING UPDATE

**Lab Location:** GEC, SGMC & WAH  
**Department:** Core Lab

**Date Distributed:** 4/16/2018  
**Due Date:** 5/8/2018  
**Implementation:** 5/8/2018

### DESCRIPTION OF PROCEDURE

<b>Name of procedure:</b>
<b>Printing Patient Reports from Laboratory Instruments SGAH.L991 v0</b>
<b>Description:</b>
<p>This is a new SOP that provides detailed steps for how to ‘turn on’ patient report printing from the core lab instruments (often used during downtime).</p> <p>Once implemented, the SOP will be kept in the Group Lead/Tech in Charge manual.</p> <p><b>This SOP will be implemented on May 8, 2018</b></p>

Document your compliance with this training update by taking the quiz in the MTS system.

Non-Technical SOP

<b>Title</b>	<b>Printing Patient Reports from Laboratory Instruments</b>	
<b>Prepared by</b>	Ash Chini, Leslie Barrett	Date: 4/10/2018
<b>Owner</b>	Robert SanLuis	Date: 4/10/2018

<b>Laboratory Approval</b>		
<b>Print Name and Title</b>	<b>Signature</b>	<b>Date</b>
<i>Refer to the electronic signature page for approval and approval dates.</i>		
Local Issue Date:		Local Effective Date:

<b>Review:</b>		
<b>Print Name</b>	<b>Signature</b>	<b>Date</b>

## **TABLE OF CONTENTS**

1. PURPOSE.....	2
2. SCOPE.....	2
3. RESPONSIBILITY.....	2
4. DEFINITIONS.....	2
5. PROCEDURE.....	2
6. RELATED DOCUMENTS .....	3
7. REFERENCES .....	3
8. REVISION HISTORY.....	3
9. ADDENDA AND APPENDICES .....	3

---

### **1. PURPOSE**

This procedure provides instructions to re-establish patient report printing function for laboratory instruments.

### **2. SCOPE**

This procedure may be used during computer down time or whenever patient reports need to be generated from the instrument(s).

### **3. RESPONSIBILITY**

All core lab staff may perform this procedure.

### **4. DEFINITIONS**

None

### **5. PROCEDURE**

#### **A. Stago**

1. From the Home Page – Escape – Status – On-line Print Out
2. When “**On-line Print Out**” is set to **Yes**, it means the instrument will print every patient result automatically. If it is set to **No**, it means it will not print patient results.

#### **B. Centaur CP**

1. From the Home Page – Set up – System – Run Time Report
2. When “**Run Time Report**” is set to **ON**, the instrument will print every patient result automatically. If it is set to **OFF**, it will not print patient results.

#### **C. IRIS**

1. From the Home Page – Setting – Release
2. When the **Printer has a check mark** next to it, the instrument will print every patient result automatically. If there is **no check mark**, it will not print patient results.

**D. Vista**

1. From the Home Page – Advanced – Configuration – System Configuration – Automatically Print Test Results
2. A **Yes** means the instrument will print every patient result automatically; a **No** means it will not print patient results.

**E. Sysmex**

1. From the Home Page – IPU Setting – Auto Output – The box in **front of GP, under Negative Data**
2. To print all (positive and negative patients), check the box. To print only positive patients, uncheck the box.

**F. Xpand**

1. From the Home Page – System Configuration – Select Printer – System Printer
2. To print all patient reports and every QC run:
  - System Printer “**On Line – Print all**”
  - Axillary Printer to “**OFF Line**”
3. To turn off patient reports and QC run:
  - System Printer to “**On Line No Test Report**”
  - Axillary Printer to “**OFF Line**”

**6. RELATED DOCUMENTS**

N/A

**7. REFERENCES**

None

**8. REVISION HISTORY**

<b>Version</b>	<b>Date</b>	<b>Reason for Revision</b>	<b>Revised By</b>	<b>Approved By</b>

**9. ADDENDA AND APPENDICES**

None