

## TRAINING UPDATE

**Lab Location:** SGMC & WAH  
**Department:** Core Lab

**Date Distributed:** 9/27/2018  
**Due Date:** 10/27/2018  
**Implementation:** 10/22/2018

### DESCRIPTION OF REVISION

<b>Name of procedure:</b>
<b>Plating Chart for Media by Source and Test Code AG.F191 v3</b>
<b>Description of change(s):</b>
<p><i>Changes made to make Form match practice</i></p> <p>Removed MRSA culture and a few environmental cultures we no longer receive</p> <p>Changed respiratory to not send raw specimen to Chantilly (requested by CHY)</p> <p><b>This revised Form will be implemented on October 22, 2018</b></p>

Document your compliance with this training update by taking the quiz in the MTS system.

### PLATING CHART FOR MEDIA BY SOURCE AND TEST CODE

Culture	Test Code	Blood	MAC	CHOC	Martin/ Lewis	ParaPak C&S vial	CNA	Ana Blood	Thio	Gram Stain	V agar	Misc	Send specimen to Chantilly
Stool Culture, Enteric Profile X <i>Salmonella, Shigella,</i> <i>Campylobacter,</i> <i>Aeromonas,</i> <i>Plesiomonas,</i> <i>Yersinia &amp; Vibrio</i>	15292					X							Yes
Stool, <i>E. coli</i> O157	4221					X							Yes
Respiratory Culture	769	X	X	X			X						NO
Eye Culture	76950	X	X	X			X						Yes
Ear Culture	76951	X	X	X			X						Yes
Throat Culture	5870	X		X									Yes
Group A Strep Only Culture	6470	X											Yes
Genital Culture	778		X	X			X				X		Yes
GBS Screen	14537											Send swab to CHY	Yes
GC Screen	657				X								Yes
Urine Culture	775	X	X									0.001 mL loop	Yes
Urine, sterile source	2983	X (0.01 + 0.001)	X (0.01 + 0.001)									0.001 mL and 0.01 mL loops	Yes
CSF Culture	127350	X	X	X			X		X	X			NO
Sterile body Fluid Culture	1273	X	X	X			X		X	X			NO
Wound Culture	783 / 78351	X	X	X			X						Yes
Tissue Culture	78350	X	X	X			X		X	X			NO
Surgical Culture	78353	X	X	X			X		X				NO
Prosthetic Joint Culture	78355	X	X	X			X		X				NO
Anaerobic Culture	15871	X						X				ana BBE/LKV biplate	depends on source
VRE Screen	8557						X						Yes
MDRO Screen	78354		X									Chromagar CRE	Yes
<i>Staph. aureus</i> Nasal Screen	900942	X					X						Yes
IV Catheter tip	78352	X											NO
Type 1 Lab water	4120											1 mL Standard Methods Agar	Yes

- Germantown Emergency Center
- Shady Grove Medical Center
- Washington Adventist Hospital

Culture	Test Code	Blood	MAC	CHOC	Martin/Lewis	ParaPak C&S vial	CNA	Ana Blood	Thio	Gram Stain	V agar	Misc	Send specimen to Chantilly
Environmental Culture	6320												Yes, if available
Pharmacy glove tips												TSA plate	
Mold settle plate												SAB	
Bacterial settle plate												TSA plate	
Pharmacy hood surface												Swab in T-soy broth	
Infection Control												Varies with sample, Contact microbiologist	
Positive Blood Culture	18100 AER 18101 ANA	X X	X X	X X			X X	X X				SAB if yeast seen on Gram stain	NO