

Laboratories at Adventist Healthcare White Oak Medical Center, Shady Grove Medical Center and Germantown Emergency Center

Date: October 15, 2020

LABORATORY ALERT

Subject: High Sensitivity Troponin I

The Laboratory is informing you of the following test change:

Effective date:	November 12, 2020	
Test Change:	Current Siemens Troponin I has been discontinued by the manufacturer. Siemens is standardizing to a High Sensitivity Troponin I (TNIH). The new TNIH will be reported in pg/mL in whole numbers.	
Test Name:	No change in ordering required. (Troponin or Troponin I)	
Specimen type:	No change in specimen type. (Mint Green Top: Lithium Heparin Plasma)	
Comparison:	"Current" Troponin I (CTNI)	"New" High Sensitivity Troponin I (TNIH)
Expected Results:	Reference Range: <0.05 ng/mL 0.051-0.089 ng/mL H (High) >0.09 ng/mL cH (Critical High)	Reference Range: <83 pg/mL 83-100 pg/mL H (High) >100 pg/mL cH (Critical High)
Critical Value Calls:	First Critical > 0.09 ng/mL Subsequent Critical Results Prior Critical Delta Threshold 0.10 - 1.00 ng/mL 10X prior value 1.01 - 20.00 ng/mL Increase of 5.00 20.01 ng/mL - more Increase of 25.00	First Critical > 100 pg/mL Subsequent Critical Results Prior Critical Delta Threshold 101 - 500 pg/mL Value Doubles 501 - 1000 pg/mL Increase of 250 1001 pg/mL - more Increase of 1,000
Additional information:	Any physician that would like to see comparison patient data it will be provided at your request. The spreadsheet has multiple tabs: Lab Alert, Critical value changes Combined cutoff data, Male cutoff data, Female cutoff data, Serial Collections. Please email rsanluis@adventisthealthcare.com .	
Contacts:	Rob SanLuis, System Lab Director 240-620-3413 Dr. Cacciabeve, Medical Director	