

TRAINING UPDATE

Lab Location: SGMC & WOMC
Department: Core Lab

Date Distributed: 4/2/2021
Due Date: 4/30/2021

DESCRIPTION OF PROCEDURE REVISION

Name of procedure:

OSMO (urine / serum) Patient Log AG.F134.6

OSMO Calibration Log AG.F110.4

Description of change(s):

OSMO (urine / serum) Patient Log -

Removed requirement to record QC lot and expiration info
(already logged into Unity so no need to duplicate)

OSMO Calibration Log -

Changed the review boxes at bottom from 'weekly' to 'monthly':

- There is no need to record the 290 Standard on this form that is part of a patient testing run *(it's already documented on the OSMO Patient log)*
- This form is only used when CALIBRATION is performed and that is only required quarterly or for other special circumstances

These revised forms will be implemented on April 13, 2021

Document your compliance with this training update by taking the quiz in the MTS system.

Urine / Serum OSMO Patient Log

Note: Remain at the analyzer throughout the testing process.

Tech Code _____ Date _____		Record Results			
290 Standard (290 +/-2) <i>Perform in duplicate once per shift</i>	Lot #: Exp. Date:				
Must be run in this order*		Serum		Urine	
Control Level 1	<i>Lot # & exp. are recorded in Unity</i>				
Patient Name / MR#					
Patient Name / MR#					
Patient Name / MR#					
Control Level 2	<i>Lot # & exp. are recorded in Unity</i>				

* Bracket QC with each patient run. Run size is limited to 3 specimens. If more than 3 samples are to be tested, bracket each group of 3 between QC levels

Tech Code _____ Date _____		Record Results			
290 Standard (290 +/-2) <i>Perform in duplicate once per shift</i>	Lot #: Exp. Date:				
Must be run in this order*		Serum		Urine	
Control Level 1	<i>Lot # & exp. are recorded in Unity</i>				
Patient Name / MR#					
Patient Name / MR#					
Patient Name / MR#					
Control Level 2	<i>Lot # & exp. are recorded in Unity</i>				

Weekly review:	Weekly review:	Weekly review:
Weekly review:	Weekly review:	Monthly review:



- Germantown Emergency Center
- Shady Grove Medical Center
- White Oak Medical Center

OSMO CALIBRATION LOG

Date	290 STD (+/- 2)				Calibration/Recalibration				Corrective Action / STD/CAL Accepted <input checked="" type="checkbox"/>	Tech Code
	Lot	Exp.	1 st Run	2 nd Run	50 STD Lot/Exp.	850 STD Lot/Exp.	2000 STD Lot/Exp.	Calibration Code		
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	
					/	/	/		STD/CAL Accepted <input type="checkbox"/>	

Calibration Codes: 1-New Lot 2-Instrument Related 3-QC Problem 4-Calibration Due

Monthly review:	Monthly review:	Monthly review:
Monthly review:	Monthly review:	Monthly review: