


# Test Results

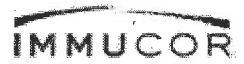

White Oak Medical Center - 11890 Healing Way Silver Spring, MD 20904

Instrument ID: M20432 Instrument name: ECOWO

Report generated: Thursday, November 14, 2024 10:18 AM

Test name	Group Screen	Software version	2.2.0.39
Operator	mantar	Assay version	2.0U
Batch number	46165	Instrument ID	M20432
Profile name	Group Screen	Instrument name	ECOWO



## Results - Summary

Sample ID	Group	Screen
WB corQC 1	A Pos	Positive
WB corQC 2	B Neg	Positive
WB corQC 3	O Pos	Negative

## Reagent Receipt QC

Reagent	Anti-A
Lot #	1111
Expiration	12/31/2025
Date	11/15/24 Tech 129

## Results - Group

Sample ID	Interp.	Flags	Mono Ctrl	Anti-A	Anti-B	Anti-D1	Anti-D2	A1 Cells	B Cells
232333245208415598 WB corQC 1	A Pos		0	3+	0	3+	3+	0	4+
233333245208624606 WB corQC 2	B Neg		0	0	3+	0	0	4+	0
234333245208867527 WB corQC 3	O Pos		0	0	0	3+	3+	4+	4+

## Results - Screen

Sample ID	Interp.	Flags	Screen 1	Screen 2	Screen 3	Pos Ctrl
232333245208415598 WB corQC 1	Positive		0	3+	3+	4+
233333245208624606 WB corQC 2	Positive		3+	3+	0	4+
234333245208867527 WB corQC 3	Negative		0	0	0	3+

## Supplies